



Alcohol Testing Program

INSTRUMENT PROCESSING SHEET

Agency Collier County Sheriff's Office S/N 80-001201

Date In 10/13/2016 Date Out 10/14/2016 Ship P/U H/D CMI EE

Intake Performed By PEARL

Registration
 Annual
 Return from CMI
 Return from Enforcement
 Electronics
 Other _____

Visual Inspection:
OK Case OK Handle
OK Dry Gas Holder OK Feet
OK Keyboard/Plug OK Back/Plugs
OK Screws tight OK Breath Hose

Other Equipment:
 Power cord
 Printer Cable
 Other ANTI STATIC BAG

Notes: _____

Quality Checks Performed By DEER

Breath Tube Screen
 Replace O-Rings
 Instrument Set Up Verified
 R-Value 215
 Flow Verification (L/s)
 Flow Column # ATP 101
 32mm 152 (.139 - .169)
 36mm 175 (.156 - .190)
 53mm 242 (.228 - .278)
 103mm 503 (.447 - .547)

Barometric Pressure Check
 Gauge ID # 281663
 Stability Checks

Simulator	Serial #	Lot #/Exp
0.05	<u>SD3967</u>	<u>201507A</u> <u>02/14/2017</u>
0.08	<u>SD3968</u>	<u>201601F</u> <u>01/26/2018</u>
0.20	<u>SD3969</u>	<u>201505A</u> <u>05/12/2017</u>
0.08 DGS	N/A	<u>A6600504</u> <u>01/05/2018</u>

Flow Calibration Performed By _____

Flow Calibration N/A
 Flow Calibration Complete
 Flow Column # _____
 5L/min - 17mm
 15L/min - 43mm
 30L/min - 103mm
 R-Value _____
 Post Calibration Verification (L/s)
 Flow Column # _____
 32mm _____ (.139 - .169)
 36mm _____ (.156 - .190)
 53mm _____ (.228 - .278)
 103mm _____ (.447 - .547)

Maintenance Performed By _____

Battery Replacement
 Dry Gas Regulator Replacement
 Breath Tube Replacement
 Other _____

Suggested Service

RECEIVED
OCT 27 2016
FDLE
Alcohol Testing Program

Optical Bench Calibration Performed By _____

Optical Bench Calibration N/A
 Optical Bench Calibration Complete
 Barometric Pressure Gauge ID # _____

Simulator	Serial Number	Lot Number	Expiration
0.000		N/A	N/A
0.040			
0.100			
0.200			
0.400			
0.080 DGS	N/A		

Post Calibration Stability Checks

Simulator	Serial Number	Lot Number	Expiration
0.05			
0.08			
0.20			
0.08 DGS	N/A		

Department Inspection Performed By DEER

Barometric Pressure Gauge ID# 28199 1017 Gauge
1018 Instrument

Mouth Alcohol Solution Lot # 2016 A
 Acetone Stock Solution Lot # 2016 B

Simulator	Serial Number
0.00	<u>SD 3965</u>
Interferent	<u>SD 3966</u>
0.05	<u>SD 3967</u>
0.08	<u>SD 3968</u>
0.20	<u>SD 3969</u>

Attachments

Form 41
 Pre-Stability Tests
 Flow Calibration
 Optical Bench Cal
 Post-Stability Tests
 Other _____

Notes: **E-MAILED** **APPROVED**
10/14/2016
DC/DR BIC FROM 10/26/16

Instrument Complies with Chapter 11D-8, FAC
 Instrument Does Not Comply with Chapter 11D-8, FAC
 Return to/Place into Evidentiary Use
 Remain Out of Evidentiary Use
 Conduct an Agency Inspection Before Evidentiary Use

Brett Kirkland
Quality Control Review

10/27/16
Date

TYPE OF TEST	SERIAL NUMBER	AGENCY	DATE	PERFORMED BY
Stabilities	80-001201	Collier County Sheriff's Office	10/14/2016	<i>Bill</i>

JK

0.05g/210L	0.08g/210L	0.20g/210L	DGS 0.08g/210L
<p>0.047 to 0.053 <input checked="" type="checkbox"/></p> <p>COLLIER COUNTY SC Intoxillizer - Alcohol Analyzer Model: 8000 SN 80-001201 10/14/2016 Software: 8100.27</p> <p>Test g/210L Time</p> <p>Rip Blank 0.000 08:08 Control Test 0.051 08:09 Rip Blank 0.000 08:09 Control Test 0.051 08:13 Rip Blank 0.000 08:10 Control Test 0.051 08:11 Rip Blank 0.000 08:12 Control Test Stats Average 0.0510 Std Dev 0.0000 Rel Std Dev(%) 0.0000</p> <p><i>Bill</i> Operator's Signature</p>	<p>0.077 to 0.083 <input checked="" type="checkbox"/></p> <p>COLLIER COUNTY SC Intoxillizer - Alcohol Analyzer Model: 8000 SN 80-001201 10/14/2016 Software: 8100.27</p> <p>Test g/210L Time</p> <p>Rip Blank 0.000 08:13 Control Test 0.081 08:13 Rip Blank 0.000 08:14 Control Test 0.081 08:14 Rip Blank 0.000 08:15 Control Test 0.081 08:16 Rip Blank 0.000 08:16 Control Test Stats Average 0.0810 Std Dev 0.0000 Rel Std Dev(%) 0.0000</p> <p><i>Bill</i> Operator's Signature</p>	<p>0.194 to 0.206 <input checked="" type="checkbox"/></p> <p>COLLIER COUNTY SC Intoxillizer - Alcohol Analyzer Model: 8000 SN 80-001201 10/14/2016 Software: 8100.27</p> <p>Test g/210L Time</p> <p>Rip Blank 0.000 08:17 Control Test 0.203 08:18 Rip Blank 0.000 08:19 Control Test 0.204 08:19 Rip Blank 0.000 08:20 Control Test 0.204 08:21 Rip Blank 0.000 08:21 Control Test Stats Average 0.2037 Std Dev 0.0006 Rel Std Dev(%) 0.2835</p> <p><i>Bill</i> Operator's Signature</p>	<p>0.077 to 0.083 <input checked="" type="checkbox"/></p> <p>COLLIER COUNTY SC Intoxillizer - Alcohol Analyzer Model: 8000 SN 80-001201 10/14/2016 Software: 8100.27</p> <p>Test g/210L Time</p> <p>Rip Blank 0.000 08:22 Control Test 0.079 08:23 Rip Blank 0.000 08:23 Control Test 0.080 08:23 Rip Blank 0.000 08:24 Control Test 0.080 08:24 Rip Blank 0.000 08:25 Control Test Stats Average 0.0797 Std Dev 0.0005 Rel Std Dev(%) 0.7247</p> <p><i>Bill</i> Operator's Signature</p>

OPM

AUG 15 2016



Alcohol Testing Program

INSTRUMENT PROCESSING SHEET

Agency Collier County Sheriff's Office S/N 80-001201

Date In 8/8/2016 Date Out 8/8/2016

Ship P/U H/D CMI **FDLE**

Alcohol Testing Program

Intake Performed By DELL

Registration
 Annual
 Return from CMI
 Return from Enforcement Electronics
 Other _____

Visual Inspection:
OK Case OK Handle
OK Dry Gas Holder OK Feet
OK Keyboard/Plug OK Back/Plugs
OK Screws tight OK Breath Hose

Other Equipment:
 Power cord
 Printer Cable
 Other _____

Notes: _____

Quality Checks Performed By DELL

Breath Tube Screen
 Replace O-Rings
 Instrument Set Up Verified
 R-Value 214
 Flow Verification (L/s)
 Flow Column # ATP 101
 32mm 148 (.139 - .169)
 36mm 171 (.156 - .190)
 53mm 238 (.228 - .278)
 103mm 503 (.447 - .547)

Barometric Pressure Check
 Gauge ID # 28663

Stability Checks

Simulator	Serial #	Lot #/Exp
0.05	SD 3963	201507A 07/14/2017
0.08	SD 3968	201601F 01/26/2018
0.20	SD 3969	201505A 05/12/2017
0.08 DGS	N/A	A6600504 01/05/2018

Flow Calibration Performed By _____

Flow Calibration N/A
 Flow Calibration Complete
 Flow Column # _____
 5L/min - 17mm
 15L/min - 53mm
 30L/min - 103mm

R-Value _____
 Post Calibration Verification (L/s)
 Flow Column # _____
 32mm _____ (.139 - .169)
 36mm _____ (.156 - .190)
 53mm _____ (.228 - .278)
 103mm _____ (.447 - .547)

Maintenance Performed By _____

Battery Replacement
 Dry Gas Regulator Replacement
 Breath Tube Replacement
 Other _____

Suggested Service

Optical Bench Calibration Performed By _____

Optical Bench Calibration N/A
 Optical Bench Calibration Complete

Barometric Pressure Gauge ID # _____

Simulator	Serial Number	Lot Number	Expiration
0.000		N/A	N/A
0.040			
0.100			
0.200			
0.400			
0.080 DGS	N/A		

Post Calibration Stability Checks

Simulator	Serial Number	Lot Number	Expiration
0.05			
0.08			
0.20			
0.08 DGS	N/A		

Department Inspection Performed By DELL

Barometric Pressure 1014 Gauge
 ID# 28159 1012 Instrument

Mouth Alcohol Solution Lot # 2016-A
 Acetone Stock Solution Lot # 2016-B

Simulator	Serial Number
0.00	SD 3963
Interferent	SD 3966
0.05	SD 3967
0.08	SD 3968
0.20	SD 3969

Attachments

Form 41
 Pre-Stability Tests
 Flow Calibration

Optical Bench Cal
 Post-Stability Tests
 Other _____

Notes: **E-MAILED**
8/8/2016 **APPROVED**

OC/OC OK UJGM 8/12/16

Brett Kirkland

Instrument Complies with Chapter 11D-8, FAC
 Instrument Does Not Comply with Chapter 11D-8, FAC

Return to/Place into Evidentiary Use
 Remain Out of Evidentiary Use
 Conduct an Agency Inspection Before Evidentiary Use

8/15/16

Quality Control Review

Date

TYPE OF TEST	SERIAL NUMBER	AGENCY	DATE	PERFORMED BY
Stabilities	80-001201	Collier County Sheriff's Office	08/08/2016	<i>WLL</i>

AK

0.05g/210L <input checked="" type="checkbox"/>	0.08g/210L <input checked="" type="checkbox"/>	0.20g/210L <input checked="" type="checkbox"/>	DGS 0.08g/210L <input checked="" type="checkbox"/>
<p>COLLIER COUNTY 52 Intoxilizer - Alcotest Analyzer Model 8003 SN 80-001201 08/08/2016 Software: 8100.27</p> <p>Test 9/210L Time</p> <p>Air Blank 0.000 10:13 Control Test 0.050 10:13 Air Blank 0.000 10:14 Control Test 0.050 10:15 Air Blank 0.000 10:15 Control Test 0.050 10:15 Air Blank 0.000 10:16</p> <p>Control Test Stats Average 0.0500 Std Dev 0.0000 Rel Std Dev(%) 0.0000</p> <p><i>WLL</i> Operator's Signature</p>	<p>COLLIER COUNTY 52 Intoxilizer - Alcotest Analyzer Model 8000 SN 80-001201 08/08/2016 Software: 8100.27</p> <p>Test 9/210L Time</p> <p>Air Blank 0.000 10:18 Control Test 0.080 10:19 Air Blank 0.000 10:19 Control Test 0.080 10:20 Air Blank 0.000 10:20 Control Test 0.080 10:21 Air Blank 0.000 10:21</p> <p>Control Test Stats Average 0.0800 Std Dev 0.0000 Rel Std Dev(%) 0.0000</p> <p><i>WLL</i> Operator's Signature</p>	<p>COLLIER COUNTY 52 Intoxilizer - Alcotest Analyzer Model 8000 SN 80-001201 08/08/2016 Software: 8100.27</p> <p>Test 9/210L Time</p> <p>Air Blank 0.500 10:23 Control Test 0.200 10:24 Air Blank 0.500 10:24 Control Test 0.200 10:25 Air Blank 0.000 10:25 Control Test 0.200 10:25 Air Blank 0.000 10:27</p> <p>Control Test Stats Average 0.2023 Std Dev 0.0012 Rel Std Dev(%) 0.5707</p> <p><i>WLL</i> Operator's Signature</p>	<p>COLLIER COUNTY 52 Intoxilizer - Alcotest Analyzer Model 8000 SN 80-001201 08/08/2016 Software: 8100.27</p> <p>Test 9/210L Time</p> <p>Air Blank 0.000 10:28 Control Test 0.080 10:29 Air Blank 0.000 10:29 Control Test 0.080 10:30 Air Blank 0.000 10:30 Control Test 0.080 10:30 Air Blank 0.000 10:31</p> <p>Control Test Stats Average 0.0803 Std Dev 0.0005 Rel Std Dev(%) 0.7187</p> <p><i>WLL</i> Operator's Signature</p>

WLL