



Alcohol Testing Program

INSTRUMENT PROCESSING SHEET

Agency Santa Rosa County S/N 80-001303

Date In 9/30/16 Date Out 10/4/16 Ship P/U H/D CMI EE

Intake Performed By [Signature]

Registration
 Annual
 Return from CMI
 Return from Enforcement Electronics
 Other _____

Visual Inspection:
 Case Handle
 Dry Gas Holder Feet
 Keyboard/Plug Back/Plugs
 Screws tight Breath Hose

Other Equipment:
 Power cord
 Printer Cable
 Other Static Bag

Notes: _____

Quality Checks Performed By [Signature]

Breath Tube Screen
 Replace O-Rings
 Instrument Set Up Verified
 R-Value 169
 Flow Verification (L/s)
 Flow Column # AP102
 32mm 150 (.139 - .169)
 36mm 179 (.156 - .190)
 53mm 242 (.228 - .278)
 103mm 503 (.447 - .547)

Barometric Pressure Check
 Gauge ID # 28427

Stability Checks

| Simulator | Serial # | Lot #/Exp |
|-----------|----------|--------------------|
| 0.05 | SD1018 | 201507A 7/14/17 |
| 0.08 | SD1011 | 201601F 1/20/18 |
| 0.20 | SD1025 | 201604C 4/5/18 |
| 0.08 DGS | N/A | AG612405 5/3/18 |

Flow Calibration Performed By _____

Flow Calibration N/A
 Flow Calibration Complete
 Flow Column # _____
 5L/min - 17mm
 15L/min - 53mm
 30L/min - 103mm

R-Value _____
 Post Calibration Verification (L/s)
 Flow Column # _____
 32mm _____ (.139 - .169)
 36mm _____ (.156 - .190)
 53mm _____ (.228 - .278)
 103mm _____ (.447 - .547)

Maintenance Performed By _____

Battery Replacement
 Dry Gas Regulator Replacement
 Breath Tube Replacement
 Other _____

Suggested Service

RECEIVED
OCT 06 2016
FDLE
Alcohol Testing Program

Optical Bench Calibration Performed By _____

Optical Bench Calibration N/A
 Optical Bench Calibration Complete

Barometric Pressure Gauge ID # _____

| Simulator | Serial Number | Lot Number | Expiration |
|-----------|---------------|------------|------------|
| 0.000 | | N/A | N/A |
| 0.040 | | | |
| 0.100 | | | |
| 0.200 | | | |
| 0.400 | | | |
| 0.080 DGS | N/A | | |

Post Calibration Stability Checks

| Simulator | Serial Number | Lot Number | Expiration |
|-----------|---------------|------------|------------|
| 0.05 | | | |
| 0.08 | | | |
| 0.20 | | | |
| 0.08 DGS | N/A | | |

Department Inspection Performed By [Signature]

Barometric Pressure 1014 Gauge
 ID# 28422 1609 Instrument

Mouth Alcohol Solution Lot # 2016-A
 Acetone Stock Solution Lot # 2016-B

| Simulator | Serial Number |
|-------------|---------------|
| 0.00 | SD1019 |
| Interferent | SD1021 |
| 0.05 | SD1018 |
| 0.08 | SD1011 |
| 0.20 | SD1025 |

Attachments

Form 41
 Pre-Stability Tests
 Flow Calibration

Optical Bench Cal
 Post-Stability Tests
 Other _____

Notes: QC: SP

Instrument Complies with Chapter 11D-8, FAC
 Instrument Does Not Comply with Chapter 11D-8, FAC

Return to/Place into Evidentiary Use
 Remain Out of Evidentiary Use

Conduct an Agency Inspection Before Evidentiary Use

Brett Henderson

Quality Control Review

10/6/16
Date

Stability Tests - Santa Rosa CSD #80-001303

SANTA ROSA COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001303
10/04/2016
Software: 8100.27

| Test | 9/210L | Time |
|--------------------|--------|-------|
| Air Blank | 0.000 | 11:06 |
| Control Test | 0.048 | 11:06 |
| Air Blank | 0.000 | 11:07 |
| Control Test | 0.049 | 11:08 |
| Air Blank | 0.000 | 11:08 |
| Control Test | 0.043 | 11:09 |
| Air Blank | 0.000 | 11:09 |
| Control Test Stats | | |
| Average | 0.0487 | |
| Std Dev | 0.0006 | |
| Rel Std Dev(%) | 1.1863 | |

[Signature]
Operator's Signature

SANTA ROSA COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001303
10/04/2016
Software: 8100.27

| Test | 9/210L | Time |
|--------------------|--------|-------|
| Air Blank | 0.000 | 11:10 |
| Control Test | 0.080 | 11:11 |
| Air Blank | 0.000 | 11:11 |
| Control Test | 0.079 | 11:12 |
| Air Blank | 0.000 | 11:12 |
| Control Test | 0.079 | 11:13 |
| Air Blank | 0.000 | 11:14 |
| Control Test Stats | | |
| Average | 0.0793 | |
| Std Dev | 0.0006 | |
| Rel Std Dev(%) | 0.7277 | |

[Signature]
Operator's Signature

SANTA ROSA COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001303
10/04/2016
Software: 8100.27

| Test | 9/210L | Time |
|--------------------|--------|-------|
| Air Blank | 0.000 | 11:15 |
| Control Test | 0.194 | 11:15 |
| Air Blank | 0.000 | 11:16 |
| Control Test | 0.196 | 11:17 |
| Air Blank | 0.000 | 11:17 |
| Control Test | 0.197 | 11:18 |
| Air Blank | 0.000 | 11:18 |
| Control Test Stats | | |
| Average | 0.1937 | |
| Std Dev | 0.0015 | |
| Rel Std Dev(%) | 0.7807 | |

[Signature]
Operator's Signature

SANTA ROSA COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001303
10/04/2016
Software: 8100.27

| Test | 9/210L | Time |
|--------------------|--------|-------|
| Air Blank | 0.000 | 11:23 |
| Control Test | 0.082 | 11:23 |
| Air Blank | 0.000 | 11:24 |
| Control Test | 0.082 | 11:24 |
| Air Blank | 0.000 | 11:24 |
| Control Test | 0.082 | 11:25 |
| Air Blank | 0.000 | 11:25 |
| Control Test Stats | | |
| Average | 0.0820 | |
| Std Dev | 0.0000 | |
| Rel Std Dev(%) | 0.0000 | |

DGS

[Signature] SP
[Signature] BSX
Operator's Signature