

**Intoxilyzer 8000**  
**Evidentiary Instrument Set Up Procedures for Department Inspectors**

(Note: Select = Place cursor under)

1. Press Esc, Esc
2. Enter Last Name , First Name, MI at prompts
3. Press 3
4. Enter Password
5. Select S (Set Up) and Press Enter
6. Select E (Time/Date) – Press Enter  
Verify date and time; Change if necessary
7. Select L (Set Agency) – Press Enter  
Enter the owning agency's name
8. Select P (Printer Set Up) – Press Enter  
*Inhibit Internal Printer*- Enter "N"  
*Print Copy Count* – Enter "1"
9. Select G (General Set Up) – Press Enter  
*Display Volume* – Enter "N"  
*Display Third Digit* – Enter "Y"  
*Disable on Mem Full* – Enter "Y"  
*Show Prelim Results* – Enter "N"  
*Enable Data Stream* – Enter "N"
10. Select C (Comms Transfer Set Up) - Press Enter  
*Enter Database Phone Number* (Use prefix if necessary placing comma after the prefix)  
*Max Tries* – Enter "3"  
*Max Time* – Enter "5"
11. Select T (Configure Start Test) – Press Enter  
*Data Entry Mode* – Select Enabled  
*Start Test Sequence* – Select DACABAWABA(WABA)CAD  
*Select Cal (D/W/I)* – Enter "D"  
*Enter Target Value* – Enter "0.080"
12. Select V (Serial Number/Version Info) – Press Enter  
Verify Serial Number and Software Version. If incorrect, terminate set up and contact ATP Manager. If correct, press Esc
13. Select X (Disable/Enable Instrument)  
If "Enable Instrument" is displayed, press Enter.  
If "Disable Instrument" is displayed, continue on with Step 14.
14. Select Z (Change Password(s))  
Enter Level to Change – Enter "2" to change Agency Inspector password.  
Enter "3" to change Department Inspector password.
15. Press Esc until instrument returns to Ready Mode.