



INSTRUMENT PROCESSING SHEET

Agency Ocala Police Department

S/N 80-005454

Florida Department of
Law Enforcement

Date In 12/22/2022 DI Completion Date N/A

☐ Ship ☐ P/U ☐ H/D ☒ CMI ☐ EE

Intake	By	PN	Quality Checks	By	Date	Flow Calibration	By	Date																								
<input checked="" type="checkbox"/> Annual <input type="checkbox"/> Registration <input type="checkbox"/> Return from CMI / EE Visual Inspection: <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight Other Equipment/ Accessories: <input type="checkbox"/> Power cord <input type="checkbox"/> Printer Cable <input checked="" type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable Notes: <u>Instrument would not come out of Standby Mode. The Int Print check would fail during the Diagnostic Check; unable to perform inspection.</u> <u>Instrument is missing two feet and has slight damage to printer cover.</u>			<input type="checkbox"/> Breath Tube Screen <input type="checkbox"/> Replace External O-Rings <input type="checkbox"/> Instrument Set Up Verified <input type="checkbox"/> R-Value _____ <input type="checkbox"/> Flow Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547) <input type="checkbox"/> Barometric Pressure Check Gauge ID # _____ <input type="checkbox"/> Stability Checks			Flow Column # _____ <input type="checkbox"/> 5L/min – 17mm <input type="checkbox"/> 15L/min – 53mm <input type="checkbox"/> 30L/min – 103mm <input type="checkbox"/> R-Value _____ <input type="checkbox"/> Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547)																										
			<table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #/Exp	0.050			0.080			0.200			0.080 DGS	N/A															
Simulator	Serial #	Lot #/Exp																														
0.050																																
0.080																																
0.200																																
0.080 DGS	N/A																															
			<table border="1"> <thead> <tr> <th colspan="2">Maintenance</th> <th>By</th> </tr> </thead> <tbody> <tr> <td><input type="checkbox"/> Battery Replacement</td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> Dry Gas Regulator Replacement</td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> Breath Tube Replacement</td> <td></td> <td></td> </tr> <tr> <td><input type="checkbox"/> Other _____</td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> </tr> </tbody> </table>						Maintenance		By	<input type="checkbox"/> Battery Replacement			<input type="checkbox"/> Dry Gas Regulator Replacement			<input type="checkbox"/> Breath Tube Replacement			<input type="checkbox"/> Other _____											
Maintenance		By																														
<input type="checkbox"/> Battery Replacement																																
<input type="checkbox"/> Dry Gas Regulator Replacement																																
<input type="checkbox"/> Breath Tube Replacement																																
<input type="checkbox"/> Other _____																																

Calibration Adjustment	By	Department Inspection	By																																																																						
Barometric Pressure Gauge _____ ID # _____ <table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> <input type="checkbox"/> Post Calibration Adjustment Stability Checks <table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> Notes/Suggested Service: <u>At request of Agency, instrument sent to repair.</u> <u>Int. Print check would not pass (see 'Other' attachment).</u> <u>Printer was checked to determine if printer paper was loaded correctly and two attempts were made to determine if instrument would pass Int Printer check.</u> <u>PN 12/27/2022</u>	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A				Barometric Pressure ID# _____ Gauge _____ Instrument _____ Mouth Alcohol Solution Lot # _____ Acetone Stock Solution Lot # _____ <table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> </tr> <tr> <td>Interferent</td> <td></td> </tr> <tr> <td>0.050</td> <td></td> </tr> <tr> <td>0.080</td> <td></td> </tr> <tr> <td>0.200</td> <td></td> </tr> </tbody> </table> <table border="1"> <thead> <tr> <th colspan="2">Attachments</th> </tr> </thead> <tbody> <tr> <td><input type="checkbox"/> Form 41</td> <td><input type="checkbox"/> Post-Stability Checks</td> </tr> <tr> <td><input type="checkbox"/> Stability Checks</td> <td><input type="checkbox"/> Flow Calibration</td> </tr> <tr> <td><input type="checkbox"/> Calibration Certificate</td> <td><input type="checkbox"/> Form 40</td> </tr> <tr> <td><input type="checkbox"/> Calibration Adjustment</td> <td><input checked="" type="checkbox"/> Other <u>Printer Failure + Form 51</u></td> </tr> </tbody> </table> <input type="checkbox"/> Instrument Complies with Chapter 11D-8, FAC <input checked="" type="checkbox"/> Instrument Does Not Comply with Chapter 11D-8, FAC <input type="checkbox"/> Return to/Place into Evidentiary Use <input checked="" type="checkbox"/> Remain Out of Evidentiary Use <input type="checkbox"/> Conduct an Agency Inspection Before Evidentiary Use	Simulator	Serial Number	0.000		Interferent		0.050		0.080		0.200		Attachments		<input type="checkbox"/> Form 41	<input type="checkbox"/> Post-Stability Checks	<input type="checkbox"/> Stability Checks	<input type="checkbox"/> Flow Calibration	<input type="checkbox"/> Calibration Certificate	<input type="checkbox"/> Form 40	<input type="checkbox"/> Calibration Adjustment	<input checked="" type="checkbox"/> Other <u>Printer Failure + Form 51</u>	
Simulator	Serial #	Lot #	Expiration																																																																						
0.000		N/A	N/A																																																																						
0.040																																																																									
0.100																																																																									
0.200																																																																									
0.300																																																																									
0.080 DGS	N/A																																																																								
Simulator	Serial #	Lot #	Expiration																																																																						
0.050																																																																									
0.080																																																																									
0.200																																																																									
0.080 DGS	N/A																																																																								
Simulator	Serial Number																																																																								
0.000																																																																									
Interferent																																																																									
0.050																																																																									
0.080																																																																									
0.200																																																																									
Attachments																																																																									
<input type="checkbox"/> Form 41	<input type="checkbox"/> Post-Stability Checks																																																																								
<input type="checkbox"/> Stability Checks	<input type="checkbox"/> Flow Calibration																																																																								
<input type="checkbox"/> Calibration Certificate	<input type="checkbox"/> Form 40																																																																								
<input type="checkbox"/> Calibration Adjustment	<input checked="" type="checkbox"/> Other <u>Printer Failure + Form 51</u>																																																																								
Tech Review / Date _____		Admin Review / Date _____																																																																							

80-005454

RLS

[illegible][illegible]

PA

[illegible]

23

Return Material Authorization

Ship to: ☒ CMI, Inc.

☐ Enforcement Electronics

Shipment to repair facility authorized by: William Joedicke on 12/27/2022

Items Returned: Instrument ☒ Supplies ☐ Other ☐ Describe: _____

Instrument Model: Intoxilyzer 8000 Serial Number: 80-005454

Bill To Address:

Ocala Police Department

Ship to Address:

FDLE Off-Site Mail Facility

Florida Department of Law Enforcement

Alcohol Testing Program

813 B Lake Bradford Road

Tallahassee, FL 32304

Reason for Return:

Instrument will not come out of Standby Mode; Int Print test fail.

Printer was checked to determine if printer paper was loaded correctly.

Please choose one of the following options:

☐ 1. I _____, authorize all repairs.

☐ 2. I _____, authorize repairs up to \$_____.

☒ 3. I require an estimate **BEFORE** any repairs will be authorized and/ or conducted.

Please contact: Name: William Joedicke

Phone #: 352-789-9378 Email: wjoedicke@ocalapd.org

ATP Contact Name: Phil Nicodemo ATP Email: philipnicodemo@fdle.state.fl.us

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: Ocala Police Department
Time of Inspection: N/A

Date of Inspection: 2022

Serial Number: 80-005454
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK			Date and/or Time Adjusted		
Minimum Sample Volume Check: OK			Barometric Pressure Sensor Check: OK		
Alcohol Free Subject Test: 0.000			Mouth Alcohol Test: Slope Not Met		
Interferent Detect Test: Interferent Detect			Diagnostic Check (Post-Inspection): OK		

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#: Exp:	0.08g/210L Test (g/210L) Lot#: Exp:	0.20g/210L Test (g/210L) Lot#: Exp:	0.08 g/210L Dry Gas Std Test (g/210L) Lot#: Exp:

Standard Deviations				
---------------------	--	--	--	--

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: _____ Number of Simulators Used: _____

Remarks:

Department Inspection was not performed for 2022. Instrument was in need of repair when it arrived at Florida Department of Law Enforcement and was sent to an authorized repair facility. The instrument did not return during the calendar year.

Compliance not determined.

The above instrument complies () does not comply () with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.



Phil Nicodemo

Signature and Printed Name

12/28/2022

Date