



INSTRUMENT PROCESSING SHEET

Agency Miami-Dade Police Department

S/N 80-000881

Florida Department of Law Enforcement

Date In 01/15/2019 DI Completion Date 1/15/2019

Ship P/U H/D CMI EE

Intake Performed By <u>DEAR</u> <input checked="" type="checkbox"/> Annual <input type="checkbox"/> Registration <input type="checkbox"/> Return from CMI / EE Visual Inspection: <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight Other Equipment/ Accessories: <input type="checkbox"/> Power cord <input type="checkbox"/> Printer Cable <input type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable Notes: _____ _____ _____		Quality Checks Performed By <u>DEAR</u> <input checked="" type="checkbox"/> Breath Tube Screen <input checked="" type="checkbox"/> Replace External O-Rings <input checked="" type="checkbox"/> Instrument Set Up Verified <input checked="" type="checkbox"/> R-Value <u>245</u> <input checked="" type="checkbox"/> Flow Verification (L/s) Flow Column # <u>ATP 106</u> 32 mm <u>.128</u> (.139 - .169) 36 mm <u>.144</u> (.156 - .190) 53 mm <u>.210</u> (.228 - .278) 103 mm <u>.500</u> (.447 - .547) <input checked="" type="checkbox"/> Barometric Pressure Check Gauge ID # <u>28663</u> <input checked="" type="checkbox"/> Stability Checks		Flow Calibration Performed By <u>DEAR</u> Flow Column # <u>ATP104</u> <input checked="" type="checkbox"/> 5L/min - 17mm <input checked="" type="checkbox"/> 15L/min - 53mm <input checked="" type="checkbox"/> 30L/min - 103mm <input checked="" type="checkbox"/> R-Value <u>245</u> <input checked="" type="checkbox"/> Post Calibration Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>.148</u> (.139 - .169) 36 mm <u>.167</u> (.156 - .190) 53 mm <u>.234</u> (.228 - .278) 103 mm <u>.515</u> (.447 - .547)																
Final Release Date FDLE JAN 23 2019 Alcohol Testing Program		<table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>SD3967</td> <td>201707D 07/25/2019</td> </tr> <tr> <td>0.080</td> <td>SD3968</td> <td>201707E 07/25/2019</td> </tr> <tr> <td>0.200</td> <td>SD3969</td> <td>201707C 07/24/2019</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG805701 02/26/2020</td> </tr> </tbody> </table>		Simulator	Serial #	Lot #/Exp	0.050	SD3967	201707D 07/25/2019	0.080	SD3968	201707E 07/25/2019	0.200	SD3969	201707C 07/24/2019	0.080 DGS	N/A	AG805701 02/26/2020	Maintenance Performed By <u>DEAR</u> <input type="checkbox"/> Battery Replacement <input type="checkbox"/> Dry Gas Regulator Replacement <input type="checkbox"/> Breath Tube Replacement <input checked="" type="checkbox"/> Other <u>Form load/Changed pass</u>	
Simulator	Serial #	Lot #/Exp																		
0.050	SD3967	201707D 07/25/2019																		
0.080	SD3968	201707E 07/25/2019																		
0.200	SD3969	201707C 07/24/2019																		
0.080 DGS	N/A	AG805701 02/26/2020																		
		Temperature Checks Performed By <u>DEAR</u> <input checked="" type="checkbox"/> Lab Temp °C <u>22.48C</u> External Digital Therm. ID#: <u>300949/300518</u> <u>DEAR</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>SD3967</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>SD3968</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>SD3969</u>																		

Calibration Adjustment Performed By _____ Barometric Pressure Gauge _____ ID # _____				Department Inspection Performed By <u>DEAR</u> Barometric Pressure ID# <u>68639</u> Gauge <u>1021</u> Instrument <u>1018</u> Mouth Alcohol Solution Lot # <u>2017-B</u> Acetone Stock Solution Lot # <u>2018-A</u>																																											
<table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> <th>Lot Number</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>				Simulator	Serial Number	Lot Number	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			<table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>SD3965</td> </tr> <tr> <td>Interferent</td> <td>SD3966</td> </tr> <tr> <td>0.050</td> <td>SD3967</td> </tr> <tr> <td>0.080</td> <td>SD3968</td> </tr> <tr> <td>0.200</td> <td>SD3969</td> </tr> </tbody> </table>				Simulator	Serial Number	0.000	SD3965	Interferent	SD3966	0.050	SD3967	0.080	SD3968	0.200	SD3969
Simulator	Serial Number	Lot Number	Expiration																																												
0.000		N/A	N/A																																												
0.040																																															
0.100																																															
0.200																																															
0.300																																															
0.080 DGS	N/A																																														
Simulator	Serial Number																																														
0.000	SD3965																																														
Interferent	SD3966																																														
0.050	SD3967																																														
0.080	SD3968																																														
0.200	SD3969																																														
<input type="checkbox"/> Post Calibration Adjustment Stability Checks				Attachments <input checked="" type="checkbox"/> Form 41 <input type="checkbox"/> Post-Stability Checks <input checked="" type="checkbox"/> Stability Checks <input checked="" type="checkbox"/> Flow Calibration <input checked="" type="checkbox"/> Calibration Certificate <input type="checkbox"/> Form 40 <input type="checkbox"/> Calibration Adjustment <input type="checkbox"/> Other _____																																											
<table border="1"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> <th>Lot Number</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>				Simulator	Serial Number	Lot Number	Expiration	0.050				0.080				0.200				0.080 DGS	N/A																										
Simulator	Serial Number	Lot Number	Expiration																																												
0.050																																															
0.080																																															
0.200																																															
0.080 DGS	N/A																																														

Notes/Suggested Service: E-mailed **APPROVED**

Instrument Complies with Chapter 11D-8, FAC
 Instrument Does Not Comply with Chapter 11D-8, FAC
 Return to/Place into Evidentiary Use
 Remain Out of Evidentiary Use
 Conduct an Agency Inspection Before Evidentiary Use

DEAR 1/23/19 DEAR 1/23/19
 Tech Review / Date Admin Review / Date

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: MIAMI DADE PD
Time of Inspection: 14:08

Date of Inspection: 01/15/2019

Serial Number: 80-000881
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:201707D Exp: 07/25/2019	0.08g/210L Test (g/210L) Lot#:201707E Exp: 07/25/2019	0.20g/210L Test (g/210L) Lot#:201707C Exp: 07/24/2019	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG805701 Exp: 02/26/2020
0.000	0.050	0.080	0.198	0.081
0.000	0.051	0.082	0.201	0.082
0.000	0.050	0.081	0.202	0.081
0.000	0.050	0.082	0.202	0.082
0.000	0.050	0.082	0.201	0.081
0.000	0.051	0.081	0.201	0.081
0.000	0.050	0.082	0.201	0.082
0.000	0.050	0.081	0.201	0.081
0.000	0.050	0.081	0.200	0.081
0.000	0.050	0.082	0.200	0.081

Standard Deviations	0.0004	0.0006	0.0011	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0006 Number of Simulators Used: 5

Remarks:

DOM

The above instrument complies () does not comply () with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

David Reyes Rivera

DAVID E REYES-RIVERA

Signature and Printed Name

01/15/2019
Date

*1/23/19
JO*

4980

TYPE OF TEST	SERIAL NUMBER	AGENCY	DATE	PERFORMED BY
Stabilities	80-000881	Miami-Dade Police Department	01/15/2019	<i>Dea</i>

0.05g/210L	0.08g/210L	0.20g/210L	DGS 0.08g/210L
SN: SD3967 Temp: 34.10c	SN: SD3968 Temp: 34.09c	SN: SD3969 Temp: 34.10c	Lot AG805701
0.047 to 0.053 <input checked="" type="checkbox"/>	0.077 to 0.083 <input checked="" type="checkbox"/>	0.194 to 0.206 <input checked="" type="checkbox"/>	0.077 to 0.083 <input checked="" type="checkbox"/>

<p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p>	<p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p>	<p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p>	<p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p> <p>MIAMI DADE PD Intoxilyzer - Alcohol Analyzer Model: 8000 01/15/2019 Software: 8100.27</p>																																																																																																																																																
<table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:05</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:06</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:06</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:07</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:07</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:08</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:09</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0500</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table>	Test	g/210L	Time	Air Blank	0.000	11:05	Control Test	0.050	11:06	Air Blank	0.000	11:06	Control Test	0.050	11:07	Air Blank	0.000	11:07	Control Test	0.050	11:08	Air Blank	0.000	11:09	Control Test Stats			Average	0.0500		Std Dev	0.0000		Rel Std Dev(%)	0.0000		<table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:10</td></tr> <tr><td>Control Test</td><td>0.081</td><td>11:11</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:11</td></tr> <tr><td>Control Test</td><td>0.082</td><td>11:12</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:12</td></tr> <tr><td>Control Test</td><td>0.081</td><td>11:13</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:14</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0813</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7095</td><td></td></tr> </tbody> </table>	Test	g/210L	Time	Air Blank	0.000	11:10	Control Test	0.081	11:11	Air Blank	0.000	11:11	Control Test	0.082	11:12	Air Blank	0.000	11:12	Control Test	0.081	11:13	Air Blank	0.000	11:14	Control Test Stats			Average	0.0813		Std Dev	0.0006		Rel Std Dev(%)	0.7095		<table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:15</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:16</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:16</td></tr> <tr><td>Control Test</td><td>0.201</td><td>11:17</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:17</td></tr> <tr><td>Control Test</td><td>0.201</td><td>11:18</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:19</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.2007</td><td></td></tr> <tr><td>Std Dev</td><td>0.0016</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2877</td><td></td></tr> </tbody> </table>	Test	g/210L	Time	Air Blank	0.000	11:15	Control Test	0.200	11:16	Air Blank	0.000	11:16	Control Test	0.201	11:17	Air Blank	0.000	11:17	Control Test	0.201	11:18	Air Blank	0.000	11:19	Control Test Stats			Average	0.2007		Std Dev	0.0016		Rel Std Dev(%)	0.2877		<table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:20</td></tr> <tr><td>Control Test</td><td>0.082</td><td>11:21</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:21</td></tr> <tr><td>Control Test</td><td>0.081</td><td>11:21</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:22</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:22</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:23</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0810</td><td></td></tr> <tr><td>Std Dev</td><td>0.0010</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.2346</td><td></td></tr> </tbody> </table>	Test	g/210L	Time	Air Blank	0.000	11:20	Control Test	0.082	11:21	Air Blank	0.000	11:21	Control Test	0.081	11:21	Air Blank	0.000	11:22	Control Test	0.080	11:22	Air Blank	0.000	11:23	Control Test Stats			Average	0.0810		Std Dev	0.0010		Rel Std Dev(%)	1.2346	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:05																																																																																																																																																	
Control Test	0.050	11:06																																																																																																																																																	
Air Blank	0.000	11:06																																																																																																																																																	
Control Test	0.050	11:07																																																																																																																																																	
Air Blank	0.000	11:07																																																																																																																																																	
Control Test	0.050	11:08																																																																																																																																																	
Air Blank	0.000	11:09																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0500																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:10																																																																																																																																																	
Control Test	0.081	11:11																																																																																																																																																	
Air Blank	0.000	11:11																																																																																																																																																	
Control Test	0.082	11:12																																																																																																																																																	
Air Blank	0.000	11:12																																																																																																																																																	
Control Test	0.081	11:13																																																																																																																																																	
Air Blank	0.000	11:14																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0813																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7095																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:15																																																																																																																																																	
Control Test	0.200	11:16																																																																																																																																																	
Air Blank	0.000	11:16																																																																																																																																																	
Control Test	0.201	11:17																																																																																																																																																	
Air Blank	0.000	11:17																																																																																																																																																	
Control Test	0.201	11:18																																																																																																																																																	
Air Blank	0.000	11:19																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.2007																																																																																																																																																		
Std Dev	0.0016																																																																																																																																																		
Rel Std Dev(%)	0.2877																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:20																																																																																																																																																	
Control Test	0.082	11:21																																																																																																																																																	
Air Blank	0.000	11:21																																																																																																																																																	
Control Test	0.081	11:21																																																																																																																																																	
Air Blank	0.000	11:22																																																																																																																																																	
Control Test	0.080	11:22																																																																																																																																																	
Air Blank	0.000	11:23																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0810																																																																																																																																																		
Std Dev	0.0010																																																																																																																																																		
Rel Std Dev(%)	1.2346																																																																																																																																																		
<i>Dea</i> Operator's Signature	<i>Dea</i> Operator's Signature	<i>Dea</i> Operator's Signature	<i>Dea</i> Operator's Signature																																																																																																																																																

1/23/19
Dea



Florida Department of Law Enforcement
Alcohol Testing Program
4700 Terminal Drive, Suite 1
Ft. Myers, FL 33907

Calibration Certificate

This is to certify the calibration of Intoxilyzer 8000 serial number 80-000881, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-000881</u>	UNCERTAINTY* ±	
Owning Agency:	<u>MIAMI DADE PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>01/15/2019</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>14:08</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Thermometer temperatures are checked with NIST traceable Eutechnics 4400 digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the uses of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

01/15/2019 _____
Date

David Reyes Rivera
DAVID E REYES-RIVERA,
Department Inspector

FDLE/ATP Form 69 July 2018
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality

1/23/19
JD

2008

WBC

TYPE OF TEST	SERIAL NUMBER	AGENCY	DATE	PERFORMED BY
Flow calibration	80-000881	Miami-Dade Police Department	01/15/2019	DEA

MIAMI DADE PD
 Intoxilyzer - Alconcl Analyzer
 Model 8000 SN 80-000881
 01/15/2019
 Software: 8100.27

Flow Rate Calibration*****
 1: Rate (Liters/min) = 5
 SORT(Diff)) = 7.141
 2: Rate (Liters/min) = 15
 SORT(Diff)) = 11.488
 3: Rate (Liters/min) = 30
 SORT(Diff)) = 21.023

Dependent Data Scale Factor = 100000 L/min
 Independent Data Scale Factor = 256
 Rounded Slope = 689
 Rounded Intercept = -664319
 Correlation = 0.99525

1/23/19
JO