



INSTRUMENT PROCESSING SHEET

Agency Melbourne Police DepartmentS/N 80-001262Florida Department of
Law EnforcementDate In 12/6/2021DI Completion Date 12/8/2021☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake	Quality Checks	Flow Calibration																																																											
By <u>DERR</u> ☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/ Accessories: ☐ Power cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes: _____ _____ _____ _____ _____ _____ _____ _____ _____ _____	By <u>DER</u> Date <u>12/8/2021</u> ☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>181</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>0.144</u> (.139 - .169) 36 mm <u>0.167</u> (.156 - .190) 53 mm <u>0.234</u> (.228 - .278) 103 mm <u>0.503</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>28663</u> ☒ Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>MP6286</td> <td>202010A 10/05/2022</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> <td>202010B 10/05/2022</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> <td>202010D 10/06/2022</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG115904 06/08/2023</td> </tr> </tbody> </table>	Simulator	Serial #	Lot #/Exp	0.050	MP6286	202010A 10/05/2022	0.080	MP6287	202010B 10/05/2022	0.200	MP6288	202010D 10/06/2022	0.080 DGS	N/A	AG115904 06/08/2023	By _____ Date _____ Flow Column # _____ ☐ 5L/min - 17mm ☐ 15L/min - 53mm ☐ 30L/min - 103mm ☐ R-Value _____ ☐ Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547) Maintenance By _____ ☐ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Breath Tube Replacement ☐ Other _____ DI Temp. Checks By <u>DERR</u> ☒ Lab Temp °C <u>20.44C</u> External Digital Therm. ID#: <u>300503</u> ☒ 34°C +/- .2 Serial #: <u>MP6286</u> ☒ 34°C +/- .2 Serial #: <u>MP6287</u> ☒ 34°C +/- .2 Serial #: <u>MP6288</u>																																												
Simulator	Serial #	Lot #/Exp																																																											
0.050	MP6286	202010A 10/05/2022																																																											
0.080	MP6287	202010B 10/05/2022																																																											
0.200	MP6288	202010D 10/06/2022																																																											
0.080 DGS	N/A	AG115904 06/08/2023																																																											
Calibration Adjustment By <u>DERR</u> Barometric Pressure Gauge <u>1018</u> ID # <u>68639</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP5095</td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td>MP5096</td> <td>20060</td> <td>02/10/2022</td> </tr> <tr> <td>0.100</td> <td>MP5098</td> <td>20420</td> <td>09/22/2022</td> </tr> <tr> <td>0.200</td> <td>MP5100</td> <td>20160</td> <td>03/18/2022</td> </tr> <tr> <td>0.300</td> <td>MP5101</td> <td>20030</td> <td>01/21/2022</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>00521080A2</td> <td>02/05/2023</td> </tr> </tbody> </table> ☒ Post Calibration Adjustment Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>MP6286</td> <td>202010A</td> <td>10/05/2022</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> <td>202010B</td> <td>10/05/2022</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> <td>202010D</td> <td>10/06/2022</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG115904</td> <td>06/08/2023</td> </tr> </tbody> </table> Notes/Suggested Service: <u>Instrument's barometric pressure sensor is off the nominal window and will be calibrated. Optical calibration to bring values closer to nominal.</u> <u>DERR</u> _____ _____ _____ _____ _____	Simulator	Serial #	Lot #	Expiration	0.000	MP5095	N/A	N/A	0.040	MP5096	20060	02/10/2022	0.100	MP5098	20420	09/22/2022	0.200	MP5100	20160	03/18/2022	0.300	MP5101	20030	01/21/2022	0.080 DGS	N/A	00521080A2	02/05/2023	Simulator	Serial #	Lot #	Expiration	0.050	MP6286	202010A	10/05/2022	0.080	MP6287	202010B	10/05/2022	0.200	MP6288	202010D	10/06/2022	0.080 DGS	N/A	AG115904	06/08/2023	Department Inspection By <u>DERR</u> Barometric Pressure ID# <u>28199</u> Gauge <u>1020</u> Instrument <u>1018</u> Mouth Alcohol Solution Lot # <u>2021-D</u> Acetone Stock Solution Lot # <u>2021-C</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6284</td> </tr> <tr> <td>Interferent</td> <td>MP6285</td> </tr> <tr> <td>0.050</td> <td>MP6286</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> </tr> </tbody> </table> Attachments ☒ Form 41 ☒ Post-Stability Checks ☒ Stability Checks ☐ Flow Calibration ☒ Calibration Certificate ☒ Form 40 ☒ Calibration Adjustment ☐ Other _____ ☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use Israel Soto Digitally signed by Israel Soto Date: 2021.12.08 12:28:56 +05'00' <u>11:41:54</u> Tech Review / Date _____ Admin Review / Date _____	Simulator	Serial Number	0.000	MP6284	Interferent	MP6285	0.050	MP6286	0.080	MP6287	0.200	MP6288
Simulator	Serial #	Lot #	Expiration																																																										
0.000	MP5095	N/A	N/A																																																										
0.040	MP5096	20060	02/10/2022																																																										
0.100	MP5098	20420	09/22/2022																																																										
0.200	MP5100	20160	03/18/2022																																																										
0.300	MP5101	20030	01/21/2022																																																										
0.080 DGS	N/A	00521080A2	02/05/2023																																																										
Simulator	Serial #	Lot #	Expiration																																																										
0.050	MP6286	202010A	10/05/2022																																																										
0.080	MP6287	202010B	10/05/2022																																																										
0.200	MP6288	202010D	10/06/2022																																																										
0.080 DGS	N/A	AG115904	06/08/2023																																																										
Simulator	Serial Number																																																												
0.000	MP6284																																																												
Interferent	MP6285																																																												
0.050	MP6286																																																												
0.080	MP6287																																																												
0.200	MP6288																																																												



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
4700 Terminal Drive, Suite 1
Ft. Myers, FL 33907

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001262, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001262</u>	UNCERTAINTY* $\pm$
Owning Agency:	<u>MELBOURNE P.D.</u>	0.050 g/ 210 L 0.005
Calibration Date:	<u>12/08/2021</u>	0.080 g/ 210 L 0.004
Calibration Time:	<u>11:10</u>	0.200 g/ 210 L 0.007
		0.080 g/ 210 L Dry Gas Control 0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards. This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

FDLE/ATP Form 69 December 2021
Issuing Authority: Alcohol Testing Program

12/08/2021

Date


DAVID E REYES-RIVERA,

Department Inspector

Service • Integrity • Respect • Quality

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: MELBOURNE P.D.

Time of Inspection: 11:10

Date of Inspection: 12/08/2021

Serial Number: 80-001262

Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202010A Exp: 10/05/2022	0.08g/210L Test (g/210L) Lot#:202010B Exp: 10/05/2022	0.20g/210L Test (g/210L) Lot#:202010D Exp: 10/06/2022	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG115904 Exp: 06/08/2023
0.000	0.049	0.078	0.198	0.079
0.000	0.049	0.079	0.199	0.079
0.000	0.049	0.078	0.198	0.079
0.000	0.049	0.079	0.198	0.079
0.000	0.049	0.079	0.198	0.078
0.000	0.049	0.078	0.198	0.079
0.000	0.049	0.079	0.198	0.078
0.000	0.048	0.078	0.198	0.078
0.000	0.049	0.079	0.198	0.078
0.000	0.049	0.079	0.199	0.078

Standard Deviations	0.0003	0.0005	0.0004	0.0005
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

David E Reyes-Rivera DAVID E REYES-RIVERA
Signature and Printed Name

12/08/2021
Date

Type of Test	Serial Number	Agency	Date	Performed By
Post Stabilities	80-001262	Melbourne Police Department	12/8/2021	DERR <i>ML</i>

0.05g/210L	0.08g/210L	0.20g/210L	DGS 0.08g/210L
0.047 to 0.053 ☒	0.077 to 0.083 ☒	0.194 to 0.206 ☒	0.077 to 0.083 ☒
MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 Test g/210L Time Air Blank 0.000 09:17 Control Test 0.048 09:17 Air Blank 0.000 09:18 Control Test 0.048 09:19 Air Blank 0.000 09:19 Control Test 0.049 09:20 Air Blank 0.000 09:20 Control Test Status Average 0.0483 Std Dev 0.0006 Rel Std Dev(%) 1.1945	MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-011262 Test g/210L Time Air Blank 0.000 09:21 Control Test 0.079 09:22 Air Blank 0.000 09:22 Control Test 0.078 09:23 Air Blank 0.000 09:24 Control Test 0.079 09:24 Air Blank 0.000 09:25 Control Test Status Average 0.0787 Std Dev 0.0006 Rel Std Dev(%) 0.7359	MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 Test g/210L Time Air Blank 0.000 09:26 Control Test 0.198 09:26 Air Blank 0.000 09:27 Control Test 0.198 09:28 Air Blank 0.000 09:28 Control Test 0.199 09:29 Air Blank 0.000 09:29 Control Test Status Average 0.1983 Std Dev 0.0006 Rel Std Dev(%) 0.2911	MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 Test g/210L Time Air Blank 0.000 09:30 Control Test 0.080 09:31 Air Blank 0.000 09:31 Control Test 0.079 09:32 Air Blank 0.000 09:32 Control Test 0.079 09:32 Air Blank 0.000 09:33 Control Test Status Average 0.0793 Std Dev 0.0006 Rel Std Dev(%) 0.7277
<i>ML</i> Operator's Signature	<i>ML</i> Operator's Signature	<i>ML</i> Operator's Signature	<i>ML</i> Operator's Signature

Type of Test	Serial Number	Agency	Date	Performed By
Stabilities	80-001262	Melbourne Police Department	12/08/2021	DERR <i>WLL</i>

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083																																																																																																																																																
MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 <table><thead><tr><th>Test</th><th>g/210L</th><th>Time</th></tr></thead><tbody><tr><td>Air Blank</td><td>0.000</td><td>08:01</td></tr><tr><td>Control Test</td><td>0.049</td><td>08:02</td></tr><tr><td>Air Blank</td><td>0.001</td><td>08:02</td></tr><tr><td>Control Test</td><td>0.049</td><td>08:03</td></tr><tr><td>Air Blank</td><td>0.001</td><td>08:03</td></tr><tr><td>Control Test</td><td>0.048</td><td>08:04</td></tr><tr><td>Air Blank</td><td>0.001</td><td>08:05</td></tr><tr><td>Control Test Stats</td><td></td><td></td></tr><tr><td>Average</td><td>0.0487</td><td></td></tr><tr><td>Std Dev</td><td>0.0006</td><td></td></tr><tr><td>Rel Std Dev(%)</td><td>1.1863</td><td></td></tr></tbody></table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	08:01	Control Test	0.049	08:02	Air Blank	0.001	08:02	Control Test	0.049	08:03	Air Blank	0.001	08:03	Control Test	0.048	08:04	Air Blank	0.001	08:05	Control Test Stats			Average	0.0487		Std Dev	0.0006		Rel Std Dev(%)	1.1863		MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 <table><thead><tr><th>Test</th><th>g/210L</th><th>Time</th></tr></thead><tbody><tr><td>Air Blank</td><td>0.000</td><td>08:06</td></tr><tr><td>Control Test</td><td>0.078</td><td>08:06</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:07</td></tr><tr><td>Control Test</td><td>0.078</td><td>08:08</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:08</td></tr><tr><td>Control Test</td><td>0.079</td><td>08:09</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:09</td></tr><tr><td>Control Test Stats</td><td></td><td></td></tr><tr><td>Average</td><td>0.0783</td><td></td></tr><tr><td>Std Dev</td><td>0.0006</td><td></td></tr><tr><td>Rel Std Dev(%)</td><td>0.7370</td><td></td></tr></tbody></table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	08:06	Control Test	0.078	08:06	Air Blank	0.000	08:07	Control Test	0.078	08:08	Air Blank	0.000	08:08	Control Test	0.079	08:09	Air Blank	0.000	08:09	Control Test Stats			Average	0.0783		Std Dev	0.0006		Rel Std Dev(%)	0.7370		MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 <table><thead><tr><th>Test</th><th>g/210L</th><th>Time</th></tr></thead><tbody><tr><td>Air Blank</td><td>0.000</td><td>08:11</td></tr><tr><td>Control Test</td><td>0.199</td><td>08:11</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:12</td></tr><tr><td>Control Test</td><td>0.198</td><td>08:13</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:13</td></tr><tr><td>Control Test</td><td>0.197</td><td>08:14</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:14</td></tr><tr><td>Control Test Stats</td><td></td><td></td></tr><tr><td>Average</td><td>0.1980</td><td></td></tr><tr><td>Std Dev</td><td>0.0010</td><td></td></tr><tr><td>Rel Std Dev(%)</td><td>0.5051</td><td></td></tr></tbody></table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	08:11	Control Test	0.199	08:11	Air Blank	0.000	08:12	Control Test	0.198	08:13	Air Blank	0.000	08:13	Control Test	0.197	08:14	Air Blank	0.000	08:14	Control Test Stats			Average	0.1980		Std Dev	0.0010		Rel Std Dev(%)	0.5051		MELBOURNE P.D. Intoxilyzer - Alcohol Analyzer Model 8000 12/08/2021 Software: 8100.27 SN 80-001262 <table><thead><tr><th>Test</th><th>g/210L</th><th>Time</th></tr></thead><tbody><tr><td>Air Blank</td><td>0.000</td><td>08:16</td></tr><tr><td>Control Test</td><td>0.080</td><td>08:16</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:17</td></tr><tr><td>Control Test</td><td>0.080</td><td>08:17</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:18</td></tr><tr><td>Control Test</td><td>0.081</td><td>08:18</td></tr><tr><td>Air Blank</td><td>0.000</td><td>08:19</td></tr><tr><td>Control Test Stats</td><td></td><td></td></tr><tr><td>Average</td><td>0.0803</td><td></td></tr><tr><td>Std Dev</td><td>0.0006</td><td></td></tr><tr><td>Rel Std Dev(%)</td><td>0.7187</td><td></td></tr></tbody></table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	08:16	Control Test	0.080	08:16	Air Blank	0.000	08:17	Control Test	0.080	08:17	Air Blank	0.000	08:18	Control Test	0.081	08:18	Air Blank	0.000	08:19	Control Test Stats			Average	0.0803		Std Dev	0.0006		Rel Std Dev(%)	0.7187	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	08:01																																																																																																																																																	
Control Test	0.049	08:02																																																																																																																																																	
Air Blank	0.001	08:02																																																																																																																																																	
Control Test	0.049	08:03																																																																																																																																																	
Air Blank	0.001	08:03																																																																																																																																																	
Control Test	0.048	08:04																																																																																																																																																	
Air Blank	0.001	08:05																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0487																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	1.1863																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	08:06																																																																																																																																																	
Control Test	0.078	08:06																																																																																																																																																	
Air Blank	0.000	08:07																																																																																																																																																	
Control Test	0.078	08:08																																																																																																																																																	
Air Blank	0.000	08:08																																																																																																																																																	
Control Test	0.079	08:09																																																																																																																																																	
Air Blank	0.000	08:09																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0783																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7370																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	08:11																																																																																																																																																	
Control Test	0.199	08:11																																																																																																																																																	
Air Blank	0.000	08:12																																																																																																																																																	
Control Test	0.198	08:13																																																																																																																																																	
Air Blank	0.000	08:13																																																																																																																																																	
Control Test	0.197	08:14																																																																																																																																																	
Air Blank	0.000	08:14																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.1980																																																																																																																																																		
Std Dev	0.0010																																																																																																																																																		
Rel Std Dev(%)	0.5051																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	08:16																																																																																																																																																	
Control Test	0.080	08:16																																																																																																																																																	
Air Blank	0.000	08:17																																																																																																																																																	
Control Test	0.080	08:17																																																																																																																																																	
Air Blank	0.000	08:18																																																																																																																																																	
Control Test	0.081	08:18																																																																																																																																																	
Air Blank	0.000	08:19																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0803																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7187																																																																																																																																																		

Florida Department of Law Enforcement Alcohol Testing Program

AGENCY INSPECTION REPORT - INTOXILYZER 8000

Agency: MELBOURNE P.D.

Time of Inspection: 12:37

Date of Inspection: 12/07/2021

Serial Number: 80-001262

Software: 8100.27

Check or Test	YES	NO
Date and/or Time Adjusted		No
Diagnostic Check (Pre-Inspection): OK		No
Alcohol Free Subject Test: 0.000		No
Mouth Alcohol Test: Slope Not Met		No
Interferent Detect Test: Interferent Detect		No
Diagnostic Check (Post-Inspection): OK		No

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#: _____ Exp: _____	0.08g/210L Test (g/210L) Lot#: _____ Exp: _____	0.20g/210L Test (g/210L) Lot#: _____ Exp: _____	0.08 g/210L Dry Gas Std Test (g/210L) Lot#: _____ Exp: _____

Number of Simulators Used: _____

Remarks:

COMPLIANCE NOT DETERMINED, AI NOT CONDUCTED.

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I hold a valid Florida Department of Law Enforcement Agency Inspector Permit and that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.



DAVID E REYES-RIVERA

Signature and Printed Name

12/07/2021
Date