



INSTRUMENT PROCESSING SHEET

Agency FFWCCS/N 80-007500Florida Department of
Law EnforcementDate In 8/23/2021DI Completion Date 8/24/2021 Ship P/U H/D CMI EE

Intake By <u>RAW</u> <input checked="" type="checkbox"/> Annual <input checked="" type="checkbox"/> Registration <input type="checkbox"/> Return from CMI / EE Visual Inspection: <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight Other Equipment/ Accessories: <input checked="" type="checkbox"/> Power cord <input type="checkbox"/> Printer Cable <input checked="" type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable Notes: <u>Instrument came with two power cords.</u> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/>	Quality Checks By <u>RAW</u> Date <u>08/24/21</u> <input checked="" type="checkbox"/> Breath Tube Screen <input checked="" type="checkbox"/> Replace External O-Rings <input checked="" type="checkbox"/> Instrument Set Up Verified <input checked="" type="checkbox"/> R-Value <u>177</u> <input checked="" type="checkbox"/> Flow Verification (L/s) Flow Column # <u>ATP105</u> 32 mm <u>.148</u> (.139 - .169) 36 mm <u>.171</u> (.156 - .190) 53 mm <u>.246</u> (.228 - .278) 103 mm <u>.523</u> (.447 - .547) <input checked="" type="checkbox"/> Barometric Pressure Check Gauge ID # <u>26932</u> <input checked="" type="checkbox"/> Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td rowspan="2">0.050</td> <td rowspan="2">MP6296</td> <td>202010A</td> </tr> <tr> <td>10-05-22</td> </tr> <tr> <td rowspan="2">0.080</td> <td rowspan="2">MP6297</td> <td>202010B</td> </tr> <tr> <td>10-05-22</td> </tr> <tr> <td rowspan="2">0.200</td> <td rowspan="2">MP6298</td> <td>202010D</td> </tr> <tr> <td>10-06-22</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG011102</td> </tr> <tr> <td></td> <td></td> <td>04-20-22</td> </tr> </tbody> </table>	Simulator	Serial #	Lot #/Exp	0.050	MP6296	202010A	10-05-22	0.080	MP6297	202010B	10-05-22	0.200	MP6298	202010D	10-06-22	0.080 DGS	N/A	AG011102			04-20-22	Flow Calibration By _____ Date _____ Flow Column # _____ <input type="checkbox"/> 5L/min – 17mm <input type="checkbox"/> 15L/min – 53mm <input type="checkbox"/> 30L/min – 103mm <input type="checkbox"/> R-Value _____ <input type="checkbox"/> Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547) Maintenance By _____ <input type="checkbox"/> Battery Replacement <input type="checkbox"/> Dry Gas Regulator Replacement <input type="checkbox"/> Breath Tube Replacement <input type="checkbox"/> Other _____ DI Temp. Checks By <u>RAW</u> <input checked="" type="checkbox"/> Lab Temp °C <u>22.05</u> External Digital Therm. ID#: <u>569598</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>MP6296</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>MP6297</u> <input checked="" type="checkbox"/> 34°C +/- .2 Serial #: <u>MP6298</u>																																						
Simulator	Serial #	Lot #/Exp																																																											
0.050	MP6296	202010A																																																											
		10-05-22																																																											
0.080	MP6297	202010B																																																											
		10-05-22																																																											
0.200	MP6298	202010D																																																											
		10-06-22																																																											
0.080 DGS	N/A	AG011102																																																											
		04-20-22																																																											
Calibration Adjustment By _____ Barometric Pressure Gauge _____ ID # _____ <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> <input type="checkbox"/> Post Calibration Adjustment Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			Department Inspection By <u>RAW</u> Barometric Pressure ID# <u>26932</u> Gauge <u>1013</u> Instrument <u>1012</u> Mouth Alcohol Solution Lot # <u>2021-B</u> Acetone Stock Solution Lot # <u>2021-C</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6294</td> </tr> <tr> <td>Interferent</td> <td>MP6295</td> </tr> <tr> <td>0.050</td> <td>MP6296</td> </tr> <tr> <td>0.080</td> <td>MP6297</td> </tr> <tr> <td>0.200</td> <td>MP6298</td> </tr> </tbody> </table>	Simulator	Serial Number	0.000	MP6294	Interferent	MP6295	0.050	MP6296	0.080	MP6297	0.200	MP6298
Simulator	Serial #	Lot #	Expiration																																																										
0.000		N/A	N/A																																																										
0.040																																																													
0.100																																																													
0.200																																																													
0.300																																																													
0.080 DGS	N/A																																																												
Simulator	Serial #	Lot #	Expiration																																																										
0.050																																																													
0.080																																																													
0.200																																																													
0.080 DGS	N/A																																																												
Simulator	Serial Number																																																												
0.000	MP6294																																																												
Interferent	MP6295																																																												
0.050	MP6296																																																												
0.080	MP6297																																																												
0.200	MP6298																																																												
Notes/Suggested Service: _____ <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/> <hr/>	Attachments <input checked="" type="checkbox"/> Form 41 <input type="checkbox"/> Post-Stability Checks <input checked="" type="checkbox"/> Stability Checks <input type="checkbox"/> Flow Calibration <input checked="" type="checkbox"/> Calibration Certificate <input type="checkbox"/> Form 40 <input type="checkbox"/> Calibration Adjustment <input checked="" type="checkbox"/> Other <u>Form 47</u> <input checked="" type="checkbox"/> Instrument Complies with Chapter 11D-8, FAC <input type="checkbox"/> Instrument Does Not Comply with Chapter 11D-8, FAC <input checked="" type="checkbox"/> Return to/Place into Evidentiary Use <input type="checkbox"/> Remain Out of Evidentiary Use <input checked="" type="checkbox"/> Conduct an Agency Inspection Before Evidentiary Use Israel Soto <small>Digitally signed by Israel Soto Date: 2021.10.07 09:17:21 -04'00'</small> 2021.10.07 11:14:25 -04'00' Tech Review / Date _____ Admin Review / Date _____																																																												

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: FFWCC

Time of Inspection: 13:45

Date of Inspection: 08/24/2021

Serial Number: 80-007500

Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202010A Exp: 10/05/2022	0.08g/210L Test (g/210L) Lot#:202010B Exp: 10/05/2022	0.20g/210L Test (g/210L) Lot#:202010D Exp: 10/06/2022	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG011102 Exp: 04/20/2022
0.000	0.049	0.079	0.199	0.077
0.000	0.049	0.079	0.198	0.078
0.000	0.049	0.079	0.199	0.077
0.000	0.049	0.079	0.199	0.077
0.000	0.049	0.080	0.198	0.077
0.000	0.049	0.080	0.199	0.077
0.000	0.050	0.080	0.199	0.077
0.000	0.050	0.080	0.199	0.077
0.000	0.050	0.080	0.198	0.077
0.000	0.050	0.080	0.199	0.078


Standard Deviations	0.0005	0.0005	0.0004	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies () does not comply () with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

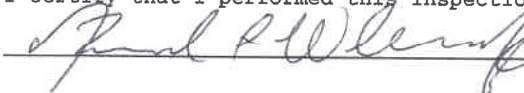

 Signature and Printed Name

RICHARD A WILLIAMS

08/24/2021
 Date

FFWCC
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-007500
 08/24/2021
 Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:43
Control Test	0.049	10:44
Air Blank	0.000	10:45
Control Test	0.049	10:45
Air Blank	0.000	10:46
Air Blank	0.000	10:47
Control Test Stats		
Average	0.0490	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

RAW

Operator's Signature

FFWCC
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-007500
 08/24/2021
 Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:54
Control Test	0.079	10:55
Air Blank	0.000	10:55
Control Test	0.079	10:56
Air Blank	0.000	10:57
Control Test	0.079	10:57
Air Blank	0.000	10:58
Control Test Stats		
Average	0.0790	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

Wet

RAW

Operator's Signature

FFWCC
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-007500
 08/24/2021
 Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	11:01
Control Test	0.199	11:02
Air Blank	0.000	11:02
Control Test	0.199	11:03
Air Blank	0.000	11:04
Control Test	0.198	11:04
Air Blank	0.000	11:05
Control Test Stats		
Average	0.1987	
Std Dev	0.0006	
Rel Std Dev(%)	0.2906	

RAW

Operator's Signature

FFWCC
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-007500
 08/24/2021
 Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	11:08
Control Test	0.078	11:09
Air Blank	0.000	11:09
Control Test	0.077	11:09
Air Blank	0.000	11:10
Control Test	0.077	11:10
Air Blank	0.000	11:11
Control Test Stats		
Average	0.0773	
Std Dev	0.0006	
Rel Std Dev(%)	0.7466	

Stability checks
 80-007500

DGS

RAW

Operator's Signature



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2729 Fort Knox Blvd.
Bldg. 2, Suite 1300
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-007500, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-007500</u>	UNCERTAINTY* ±	
Owning Agency:	<u>FFWCC</u>	0.050 g/ 210 L	0.005
Calibration Date:	<u>08/24/2021</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>13:45</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Thermometer temperatures are checked with NIST traceable Eutechnics 4400 digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the uses of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Richard A Williams
Digitally signed by Richard A Williams
Date: 2021.10.06 16:50:17 -0400

08/24/2021

Date

RICHARD A WILLIAMS,
Department Inspector

FDLE/ATP Form 69 January 2021

Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality



REQUEST FOR REGISTRATION

Florida Department of
Law Enforcement

MAKE AND MODEL OF INSTRUMENT: Intoxilyzer 8000

SERIAL NUMBER: 80-007500

OWNING AGENCY: FFWCC

DATE OF DEPARTMENT INSPECTION: 08/24/2021

AGENCY INSPECTOR: Richard Gillmore

ADDRESS: 644 Cesery Blvd Suite 3300


CITY, STATE, ZIP: Jacksonville, FL 32211

TELEPHONE NUMBER: 904-716-5958

FAX NUMBER: _____

EMAIL ADDRESS (if available): richard.gilmore@myfwc.com

For Program Office Use Only:

-  **Registration Issued**
- Instrument Added to Evidentiary Instrument Database
 - Instrument Added to Monthly Statistics Database
 - Contact Information Added to Instrument Database