



INSTRUMENT PROCESSING SHEET

Agency New Smyrna Beach Police Department

S/N 80-001152

Florida Department of
Law Enforcement

Date In 03-23-2022 DI Completion Date 03-30-2022

☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake By <u>IS</u> ☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/ Accessories: ☐ Power cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes: Instrument failed Interferent Detect test twice during last Agency Inspection, did not produce Interferent Detect for every test. Will perform Optical Bench Calibration Adjust.	Quality Checks By <u>IS</u> Date <u>03-30-2022</u> ☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>189</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP-102</u> 32 mm <u>0.160</u> (.139 - .169) 36 mm <u>0.175</u> (.156 - .190) 53 mm <u>0.242</u> (.228 - .278) 103 mm <u>0.500</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>28421</u> ☒ Stability Checks	Flow Calibration By <u>IS</u> Date <u>03-30-2022</u> Flow Column # <u>ATP-102</u> ☐ 5L/min - 17mm ☐ 15L/min - 53mm ☐ 30L/min - 103mm ☐ R-Value <u>189</u> ☐ Post Calibration Verification (L/s) Flow Column # <u>ATP-102</u> 32 mm <u>0.160</u> (.139 - .169) 36 mm <u>0.175</u> (.156 - .190) 53 mm <u>0.242</u> (.228 - .278) 103 mm <u>0.500</u> (.447 - .547)																																																													
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>MP6291</td> <td>202201C 01-11-2024</td> </tr> <tr> <td>0.080</td> <td>MP6292</td> <td>202201D 01-18-2024</td> </tr> <tr> <td>0.200</td> <td>MP6293</td> <td>202201E 01-18-2024</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG113403 05-14-2023</td> </tr> </tbody> </table>		Simulator	Serial #	Lot #/Exp	0.050	MP6291	202201C 01-11-2024	0.080	MP6292	202201D 01-18-2024	0.200	MP6293	202201E 01-18-2024	0.080 DGS	N/A	AG113403 05-14-2023	Maintenance By <u>IS</u> ☒ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Breath Tube Replacement ☒ Other <u>Replaced internal printer paper</u>																																														
Simulator	Serial #	Lot #/Exp																																																													
0.050	MP6291	202201C 01-11-2024																																																													
0.080	MP6292	202201D 01-18-2024																																																													
0.200	MP6293	202201E 01-18-2024																																																													
0.080 DGS	N/A	AG113403 05-14-2023																																																													
Calibration Adjustment By <u>IS</u> Barometric Pressure Gauge <u>1012</u> ID # <u>28662</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6294</td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td>MP6295/MP5082</td> <td>21070</td> <td>03-01-2023</td> </tr> <tr> <td>0.100</td> <td>MP5084</td> <td>21080</td> <td>03-08-2023</td> </tr> <tr> <td>0.200</td> <td>MP6297</td> <td>20510</td> <td>12-03-2022</td> </tr> <tr> <td>0.300</td> <td>MP6298</td> <td>21420</td> <td>10-20-2023</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>08121080A1</td> <td>05-05-2023</td> </tr> </tbody> </table> ☒ Post Calibration Adjustment Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>MP6291</td> <td>202201C</td> <td>01-11-2024</td> </tr> <tr> <td>0.080</td> <td>MP6292</td> <td>202201D</td> <td>01-18-2024</td> </tr> <tr> <td>0.200</td> <td>MP6293</td> <td>202201E</td> <td>01-18-2024</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG113403</td> <td>05-14-2023</td> </tr> </tbody> </table>		Simulator	Serial #	Lot #	Expiration	0.000	MP6294	N/A	N/A	0.040	MP6295/MP5082	21070	03-01-2023	0.100	MP5084	21080	03-08-2023	0.200	MP6297	20510	12-03-2022	0.300	MP6298	21420	10-20-2023	0.080 DGS	N/A	08121080A1	05-05-2023	Simulator	Serial #	Lot #	Expiration	0.050	MP6291	202201C	01-11-2024	0.080	MP6292	202201D	01-18-2024	0.200	MP6293	202201E	01-18-2024	0.080 DGS	N/A	AG113403	05-14-2023	Department Inspection By <u>IS</u> Barometric Pressure ID# <u>30793</u> Gauge <u>1014</u> Instrument <u>1011</u> Mouth Alcohol Solution Lot # <u>2021-D</u> Acetone Stock Solution Lot # <u>2021-C</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6289</td> </tr> <tr> <td>Interferent</td> <td>MP6290</td> </tr> <tr> <td>0.050</td> <td>MP6291</td> </tr> <tr> <td>0.080</td> <td>MP6292</td> </tr> <tr> <td>0.200</td> <td>MP6293</td> </tr> </tbody> </table> Attachments ☒ Form 41 ☒ Post-Stability Checks ☒ Stability Checks ☐ Flow Calibration ☒ Calibration Certificate ☐ Form 40 ☒ Calibration Adjustment ☐ Other _____ ☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use David Eliezer Reyes-Rivera Digitally signed by David Eliezer Reyes-Rivera Date: 2022.03.31 10:21:45 -0400 01 13:59:59 -04'00' Tech Review / Date Admin Review / Date		Simulator	Serial Number	0.000	MP6289	Interferent	MP6290	0.050	MP6291	0.080	MP6292	0.200	MP6293
Simulator	Serial #	Lot #	Expiration																																																												
0.000	MP6294	N/A	N/A																																																												
0.040	MP6295/MP5082	21070	03-01-2023																																																												
0.100	MP5084	21080	03-08-2023																																																												
0.200	MP6297	20510	12-03-2022																																																												
0.300	MP6298	21420	10-20-2023																																																												
0.080 DGS	N/A	08121080A1	05-05-2023																																																												
Simulator	Serial #	Lot #	Expiration																																																												
0.050	MP6291	202201C	01-11-2024																																																												
0.080	MP6292	202201D	01-18-2024																																																												
0.200	MP6293	202201E	01-18-2024																																																												
0.080 DGS	N/A	AG113403	05-14-2023																																																												
Simulator	Serial Number																																																														
0.000	MP6289																																																														
Interferent	MP6290																																																														
0.050	MP6291																																																														
0.080	MP6292																																																														
0.200	MP6293																																																														
Notes/Suggested Service: <u>Logged into instrument numerous times to ensure time/date was kept after changing battery.</u> <u>Standard Deviation out of tolerance for 0.040 test during calibration adjust, simulator o-ring not seated properly and simulator had an air leak. Replaced MP6295 simulator with MP5082 and reran test. Adjust was successful. IS</u>																																																															

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: NEW SMYRNA BEACH PD
Time of Inspection: 15:03

Date of Inspection: 03/30/2022

Serial Number: 80-001152
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202201C Exp: 01/11/2024	0.08g/210L Test (g/210L) Lot#:202201D Exp: 01/18/2024	0.20g/210L Test (g/210L) Lot#:202201E Exp: 01/18/2024	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG113403 Exp: 05/14/2023
0.000	0.049	0.080	0.202	0.080
0.000	0.049	0.080	0.202	0.080
0.000	0.049	0.080	0.202	0.080
0.000	0.049	0.080	0.202	0.080
0.000	0.049	0.080	0.202	0.080
0.000	0.048	0.080	0.203	0.080
0.000	0.047	0.081	0.203	0.080
0.000	0.048	0.081	0.202	0.080
0.000	0.048	0.080	0.203	0.080
0.000	0.048	0.079	0.202	0.080

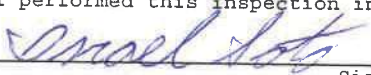
Standard Deviations	0.0006	0.0005	0.0004	0.0000
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0003 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

 ISRAEL SOTO
Signature and Printed Name

03/30/2022
Date

Stability Checks

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:51
Control Test	0.049	10:51
Air Blank	0.000	10:52
Control Test	0.049	10:52
Air Blank	0.000	10:53
Control Test	0.049	10:54
Air Blank	0.000	10:54
Control Test Stats		
Average	0.0490	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	



Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:56
Control Test	0.080	10:56
Air Blank	0.000	10:57
Control Test	0.079	10:58
Air Blank	0.000	10:58
Control Test	0.080	10:59
Air Blank	0.000	10:59
Control Test Stats		
Average	0.0797	
Std Dev	0.0006	
Rel Std Dev(%)	0.7247	

Wet



Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	11:01
Control Test	0.203	11:01
Air Blank	0.000	11:02
Control Test	0.201	11:03
Air Blank	0.000	11:03
Control Test	0.201	11:04
Air Blank	0.000	11:04
Control Test Stats		
Average	0.2017	
Std Dev	0.0012	
Rel Std Dev(%)	0.5726	



Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	11:08
Control Test	0.080	11:08
Air Blank	0.000	11:09
Control Test	0.080	11:09
Air Blank	0.000	11:10
Control Test	0.081	11:10
Air Blank	0.000	11:11
Control Test Stats		
Average	0.0803	
Std Dev	0.0006	
Rel Std Dev(%)	0.7187	

Dry



Operator's Signature



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road.
Suite B1032
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001152, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001152</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>NEW SMYRNA BEACH PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>03/30/2022</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>15:03</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Israel Soto
Soto
Digitally signed by Israel Soto
Date: 2022.03.30 15:16:09
-04'00'

03/30/2022

Date

ISRAEL SOTO,
Department Inspector

FDLE/ATP Form 69 March 2022
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality

NEW SMYRNA BEACH PD
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-001152
 03/30/2022 11:19:43

Optical Bench
 Cal. Adj *df*

Auto Calibration

pg 1 of 2

<<<<< 3um >>>>>			<<<<< 9um >>>>>		
Solution = 0.000 g/210L or 0.0000 mg/l, Samples = 4, Discarded = 1					
Sample	% Abs	(% Abs Ref)	% Abs	(% Abs Ref)	
Sample #1	0.1260	(-0.0240)	0.1170	(-0.0030)	
Sample #2	0.1140	(0.0220)	0.1040	(0.0100)	
Sample #3	0.0860	(0.0240)	0.0950	(0.0110)	
Sample #4	0.0760	(0.0260)	0.1030	(-0.0020)	
Avg % Abs	0.0920	(0.0240)	0.1007	(0.0063)	
STD DEV	0.0197	(0.0020)	0.0049	(0.0072)	
REL STD DEV	21.411	(8.333)	4.900	(114.224)	

Solution = 0.040 g/210L or 0.1905 mg/l, Samples = 4, Discarded = 1					
Sample	% Abs	(% Abs Ref)	% Abs	(% Abs Ref)	
Sample #1	0.8360	(0.0000)	1.5760	(0.0010)	
Sample #2	0.8340	(0.0020)	1.5560	(0.0030)	
Sample #3	0.8270	(0.0060)	1.5610	(0.0010)	
Sample #4	0.8390	(0.0310)	1.5470	(0.0300)	
Avg % Abs	0.8333	(0.0130)	1.5547	(0.0113)	
STD DEV	0.0060	(0.0157)	0.0071	(0.0162)	
REL STD DEV	0.723	(120.894)	0.456	(142.912)	

Solution = 0.100 g/210L or 0.4762 mg/l, Samples = 4, Discarded = 1					
Sample	% Abs	(% Abs Ref)	% Abs	(% Abs Ref)	
Sample #1	1.9230	(-0.0120)	3.6540	(-0.0110)	
Sample #2	1.9320	(-0.0120)	3.6440	(-0.0060)	
Sample #3	1.9060	(0.0140)	3.6390	(0.0010)	
Sample #4	1.9200	(0.0000)	3.6500	(-0.0070)	
Avg % Abs	1.9193	(0.0007)	3.6443	(-0.0040)	
STD DEV	0.0130	(0.0130)	0.0055	(0.0044)	
REL STD DEV	0.678	(1951.922)	0.151	(108.972)	

Solution = 0.200 g/210L or 0.9524 mg/l, Samples = 4, Discarded = 1					
Sample	% Abs	(% Abs Ref)	% Abs	(% Abs Ref)	
Sample #1	3.7810	(-0.0170)	7.0330	(0.0000)	
Sample #2	3.7260	(0.0410)	7.0010	(0.0320)	
Sample #3	3.7270	(0.0290)	6.9880	(0.0350)	
Sample #4	3.7250	(0.0310)	6.9880	(0.0400)	
Avg % Abs	3.7260	(0.0337)	6.9923	(0.0357)	
STD DEV	0.0010	(0.0064)	0.0075	(0.0040)	
REL STD DEV	0.027	(19.096)	0.107	(11.331)	

Solution = 0.300 g/210L or 1.4286 mg/l, Samples = 4, Discarded = 1					
Sample	% Abs	(% Abs Ref)	% Abs	(% Abs Ref)	
Sample #1	5.5320	(-0.0130)	10.2340	(-0.0120)	
Sample #2	5.4660	(0.0500)	10.1950	(0.0510)	
Sample #3	5.4080	(0.1190)	10.0860	(0.1370)	
Sample #4	5.4130	(0.1080)	10.1330	(0.1050)	
Avg % Abs	5.4290	(0.0923)	10.1380	(0.0977)	
STD DEV	0.0321	(0.0371)	0.0547	(0.0435)	
REL STD DEV	0.592	(40.150)	0.539	(44.505)	

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022 11:19:43

Optical Bench
Cal. Adj. 

Auto Calibration

pg 2 of 2

<<<<<	3um	>>>>>	<<<<<	9um	>>>>>
Zero Order Coef	-220.72		-130.80		
First Order Coef	2527.66		1293.38		
Second Order Coef	26.15		12.66		

Act	Fit	Residual	Act	Fit	Residual
(g/210L)	(g/210L)	(g/210L)	(g/210L)	(g/210L)	(g/210L)
0.000	0.000	-0.0003	0.000	-0.000	0.0000
0.040	0.040	0.0000	0.040	0.040	-0.0001
0.100	0.099	0.0007	0.100	0.100	0.0002
0.200	0.201	-0.0008	0.200	0.200	-0.0002
0.300	0.300	0.0003	0.300	0.300	0.0001

<<<<<	3um	>>>>>	<<<<<	9um	>>>>>
Solution = 0.080 g/210L or 0.3810 mg/l, Samples = 4, Discarded = 1					
Sample					
Sample #1	3205.00		3405.00		
Sample #2	3137.00		3373.00		
Sample #3	3242.00		3431.00		
Sample #4	3137.00		3427.00		
Avg	3172.0000		3410.3333		
STD DEV	60.6218		32.3934		
REL STD DEV	1.911		0.950		
H2O adjust (mg/l*10k)	637		399		

Barometric Pressure = 1011

*****CALIBRATION SUCCESSFUL*****

Sol Value = 0.040 g/210L ***
Fit Value = 0.1905 mg/l %%%
Samples Taken = 4, Discarded = 1
3um lo = 11715, 9um lo = 13123
<<<<< CHANNEL 1 >>>>>
Sample % Abs (% Abs Ref)
Sample #1 = 0.0010 (-0.0180)
Sample #2 = 0.0200 (-0.0030)
Sample #3 = 0.0230 (-0.0060)
Sample #4 = 0.0050 (0.0090)
Avg % Abs = 0.0160 (-0.0000)
STD DEV = 0.0096 (0.0079)
REL STD DEV = 60.273 (5113536512.000)

Sol Value = 0.040 g/210L ***
Fit Value = 0.1905 mg/l %%%
Samples Taken = 4, Discarded = 1
3um lo = 11713, 9um lo = 13121
<<<<< CHANNEL 1 >>>>>
Sample % Abs (% Abs Ref)
Sample #1 = 0.0090 (-0.0090)
Sample #2 = 0.0120 (-0.0290)
Sample #3 = 0.0110 (-0.0090)
Sample #4 = 0.0240 (-0.0070)
Avg % Abs = 0.0157 (-0.0150)
STD DEV = 0.0072 (0.0122)
REL STD DEV = 46.176 (81.104)

<<<<< CHANNEL 2 >>>>>
Sample % Abs (% Abs Ref)
Sample #1 = -0.0130 (-0.0040)
Sample #2 = -0.0050 (0.0050)
Sample #3 = -0.0100 (0.0030)
Sample #4 = 0.0000 (0.0000)
Avg % Abs = -0.0050 (0.0027)
STD DEV = 0.0050 (0.0025)
REL STD DEV = 100.000 (94.373)

<<<<< CHANNEL 2 >>>>>
Sample % Abs (% Abs Ref)
Sample #1 = 0.0110 (-0.0040)
Sample #2 = 0.0070 (-0.0070)
Sample #3 = 0.0040 (0.0080)
Sample #4 = 0.0290 (-0.0070)
Avg % Abs = 0.0133 (-0.0020)
STD DEV = 0.0137 (0.0087)
REL STD DEV = 102.378 (433.013)

First 2 0.040 tests
STD out of tolerance,
Simulator o-ring not
seated properly, simulator
had air leak

Replaced simulator, nominal
results on first page

Post Stability Checks

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	13:13
Control Test	0.050	13:14
Air Blank	0.000	13:14
Control Test	0.050	13:15
Air Blank	0.000	13:16
Control Test	0.049	13:16
Air Blank	0.000	13:17
Control Test Stats		
Average	0.0497	
Std Dev	0.0006	
Rel Std Dev(%)	1.1625	

Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	13:18
Control Test	0.079	13:18
Air Blank	0.000	13:19
Control Test	0.079	13:20
Air Blank	0.000	13:20
Control Test	0.080	13:21
Air Blank	0.000	13:22
Control Test Stats		
Average	0.0793	
Std Dev	0.0006	
Rel Std Dev(%)	0.7277	

Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	13:23
Control Test	0.202	13:23
Air Blank	0.000	13:24
Control Test	0.200	13:25
Air Blank	0.000	13:25
Control Test	0.200	13:26
Air Blank	0.000	13:26
Control Test Stats		
Average	0.2007	
Std Dev	0.0012	
Rel Std Dev(%)	0.5754	

Operator's Signature

NEW SMYRNA BEACH PD
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001152
03/30/2022
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	13:27
Control Test	0.080	13:28
Air Blank	0.000	13:28
Control Test	0.080	13:29
Air Blank	0.000	13:29
Control Test	0.080	13:29
Air Blank	0.000	13:30
Control Test Stats		
Average	0.0800	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

Operator's Signature

wet

Dry