



INSTRUMENT PROCESSING SHEET

Agency Pasco County SOS/N 80-001049Florida Department of
Law EnforcementDate In 12-6-2023DI Completion Date 12-6-2023☐ Ship ☒ P/U ☐ H/D ☐ CMI ☐ EE

Intake	By PN	Quality Checks	By IS	Date <u>12-6-2023</u>	Flow Calibration	By	Date																																										
☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/ Accessories: ☐ Power cord ☐ Printer Cable ☐ Static Bag ☐ 12V DC Cable Notes: _____ _____ _____ _____ _____ _____ _____ _____ _____ _____		☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>214</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP-105</u> 32 mm <u>0.167</u> (.139 - .169) 36 mm <u>0.183</u> (.156 - .190) 53 mm <u>0.250</u> (.228 - .278) 103 mm <u>0.515</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>28421</u> ☒ Stability Checks			☐ Flow Column # _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value _____ ☐ Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547)																																												
		<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #/Exp</th></tr></thead><tbody><tr><td>0.050</td><td>MP6291</td><td>202303K 03-29-2025</td></tr><tr><td>0.080</td><td>MP6292</td><td>202303L 03-29-2025</td></tr><tr><td>0.200</td><td>MP6293</td><td>202304C 04-05-2025</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>AG229803 10-25-2024</td></tr></tbody></table>			Simulator	Serial #	Lot #/Exp	0.050	MP6291	202303K 03-29-2025	0.080	MP6292	202303L 03-29-2025	0.200	MP6293	202304C 04-05-2025	0.080 DGS	N/A	AG229803 10-25-2024	<table border="1"><thead><tr><th colspan="2">Maintenance</th><th>By</th></tr></thead><tbody><tr><td>☐ Battery Replacement</td><td></td><td></td></tr><tr><td>☐ Dry Gas Regulator Replacement</td><td></td><td></td></tr><tr><td>☐ Breath Tube Replacement</td><td></td><td></td></tr><tr><td>☐ Other _____</td><td></td><td></td></tr><tr><td>_____</td><td></td><td></td></tr><tr><td>_____</td><td></td><td></td></tr><tr><td>_____</td><td></td><td></td></tr><tr><td>_____</td><td></td><td></td></tr></tbody></table>			Maintenance		By	☐ Battery Replacement			☐ Dry Gas Regulator Replacement			☐ Breath Tube Replacement			☐ Other _____			_____			_____			_____			_____		
Simulator	Serial #	Lot #/Exp																																															
0.050	MP6291	202303K 03-29-2025																																															
0.080	MP6292	202303L 03-29-2025																																															
0.200	MP6293	202304C 04-05-2025																																															
0.080 DGS	N/A	AG229803 10-25-2024																																															
Maintenance		By																																															
☐ Battery Replacement																																																	
☐ Dry Gas Regulator Replacement																																																	
☐ Breath Tube Replacement																																																	
☐ Other _____																																																	

Calibration Adjustment		By IS			Department Inspection																																												
Barometric Pressure Gauge <u>1023</u> ID # <u>28663</u>					Barometric Pressure ID# <u>28427</u>																																												
<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.000</td><td>MP5091</td><td>N/A</td><td>N/A</td></tr><tr><td>0.040</td><td>MP5082</td><td>22460</td><td>12-28-2024</td></tr><tr><td>0.100</td><td>MP5086</td><td>22430</td><td>11-30-2024</td></tr><tr><td>0.200</td><td>MP5084</td><td>22400</td><td>10-12-2024</td></tr><tr><td>0.300</td><td>MP5085</td><td>23070</td><td>03-06-2025</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>06723080A5</td><td>04-05-2025</td></tr></tbody></table>		Simulator	Serial #	Lot #	Expiration	0.000	MP5091	N/A	N/A	0.040	MP5082	22460	12-28-2024	0.100	MP5086	22430	11-30-2024	0.200	MP5084	22400	10-12-2024	0.300	MP5085	23070	03-06-2025	0.080 DGS	N/A	06723080A5	04-05-2025				Gauge <u>1018</u> Instrument <u>1022</u>																
Simulator	Serial #	Lot #	Expiration																																														
0.000	MP5091	N/A	N/A																																														
0.040	MP5082	22460	12-28-2024																																														
0.100	MP5086	22430	11-30-2024																																														
0.200	MP5084	22400	10-12-2024																																														
0.300	MP5085	23070	03-06-2025																																														
0.080 DGS	N/A	06723080A5	04-05-2025																																														
					Mouth Alcohol Solution Lot # <u>2023-A</u>																																												
					Acetone Stock Solution Lot # <u>2022-B</u>																																												
					<table border="1"><thead><tr><th>Simulator</th><th>Serial Number</th></tr></thead><tbody><tr><td>0.000</td><td>MP6289</td></tr><tr><td>Interferent</td><td>MP6290</td></tr><tr><td>0.050</td><td>MP6291</td></tr><tr><td>0.080</td><td>MP6292</td></tr><tr><td>0.200</td><td>MP6293</td></tr></tbody></table>			Simulator	Serial Number	0.000	MP6289	Interferent	MP6290	0.050	MP6291	0.080	MP6292	0.200	MP6293																														
Simulator	Serial Number																																																
0.000	MP6289																																																
Interferent	MP6290																																																
0.050	MP6291																																																
0.080	MP6292																																																
0.200	MP6293																																																
☒ Post Calibration Adjustment Stability Checks					Attachments																																												
<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.050</td><td>MP6291</td><td>202303K</td><td>03-29-2025</td></tr><tr><td>0.080</td><td>MP6292</td><td>202303L</td><td>03-29-2025</td></tr><tr><td>0.200</td><td>MP6293</td><td>202304C</td><td>04-05-2025</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>AG229803</td><td>10-25-2024</td></tr></tbody></table>		Simulator	Serial #	Lot #	Expiration	0.050	MP6291	202303K	03-29-2025	0.080	MP6292	202303L	03-29-2025	0.200	MP6293	202304C	04-05-2025	0.080 DGS	N/A	AG229803	10-25-2024				☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☒ Calibration Adjustment																								
Simulator	Serial #	Lot #	Expiration																																														
0.050	MP6291	202303K	03-29-2025																																														
0.080	MP6292	202303L	03-29-2025																																														
0.200	MP6293	202304C	04-05-2025																																														
0.080 DGS	N/A	AG229803	10-25-2024																																														
					☒ Post-Stability Checks ☐ Flow Calibration ☒ Form 40 ☐ Other _____																																												
Notes/Suggested Service: <u>Agency Inspector indicated instrument did not pass Interferent Detect test during Agency Inspection. Optical Bench Calibration Adjustment performed. IS</u>					☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use																																												
					Benjamin Siddoway Digitally signed by Benjamin Siddoway Date: 2023.12.06 15:23:36 -05'00'																																												
					Phil Nicodemo Digitally signed by Phil Nicodemo Date: 2023.12.06 15:44:15 -05'00'																																												
					Tech Review / Date _____ Admin Review / Date _____																																												

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: PASCO COUNTY SO
Time of Inspection: 15:16

Date of Inspection: 12/06/2023

Serial Number: 80-001049
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202303K Exp: 03/29/2025	0.08g/210L Test (g/210L) Lot#:202303L Exp: 03/29/2025	0.20g/210L Test (g/210L) Lot#:202304C Exp: 04/05/2025	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG229803 Exp: 10/25/2024
0.000	0.048	0.078	0.197	0.079
0.000	0.049	0.078	0.196	0.079
0.000	0.049	0.078	0.197	0.078
0.000	0.049	0.079	0.196	0.079
0.000	0.049	0.079	0.197	0.078
0.000	0.048	0.078	0.197	0.079
0.000	0.048	0.078	0.197	0.079
0.000	0.049	0.079	0.197	0.078
0.000	0.049	0.078	0.196	0.079
0.000	0.049	0.078	0.197	0.079

Standard Deviations	0.0004	0.0004	0.0004	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

Israel Soto

ISRAEL SOTO

Signature and Printed Name

12/06/2023
Date

Stability checks

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:02
Control Test	0.051	10:02
Air Blank	0.000	10:03
Control Test	0.050	10:04
Air Blank	0.000	10:04
Control Test	0.050	10:05
Air Blank	0.000	10:05
Control Test Stats		
Average	0.0503	
Std Dev	0.0006	
Rel Std Dev(%)	1.1471	


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:06
Control Test	0.081	10:07
Air Blank	0.000	10:08
Control Test	0.080	10:08
Air Blank	0.000	10:09
Control Test	0.080	10:10
Air Blank	0.000	10:10
Control Test Stats		
Average	0.0803	
Std Dev	0.0006	
Rel Std Dev(%)	0.7187	

wet


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:12
Control Test	0.200	10:12
Air Blank	0.000	10:13
Control Test	0.200	10:14
Air Blank	0.000	10:14
Control Test	0.200	10:15
Air Blank	0.000	10:15
Control Test Stats		
Average	0.2000	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:18
Control Test	0.080	10:18
Air Blank	0.000	10:19
Control Test	0.080	10:19
Air Blank	0.000	10:19
Control Test	0.080	10:20
Air Blank	0.000	10:20
Control Test Stats		
Average	0.0800	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

Dry


Operator's Signature



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road.
Suite B1032
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001049, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001049</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>PASCO COUNTY SO</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>12/06/2023</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>15:16</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Israel Soto
Soto
Digitally signed by Israel Soto
Date: 2023.12.06 15:21:24 -05'00'

12/06/2023

Date

ISRAEL SOTO,
Department Inspector

FDLE/ATP Form 69 March 2022
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023 10:29:48

Optical Bench
Calibration Adjustment

✓

pg 1 of 2

Auto Calibration

	<<<<<	3um	>>>>>	<<<<<	9um	>>>>>

Solution = 0.000 g/210L or 0.0000 mg/l, Samples = 4, Discarded = 1						
Sample	% Abs	(% Abs Ref)		% Abs	(% Abs Ref)	
Sample #1	0.1880	(-0.0400)		0.1240	(-0.0190)	
Sample #2	0.1810	(0.0150)		0.1200	(0.0000)	
Sample #3	0.1810	(0.0100)		0.1400	(-0.0160)	
Sample #4	0.1940	(0.0360)		0.1410	(0.0010)	
Avg % Abs	0.1853	(0.0203)		0.1337	(-0.0050)	
STD DEV	0.0075	(0.0138)		0.0118	(0.0095)	
REL STD DEV	4.050	(67.850)		8.863	(190.788)	

Solution = 0.040 g/210L or 0.1905 mg/l, Samples = 4, Discarded = 1						
Sample	% Abs	(% Abs Ref)		% Abs	(% Abs Ref)	
Sample #1	0.9700	(-0.0250)		1.5730	(0.0000)	
Sample #2	0.9400	(0.0050)		1.5440	(0.0090)	
Sample #3	0.9460	(0.0310)		1.5660	(0.0100)	
Sample #4	0.8930	(0.0630)		1.5450	(0.0280)	
Avg % Abs	0.9263	(0.0330)		1.5517	(0.0157)	
STD DEV	0.0290	(0.0291)		0.0124	(0.0107)	
REL STD DEV	3.133	(88.035)		0.801	(68.251)	

Solution = 0.100 g/210L or 0.4762 mg/l, Samples = 4, Discarded = 1						
Sample	% Abs	(% Abs Ref)		% Abs	(% Abs Ref)	
Sample #1	2.0760	(-0.0080)		3.6310	(0.0010)	
Sample #2	2.0480	(0.0360)		3.6090	(0.0280)	
Sample #3	2.0890	(0.0110)		3.6220	(0.0250)	
Sample #4	2.0360	(0.0340)		3.6200	(0.0150)	
Avg % Abs	2.0577	(0.0270)		3.6170	(0.0227)	
STD DEV	0.0278	(0.0139)		0.0070	(0.0068)	
REL STD DEV	1.351	(51.453)		0.194	(30.030)	

Solution = 0.200 g/210L or 0.9524 mg/l, Samples = 4, Discarded = 1						
Sample	% Abs	(% Abs Ref)		% Abs	(% Abs Ref)	
Sample #1	3.8550	(-0.0120)		6.8800	(0.0080)	
Sample #2	3.8250	(0.0340)		6.8520	(0.0560)	
Sample #3	3.8190	(0.0240)		6.8430	(0.0550)	
Sample #4	3.8260	(0.0510)		6.8410	(0.0620)	
Avg % Abs	3.8233	(0.0363)		6.8453	(0.0577)	
STD DEV	0.0038	(0.0137)		0.0059	(0.0038)	
REL STD DEV	0.099	(37.570)		0.086	(6.565)	

Solution = 0.300 g/210L or 1.4286 mg/l, Samples = 4, Discarded = 1						
Sample	% Abs	(% Abs Ref)		% Abs	(% Abs Ref)	
Sample #1	5.5990	(-0.0110)		10.0200	(-0.0110)	
Sample #2	5.5770	(0.0030)		9.9410	(0.0520)	
Sample #3	5.5490	(0.0550)		9.9450	(0.0790)	
Sample #4	5.5560	(0.0580)		9.9260	(0.0800)	
Avg % Abs	5.5607	(0.0387)		9.9373	(0.0703)	
STD DEV	0.0146	(0.0309)		0.0100	(0.0159)	
REL STD DEV	0.262	(79.978)		0.101	(22.585)	

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023 10:29:48

Optical Bench
Calibration Adjustment
pg 2 of 2

Auto Calibration

<<<<< 3um >>>>>

Zero Order Coef -458.20
First Order Coef 2497.60
Second Order Coef 27.90

Act (g/210L)	Fit (g/210L)	Residual (g/210L)
0.000	0.000	-0.0001
0.040	0.039	0.0005
0.100	0.101	-0.0008
0.200	0.199	0.0005
0.300	0.300	-0.0001

<<<<< 9um >>>>>

-182.10
1323.47
13.39

Act (g/210L)	Fit (g/210L)	Residual (g/210L)
0.000	-0.000	0.0001
0.040	0.040	0.0000
0.100	0.100	-0.0004
0.200	0.200	0.0004
0.300	0.300	-0.0001

<<<<< 3um >>>>>

Solution = 0.080 g/210L or 0.3810 mg/l, Samples = 4, Discarded = 1
Sample

Sample #1	2849.00	3406.00
Sample #2	2877.00	3440.00
Sample #3	2814.00	3405.00
Sample #4	2801.00	3404.00
Avg	2830.6667	3416.3333
STD DEV	40.6489	20.5020
REL STD DEV	1.436	0.600
H2O adjust (mg/l*10k)	979	393

Barometric Pressure = 1022

*****CALIBRATION SUCCESSFUL*****

Post stability checks

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	12:21
Control Test	0.049	12:22
Air Blank	0.000	12:22
Control Test	0.049	12:23
Air Blank	0.000	12:24
Control Test	0.049	12:24
Air Blank	0.000	12:25
Control Test Stats		
Average	0.0490	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	12:26
Control Test	0.078	12:27
Air Blank	0.000	12:27
Control Test	0.078	12:28
Air Blank	0.000	12:29
Control Test	0.079	12:29
Air Blank	0.000	12:30
Control Test Stats		
Average	0.0783	
Std Dev	0.0006	
Rel Std Dev(%)	0.7370	


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	12:32
Control Test	0.197	12:32
Air Blank	0.000	12:33
Control Test	0.196	12:34
Air Blank	0.000	12:34
Control Test	0.196	12:35
Air Blank	0.000	12:35
Control Test Stats		
Average	0.1963	
Std Dev	0.0006	
Rel Std Dev(%)	0.2941	


Operator's Signature

wet

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
12/06/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	12:38
Control Test	0.079	12:38
Air Blank	0.000	12:39
Control Test	0.079	12:39
Air Blank	0.000	12:39
Control Test	0.079	12:40
Air Blank	0.000	12:40
Control Test Stats		
Average	0.0790	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	


Operator's Signature

Dry

Florida Department of Law Enforcement Alcohol Testing Program

AGENCY INSPECTION REPORT - INTOXILYZER 8000

Agency: PASCO COUNTY SO
Time of Inspection: 09:12

Date of Inspection: 12/06/2023

Serial Number: 80-001049
Software: 8100.27

Check or Test	YES	NO
Date and/or Time Adjusted		No
Diagnostic Check (Pre-Inspection): OK		No
Alcohol Free Subject Test: 0.000		No
Mouth Alcohol Test: Slope Not Met		No
Interferent Detect Test: Interferent Detect		No
Diagnostic Check (Post-Inspection): OK		No

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#: Exp:	0.08g/210L Test (g/210L) Lot#: Exp:	0.20g/210L Test (g/210L) Lot#: Exp:	0.08 g/210L Dry Gas Std Test (g/210L) Lot#: Exp:

Number of Simulators Used: _____

Remarks:

BYPASSED AI TO OPERATE INSTRUMENT.COMPLIANCE NOT DETERMINED

The above instrument complies (X) does not comply () with Chapter 11D-8, FAC.

I certify that I hold a valid Florida Department of Law Enforcement Agency Inspector Permit and that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.



PHIL NICODEMO

Signature and Printed Name

12/06/2023
Date

Compliance NOT DETERMINED
PN 12/6/23



Agency Pasco County Sheriff's Office

S/N 80-001049

Date In 02-07-2023 DI Completion Date 02-07-2023

☐ Ship ☒ P/U ☐ H/D ☐ CMI ☐ EE

Calibration Adjustment

By_

Barometric Pressure Gauge _____ ID # _____

☐ Post Calibration Adjustment Stability Checks

Notes/Suggested Service:

Department Inspection

By ISBarometric Pressure ID# 30793Gauge 1023 Instrument 1021Mouth Alcohol Solution Lot # 2022-A

Acetone Stock Solution Lot # 2022-B

Attachments

- Instrument Complies with Chapter 11D-8, FAC**

- ☐
- Instrument Does Not Comply with Chapter 11D-8, FAC

- #### Return to/Place into Evidentiary Use

- ☐
- Remain Out of Evidentiary Use**

- ### ■ Conduct an Agency Inspection Before Evidentiary Use

Taylor
Gutschow

Digitally signed by Taylor
Gutschow
Date: 2023.02.07 13:23:27
+05'00'

Phil Nicodemo

Digitally signed by Phil Nicodemo
Date: 2023.02.07 14:28:31 -05'00'

Tech Review / Date

Admin Review / Date

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: PASCO COUNTY SO
Time of Inspection: 12:19

Date of Inspection: 02/07/2023

Serial Number: 80-001049
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202201C Exp: 01/11/2024	0.08g/210L Test (g/210L) Lot#:202201D Exp: 01/18/2024	0.20g/210L Test (g/210L) Lot#:202201E Exp: 01/18/2024	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG229803 Exp: 10/25/2024
0.000	0.049	0.079	0.202	0.080
0.000	0.049	0.079	0.202	0.080
0.000	0.049	0.079	0.202	0.080
0.000	0.049	0.079	0.201	0.080
0.000	0.049	0.079	0.201	0.080
0.000	0.049	0.079	0.202	0.080
0.000	0.049	0.079	0.202	0.081
0.000	0.049	0.079	0.202	0.080
0.000	0.049	0.079	0.201	0.080
0.000	0.049	0.079	0.201	0.080

Standard Deviations	0.0000	0.0000	0.0005	0.0003
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0002 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

Israel Soto

ISRAEL SOTO

Signature and Printed Name

02/07/2023
Date

stability checks

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
02/07/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	09:55
Control Test	0.049	09:55
Air Blank	0.000	09:56
Control Test	0.049	09:57
Air Blank	0.000	09:57
Control Test	0.049	09:58
Air Blank	0.000	09:58
Control Test Stats		
Average	0.0490	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
02/07/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:00
Control Test	0.080	10:01
Air Blank	0.000	10:01
Control Test	0.079	10:02
Air Blank	0.000	10:02
Control Test	0.079	10:03
Air Blank	0.000	10:04
Control Test Stats		
Average	0.0793	
Std Dev	0.0006	
Rel Std Dev(%)	0.7277	

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
02/07/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:06
Control Test	0.202	10:06
Air Blank	0.000	10:07
Control Test	0.202	10:08
Air Blank	0.000	10:08
Control Test	0.201	10:09
Air Blank	0.000	10:10
Control Test Stats		
Average	0.2017	
Std Dev	0.0006	
Rel Std Dev(%)	0.2863	

wet


Operator's Signature


Operator's Signature


Operator's Signature

PASCO COUNTY SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001049
02/07/2023
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:12
Control Test	0.081	10:12
Air Blank	0.000	10:13
Control Test	0.082	10:13
Air Blank	0.000	10:14
Control Test	0.082	10:14
Air Blank	0.000	10:14
Control Test Stats		
Average	0.0817	
Std Dev	0.0006	
Rel Std Dev(%)	0.7070	

Dry


Operator's Signature



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road.
Suite B1032
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001049, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001049</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>PASCO COUNTY SO</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>02/07/2023</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>12:19</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Israel Soto
Soto
Digitally signed by Israel Soto
Date: 2023.02.07 13:10:06
-05'00'

02/07/2023

Date

ISRAEL SOTO,
Department Inspector

FDLE/ATP Form 69 March 2022
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality