

INSTRUMENT PROCESSING SHEET

Agency Indian Harbour PD

S/N 80-001263

Florida Department of
Law Enforcement

Date In 07/03/2024 DI Completion Date 07/09/2024

☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake By <u>TDG</u> Date <u>07/09/2024</u> ☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/ Accessories: ☐ Power cord ☐ Printer Cable ☐ Static Bag ☐ 12V DC Cable Notes: _____ _____ _____ _____ _____ _____ _____ _____	Quality Checks By <u>TDG</u> Date <u>07/09/2024</u> ☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>220</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>0.164</u> (.139 - .169) 36 mm <u>0.175</u> (.156 - .190) 53 mm <u>0.246</u> (.228 - .278) 103 mm <u>0.503</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>26932</u> ☒ Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> <tr> <td>0.050</td> <td>MP6286</td> <td>202303K 03/29/2025</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> <td>202303L 03/29/2025</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> <td>202304C 04/05/2025</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG222203 08/10/2024</td> </tr> </table>	Simulator	Serial #	Lot #/Exp	0.050	MP6286	202303K 03/29/2025	0.080	MP6287	202303L 03/29/2025	0.200	MP6288	202304C 04/05/2025	0.080 DGS	N/A	AG222203 08/10/2024	Flow Calibration By _____ Date _____ Flow Column # _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value _____ ☐ Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547) Maintenance By _____ Date _____ ☐ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Breath Tube Replacement ☐ Other _____ _____ _____ _____ _____																																														
Simulator	Serial #	Lot #/Exp																																																													
0.050	MP6286	202303K 03/29/2025																																																													
0.080	MP6287	202303L 03/29/2025																																																													
0.200	MP6288	202304C 04/05/2025																																																													
0.080 DGS	N/A	AG222203 08/10/2024																																																													
Calibration Adjustment By _____ Barometric Pressure Gauge _____ ID # _____ <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </table> ☐ Post Calibration Adjustment Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </table> Notes/Suggested Service: <u>After the 0.08 wet stability check failed, it was observed the o-ring had torn. Replaced the o-ring and repeated the 0.08 wet stability check. Results were within range. (TDG)</u> <u>Added static bag and plastic caps for shipping. (TDG)</u> _____ _____ _____	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			Department Inspection By <u>TDG</u> Barometric Pressure ID# <u>26932</u> Gauge <u>1018</u> Instrument <u>1017</u> Mouth Alcohol Solution Lot # <u>2023-A</u> Acetone Stock Solution Lot # <u>2023-B</u> <table border="1" style="width:100%; border-collapse: collapse;"> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> <tr> <td>0.000</td> <td>MP6284</td> </tr> <tr> <td>Interferent</td> <td>MP6285</td> </tr> <tr> <td>0.050</td> <td>MP6286</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> </tr> </table> Attachments <table style="width:100%;"> <tr> <td style="width:50%; vertical-align: top;"> ☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment </td> <td style="width:50%; vertical-align: top;"> ☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____ </td> </tr> </table> ☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use Phil Nicodemo Tech Review / Date _____ Shayla Platt Admin Review / Date _____ Digitally signed by Phil Nicodemo Date: 2024.07.15 10:13:39 -0400 Digitally signed by Shayla Platt Date: 2024.07.15 10:35:35 -0400	Simulator	Serial Number	0.000	MP6284	Interferent	MP6285	0.050	MP6286	0.080	MP6287	0.200	MP6288	☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment	☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____
Simulator	Serial #	Lot #	Expiration																																																												
0.000		N/A	N/A																																																												
0.040																																																															
0.100																																																															
0.200																																																															
0.300																																																															
0.080 DGS	N/A																																																														
Simulator	Serial #	Lot #	Expiration																																																												
0.050																																																															
0.080																																																															
0.200																																																															
0.080 DGS	N/A																																																														
Simulator	Serial Number																																																														
0.000	MP6284																																																														
Interferent	MP6285																																																														
0.050	MP6286																																																														
0.080	MP6287																																																														
0.200	MP6288																																																														
☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment	☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____																																																														

Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	0.05g/210L 0.047 to 0.053																																																																																																																																												
INDIAN HARBOUR PD Intoxilyzer - Alcohol Analyzer Model 8000 07/09/2024 Software: 8100.27 SN 80-001263 Test <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:00</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:01</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:01</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:02</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:03</td></tr> <tr><td>Control Test</td><td>0.049</td><td>11:03</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:04</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0497</td><td></td></tr> <tr><td>Std Dev</td><td>0.0016</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.1625</td><td></td></tr> </tbody> </table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	11:00	Control Test	0.050	11:01	Air Blank	0.000	11:01	Control Test	0.050	11:02	Air Blank	0.000	11:03	Control Test	0.049	11:03	Air Blank	0.000	11:04	Control Test Stats			Average	0.0497		Std Dev	0.0016		Rel Std Dev(%)	1.1625		INDIAN HARBOUR PD Intoxilyzer - Alcohol Analyzer Model 8000 07/09/2024 Software: 8100.27 SN 80-001263 Test <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:07</td></tr> <tr><td>Control Test</td><td>0.058</td><td>11:08</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:08</td></tr> <tr><td>Control Test</td><td>0.058</td><td>11:09</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:10</td></tr> <tr><td>Control Test</td><td>0.058</td><td>11:10</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:11</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0580</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	11:07	Control Test	0.058	11:08	Air Blank	0.000	11:08	Control Test	0.058	11:09	Air Blank	0.000	11:10	Control Test	0.058	11:10	Air Blank	0.000	11:11	Control Test Stats			Average	0.0580		Std Dev	0.0000		Rel Std Dev(%)	0.0000		INDIAN HARBOUR PD Intoxilyzer - Alcohol Analyzer Model 8000 07/09/2024 Software: 8100.27 SN 80-001263 Test <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:24</td></tr> <tr><td>Control Test</td><td>0.196</td><td>11:25</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:25</td></tr> <tr><td>Control Test</td><td>0.195</td><td>11:26</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:26</td></tr> <tr><td>Control Test</td><td>0.195</td><td>11:27</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:28</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1953</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2956</td><td></td></tr> </tbody> </table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	11:24	Control Test	0.196	11:25	Air Blank	0.000	11:25	Control Test	0.195	11:26	Air Blank	0.000	11:26	Control Test	0.195	11:27	Air Blank	0.000	11:28	Control Test Stats			Average	0.1953		Std Dev	0.0006		Rel Std Dev(%)	0.2956		INDIAN HARBOUR PD Intoxilyzer - Alcohol Analyzer Model 8000 07/09/2024 Software: 8100.27 SN 80-001263 Test <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>10:21</td></tr> <tr><td>Control Test</td><td>0.080</td><td>10:21</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>10:22</td></tr> <tr><td>Control Test</td><td>0.080</td><td>10:22</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>10:23</td></tr> <tr><td>Control Test</td><td>0.080</td><td>10:23</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>10:24</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0800</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature 	Test	g/210L	Time	Air Blank	0.000	10:21	Control Test	0.080	10:21	Air Blank	0.000	10:22	Control Test	0.080	10:22	Air Blank	0.000	10:23	Control Test	0.080	10:23	Air Blank	0.000	10:24	Control Test Stats			Average	0.0800		Std Dev	0.0000		Rel Std Dev(%)	0.0000	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:00																																																																																																																																																	
Control Test	0.050	11:01																																																																																																																																																	
Air Blank	0.000	11:01																																																																																																																																																	
Control Test	0.050	11:02																																																																																																																																																	
Air Blank	0.000	11:03																																																																																																																																																	
Control Test	0.049	11:03																																																																																																																																																	
Air Blank	0.000	11:04																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0497																																																																																																																																																		
Std Dev	0.0016																																																																																																																																																		
Rel Std Dev(%)	1.1625																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:07																																																																																																																																																	
Control Test	0.058	11:08																																																																																																																																																	
Air Blank	0.000	11:08																																																																																																																																																	
Control Test	0.058	11:09																																																																																																																																																	
Air Blank	0.000	11:10																																																																																																																																																	
Control Test	0.058	11:10																																																																																																																																																	
Air Blank	0.000	11:11																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0580																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	11:24																																																																																																																																																	
Control Test	0.196	11:25																																																																																																																																																	
Air Blank	0.000	11:25																																																																																																																																																	
Control Test	0.195	11:26																																																																																																																																																	
Air Blank	0.000	11:26																																																																																																																																																	
Control Test	0.195	11:27																																																																																																																																																	
Air Blank	0.000	11:28																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.1953																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.2956																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	10:21																																																																																																																																																	
Control Test	0.080	10:21																																																																																																																																																	
Air Blank	0.000	10:22																																																																																																																																																	
Control Test	0.080	10:22																																																																																																																																																	
Air Blank	0.000	10:23																																																																																																																																																	
Control Test	0.080	10:23																																																																																																																																																	
Air Blank	0.000	10:24																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0800																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		

Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083	≤0.003 of Wet																																				
Post O-ring replacement (see note on IRS) 7/19/24 #2 INDIAN HARBOR PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001263 07/09/2024 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:16</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:17</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:17</td></tr> <tr><td>Control Test</td><td>0.078</td><td>11:18</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:19</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:19</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:20</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0787</td><td></td></tr> <tr><td>Std Dev</td><td>0.0036</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7339</td><td></td></tr> </tbody> </table> Operator's Signature: 					Test	g/210L	Time	Air Blank	0.000	11:16	Control Test	0.079	11:17	Air Blank	0.000	11:17	Control Test	0.078	11:18	Air Blank	0.000	11:19	Control Test	0.079	11:19	Air Blank	0.000	11:20	Control Test Stats			Average	0.0787		Std Dev	0.0036		Rel Std Dev(%)	0.7339	
Test	g/210L	Time																																						
Air Blank	0.000	11:16																																						
Control Test	0.079	11:17																																						
Air Blank	0.000	11:17																																						
Control Test	0.078	11:18																																						
Air Blank	0.000	11:19																																						
Control Test	0.079	11:19																																						
Air Blank	0.000	11:20																																						
Control Test Stats																																								
Average	0.0787																																							
Std Dev	0.0036																																							
Rel Std Dev(%)	0.7339																																							

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: INDIAN HARBOUR PD
Time of Inspection: 15:03

Date of Inspection: 07/09/2024

Serial Number: 80-001263
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202303K Exp: 03/29/2025	0.08g/210L Test (g/210L) Lot#:202303L Exp: 03/29/2025	0.20g/210L Test (g/210L) Lot#:202304C Exp: 04/05/2025	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:01923080A3 Exp: 02/05/2025
0.000	0.049	0.078	0.195	0.079
0.000	0.049	0.078	0.196	0.079
0.000	0.049	0.078	0.196	0.080
0.000	0.049	0.079	0.195	0.079
0.000	0.049	0.078	0.196	0.079
0.000	0.049	0.079	0.195	0.079
0.000	0.049	0.079	0.196	0.080
0.000	0.049	0.078	0.196	0.080
0.000	0.050	0.078	0.196	0.079
0.000	0.050	0.078	0.196	0.079

Standard Deviations	0.0004	0.0004	0.0004	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.



TAYLOR D GUTSCHOW

Signature and Printed Name

07/09/2024
Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
4700 Terminal Drive, Suite 1
Ft. Myers, FL 33907

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001263, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001263</u>	UNCERTAINTY* $\pm$
Owning Agency:	<u>INDIAN HARBOUR PD</u>	0.050 g/ 210 L 0.004
Calibration Date:	<u>07/09/2024</u>	0.080 g/ 210 L 0.004
Calibration Time:	<u>15:03</u>	0.200 g/ 210 L 0.007
		0.080 g/ 210 L Dry Gas Control 0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards. This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

07/09/2024

TAYLOR D GUTSCHOW,
Department Inspector

FDLE/ATP Form 69 December 2021
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality