



INSTRUMENT PROCESSING SHEET

Agency Jacksonville SO

S/N 80-001278

Florida Department of
Law Enforcement

Date In 10/28/2024 DI Completion Date 10/29/2024


☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake	By <u>ALL</u>	Date <u>10/28/2024</u>	Quality Checks	By <u>DA</u>	Date <u>10/29/2024</u>	Flow Calibration	By _____	Date _____																												
☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☐ Handle ☒ Keyboard ☐ Dry Gas Shelf ☒ Feet ☐ Breath Tube ☒ Ports ☐ Screws Tight Other Equipment/ Accessories: ☒ Power cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes: <u>arrived with no keyboard</u>			☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>191</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP103</u> 32 mm <u>0.148</u> (.139 - .169) 36 mm <u>0.164</u> (.156 - .190) 53 mm <u>0.234</u> (.228 - .278) 103 mm <u>0.496</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>28427</u> ☒ Stability Checks			☐ Flow Column # _____ ☐ 5L/min - 17mm ☐ 15L/min - 53mm ☐ 30L/min - 103mm ☐ R-Value _____ ☐ Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547)																														
			<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #/Exp</th></tr></thead><tbody><tr><td>0.050</td><td>MP5088</td><td>202406K 06/19/2026</td></tr><tr><td>0.080</td><td>MP5089</td><td>202303L 03/29/2025</td></tr><tr><td>0.200</td><td>MP5090</td><td>202304C 04/05/2025</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>AG325603 09/13/2025</td></tr></tbody></table>			Simulator	Serial #	Lot #/Exp	0.050	MP5088	202406K 06/19/2026	0.080	MP5089	202303L 03/29/2025	0.200	MP5090	202304C 04/05/2025	0.080 DGS	N/A	AG325603 09/13/2025	Maintenance By _____ Date _____ ☐ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Breath Tube Replacement ☐ Other _____															
Simulator	Serial #	Lot #/Exp																																		
0.050	MP5088	202406K 06/19/2026																																		
0.080	MP5089	202303L 03/29/2025																																		
0.200	MP5090	202304C 04/05/2025																																		
0.080 DGS	N/A	AG325603 09/13/2025																																		
Calibration Adjustment By _____			Department Inspection By <u>DA</u>																																	
Barometric Pressure Gauge _____ ID # _____			Barometric Pressure ID# <u>28427</u>																																	
<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.000</td><td></td><td>N/A</td><td>N/A</td></tr><tr><td>0.040</td><td></td><td></td><td></td></tr><tr><td>0.100</td><td></td><td></td><td></td></tr><tr><td>0.200</td><td></td><td></td><td></td></tr><tr><td>0.300</td><td></td><td></td><td></td></tr><tr><td>0.080 DGS</td><td>N/A</td><td></td><td></td></tr></tbody></table>			Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Gauge <u>1021</u> Instrument <u>1021</u> Mouth Alcohol Solution Lot # <u>2024-A</u> Acetone Stock Solution Lot # <u>2023-B</u>					
Simulator	Serial #	Lot #	Expiration																																	
0.000		N/A	N/A																																	
0.040																																				
0.100																																				
0.200																																				
0.300																																				
0.080 DGS	N/A																																			
☐ Post Calibration Adjustment Stability Checks			<table border="1"><thead><tr><th>Simulator</th><th>Serial Number</th></tr></thead><tbody><tr><td>0.000</td><td>MP5086</td></tr><tr><td>Interferent</td><td>MP5087</td></tr><tr><td>0.050</td><td>MP5088</td></tr><tr><td>0.080</td><td>MP5089</td></tr><tr><td>0.200</td><td>MP5090</td></tr></tbody></table>						Simulator	Serial Number	0.000	MP5086	Interferent	MP5087	0.050	MP5088	0.080	MP5089	0.200	MP5090																
Simulator	Serial Number																																			
0.000	MP5086																																			
Interferent	MP5087																																			
0.050	MP5088																																			
0.080	MP5089																																			
0.200	MP5090																																			
<table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.050</td><td></td><td></td><td></td></tr><tr><td>0.080</td><td></td><td></td><td></td></tr><tr><td>0.200</td><td></td><td></td><td></td></tr><tr><td>0.080 DGS</td><td>N/A</td><td></td><td></td></tr></tbody></table>			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			Attachments ☒ Form 41 ☐ Post-Stability Checks ☒ Stability Checks ☐ Flow Calibration ☒ Calibration Certificate ☐ Form 40 ☐ Calibration Adjustment ☐ Other _____													
Simulator	Serial #	Lot #	Expiration																																	
0.050																																				
0.080																																				
0.200																																				
0.080 DGS	N/A																																			
Notes/Suggested Service: <u>Used FDLE keyboard to operate instrument DA</u>			☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use																																	
			Phil Nicodemo Digitally signed by Phil Nicodemo Date: 2024.10.31 08:25:22 +04'00' Shayla Platt Digitally signed by Shayla Platt Date: 2024.11.04 08:51:51 -05'00'																																	
			Tech Review / Date _____ Admin Review / Date _____																																	

Stability Checks 80-001278 10/29/24 DA

JACKSONVILLE SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001278
10/29/2024
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	10:46
Control Test	0.049	10:47
Air Blank	0.000	10:47
Control Test	0.048	10:48
Air Blank	0.000	10:49
Control Test	0.049	10:49
Air Blank	0.000	10:50
Control Test Stats		
Average	0.0487	
Std Dev	0.0006	
Rel Std Dev(%)	1.1863	

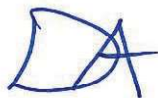

Operator's Signature

JACKSONVILLE SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001278
10/29/2024
Software: 8100.27


Test	g/210L	Time
Air Blank	0.000	10:52
Control Test	0.078	10:53
Air Blank	0.000	10:53
Control Test	0.079	10:54
Air Blank	0.000	10:55
Control Test	0.078	10:55
Air Blank	0.000	10:56
Control Test Stats		
Average	0.0783	
Std Dev	0.0006	
Rel Std Dev(%)	0.7370	

wet

Operator's Signature

JACKSONVILLE SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001278
10/29/2024
Software: 8100.27


Test	g/210L	Time
Air Blank	0.000	10:59
Control Test	0.196	11:00
Air Blank	0.000	11:01
Control Test	0.195	11:01
Air Blank	0.000	11:02
Control Test	0.195	11:03
Air Blank	0.000	11:03
Control Test Stats		
Average	0.1953	
Std Dev	0.0006	
Rel Std Dev(%)	0.2956	

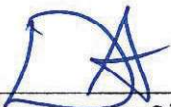
Operator's Signature

JACKSONVILLE SO
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-001278
10/29/2024
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	11:05
Control Test	0.080	11:06
Air Blank	0.000	11:06
Control Test	0.079	11:06
Air Blank	0.000	11:07
Control Test	0.080	11:07
Air Blank	0.000	11:08
Control Test Stats		
Average	0.0797	
Std Dev	0.0006	
Rel Std Dev(%)	0.7247	

DGS

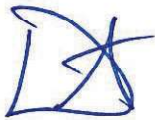

Operator's Signature

Florida Department of Law Enforcement

Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: JACKSONVILLE SO
Time of Inspection: 13:04

Date of Inspection: 10/29/2024

Serial Number: 80-001278
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202303L Exp: 03/29/2025	0.20g/210L Test (g/210L) Lot#:202304C Exp: 04/05/2025	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG325603 Exp: 09/13/2025
0.000	0.048	0.078	0.196	0.000 / 0.081
0.000	0.048	0.079	0.196	0.000 / 0.080
0.000	0.048	0.078	0.197	0.000 / 0.080
0.000	0.048	0.079	0.197	0.000 / 0.080
0.000	0.049	0.079	0.196	0.000 / 0.080
0.000	0.048	0.078	0.196	0.000 / 0.080
0.000	0.048	0.079	0.197	0.000 / 0.080
0.000	0.049	0.079	0.196	0.081 / 0.080
0.000	0.048	0.079	0.196	0.080 / 0.080
0.000	0.048	0.079	0.197	0.080 / 0.080
Standard Deviations	0.0004	0.0004	0.0005	0.0388 / 0.0003

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

08: Control Outside Tolerance- DGS NOT CONNECTED.

DGS not connected during DGS test. I connected the DGS when I realized and then repeated DGS test. 10/29/24 DA

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

DA

DESTINEE N ARMSTRONG

Signature and Printed Name

10/29/2024
Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001278 , manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001278</u>	Serial Number:	<u>80-001278</u>
Owning Agency:	<u>JACKSONVILLE SO</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>10/29/2024</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>13:04</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.
Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).
The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Digitally signed by Destinee Armstrong
Date: 2024.10.29 13:19:44 -04'00'

10/29/2024

Date

DESTINEE N ARMSTRONG,
Department Inspector

FDLE/ATP Form 69 October 2024
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality