



INSTRUMENT PROCESSING SHEET

Agency COCOA PDS/N 80-001260Florida Department of
Law EnforcementDate In 6/11/2025DI Completion Date 06/23/2025☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake	By <u>WKP</u>	Date <u>6/11/2025</u>	Quality Checks	By <u>WKP</u>	Date <u>06/17/2025</u>	Flow Calibration	By _____	Date _____																																																									
☒ Annual ☐ Registration ☐ Return from CMI / EE Visual Inspection: ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/ Accessories: ☒ Power cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes: _____ _____ _____ _____ _____ _____ _____ _____ _____ _____			☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value <u>134</u> ☒ Flow Verification (L/s) Flow Column # <u>ATP105</u> 32 mm <u>0.160</u> (.139 - .169) 36 mm <u>0.175</u> (.156 - .190) 53 mm <u>0.246</u> (.228 - .278) 103 mm <u>0.511</u> (.447 - .547) ☒ Barometric Pressure Check Gauge ID # <u>28662</u> ☒ Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #/Exp</th></tr></thead><tbody><tr><td>0.050</td><td>MP5088</td><td>202406K 06/19/2026</td></tr><tr><td>0.080</td><td>MP5089</td><td>202406L 06/19/2026</td></tr><tr><td>0.200</td><td>MP5090</td><td>202406N 06/20/2026</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>AG429602 10/22/2026</td></tr></tbody></table>			Simulator	Serial #	Lot #/Exp	0.050	MP5088	202406K 06/19/2026	0.080	MP5089	202406L 06/19/2026	0.200	MP5090	202406N 06/20/2026	0.080 DGS	N/A	AG429602 10/22/2026	Flow Column # _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value _____ ☐ Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547) Maintenance By _____ Date _____ ☐ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Breath Tube Replacement ☐ Other _____ _____ _____ _____ _____ _____																																												
						Simulator	Serial #	Lot #/Exp																																																									
						0.050	MP5088	202406K 06/19/2026																																																									
						0.080	MP5089	202406L 06/19/2026																																																									
0.200	MP5090	202406N 06/20/2026																																																															
0.080 DGS	N/A	AG429602 10/22/2026																																																															
Calibration Adjustment By _____			Department Inspection By <u>SLH</u>																																																														
Barometric Pressure Gauge _____ ID # _____ <table border="1" style="width:100%; border-collapse: collapse;"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.000</td><td></td><td>N/A</td><td>N/A</td></tr><tr><td>0.040</td><td></td><td></td><td></td></tr><tr><td>0.100</td><td></td><td></td><td></td></tr><tr><td>0.200</td><td></td><td></td><td></td></tr><tr><td>0.300</td><td></td><td></td><td></td></tr><tr><td>0.080 DGS</td><td>N/A</td><td></td><td></td></tr></tbody></table> ☐ Post Calibration Adjustment Stability Checks<table border="1" style="width:100%; border-collapse: collapse;"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot #</th><th>Expiration</th></tr></thead><tbody><tr><td>0.050</td><td></td><td></td><td></td></tr><tr><td>0.080</td><td></td><td></td><td></td></tr><tr><td>0.200</td><td></td><td></td><td></td></tr><tr><td>0.080 DGS</td><td>N/A</td><td></td><td></td></tr></tbody></table>			Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			Barometric Pressure ID# <u>28421</u> Gauge <u>1019</u> Instrument <u>1017</u> Mouth Alcohol Solution Lot # <u>2025-A</u> Acetone Stock Solution Lot # <u>2024-B</u> <table border="1" style="width:100%; border-collapse: collapse;"><thead><tr><th>Simulator</th><th>Serial Number</th></tr></thead><tbody><tr><td>0.000</td><td>MP6294</td></tr><tr><td>Interferent</td><td>MP5087</td></tr><tr><td>0.050</td><td>MP5088</td></tr><tr><td>0.080</td><td>MP5089</td></tr><tr><td>0.200</td><td>MP5090</td></tr></tbody></table>			Simulator	Serial Number	0.000	MP6294	Interferent	MP5087	0.050	MP5088	0.080	MP5089	0.200	MP5090
Simulator	Serial #	Lot #	Expiration																																																														
0.000		N/A	N/A																																																														
0.040																																																																	
0.100																																																																	
0.200																																																																	
0.300																																																																	
0.080 DGS	N/A																																																																
Simulator	Serial #	Lot #	Expiration																																																														
0.050																																																																	
0.080																																																																	
0.200																																																																	
0.080 DGS	N/A																																																																
Simulator	Serial Number																																																																
0.000	MP6294																																																																
Interferent	MP5087																																																																
0.050	MP5088																																																																
0.080	MP5089																																																																
0.200	MP5090																																																																
Notes/Suggested Service: _____ _____ _____ _____ _____ _____ _____ _____ _____			Attachments <table border="1" style="width:100%; border-collapse: collapse;"><tbody><tr><td> ☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment </td><td> ☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____ </td></tr></tbody></table>			☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment	☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____																																																										
			☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Calibration Adjustment	☐ Post-Stability Checks ☐ Flow Calibration ☐ Form 40 ☐ Other _____																																																													
☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use																																																																	
Taylor Gutschow Digitally signed by Taylor Gutschow Date: 2025.06.30 13:31:02 +04'00'			Phil Nicodemo Digitally signed by Phil Nicodemo Date: 2025.07.01 09:24:58 -04'00'																																																														
Tech Review / Date _____			Admin Review / Date _____																																																														

Stability Checks

0.05g/210L 0.047 to 0.053 ✓	0.08g/210L 0.077 to 0.083 ✓	0.20g/210L 0.194 to 0.206 ✓	DGS 0.08g/210L 0.077 to 0.083 ✓ ≤0.003 of Wet ✓																																																																																																																																																												
Performed Root Case Analysis	Performed Root Case Analysis	Performed Root Case Analysis	Performed Root Case Analysis																																																																																																																																																												
COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001260 06/17/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:09</td></tr> <tr><td>Control Test</td><td>0.047</td><td>11:09</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:10</td></tr> <tr><td>Control Test</td><td>0.047</td><td>11:11</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:11</td></tr> <tr><td>Control Test</td><td>0.047</td><td>11:12</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:12</td></tr> <tr><td>Control Test</td><td>0.047</td><td>11:12</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0470</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature: <i>manpi</i>	Test	g/210L	Time	Air Blank	0.000	11:09	Control Test	0.047	11:09	Air Blank	0.000	11:10	Control Test	0.047	11:11	Air Blank	0.000	11:11	Control Test	0.047	11:12	Air Blank	0.000	11:12	Control Test	0.047	11:12	Control Test Stats			Average	0.0470		Std Dev	0.0000		Rel Std Dev(%)	0.0000		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001260 06/17/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:16</td></tr> <tr><td>Control Test</td><td>0.081</td><td>11:16</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:17</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:18</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:18</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:19</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:20</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:20</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0800</td><td></td></tr> <tr><td>Std Dev</td><td>0.0010</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.2500</td><td></td></tr> </tbody> </table> Operator's Signature: <i>manpi</i>	Test	g/210L	Time	Air Blank	0.000	11:16	Control Test	0.081	11:16	Air Blank	0.000	11:17	Control Test	0.080	11:18	Air Blank	0.000	11:18	Control Test	0.079	11:19	Air Blank	0.000	11:20	Control Test	0.079	11:20	Control Test Stats			Average	0.0800		Std Dev	0.0010		Rel Std Dev(%)	1.2500		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001260 06/17/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:43</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:44</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:44</td></tr> <tr><td>Control Test</td><td>0.199</td><td>11:45</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:45</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:46</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:47</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:47</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1997</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2892</td><td></td></tr> </tbody> </table> Operator's Signature: <i>manpi</i>	Test	g/210L	Time	Air Blank	0.000	11:43	Control Test	0.200	11:44	Air Blank	0.000	11:44	Control Test	0.199	11:45	Air Blank	0.000	11:45	Control Test	0.200	11:46	Air Blank	0.000	11:47	Control Test	0.200	11:47	Control Test Stats			Average	0.1997		Std Dev	0.0006		Rel Std Dev(%)	0.2892		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001260 06/17/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:23</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:23</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:24</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:24</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:25</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:25</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:26</td></tr> <tr><td>Control Test</td><td>0.079</td><td>11:26</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0790</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature: <i>manpi</i>	Test	g/210L	Time	Air Blank	0.000	11:23	Control Test	0.079	11:23	Air Blank	0.000	11:24	Control Test	0.079	11:24	Air Blank	0.000	11:25	Control Test	0.079	11:25	Air Blank	0.000	11:26	Control Test	0.079	11:26	Control Test Stats			Average	0.0790		Std Dev	0.0000		Rel Std Dev(%)	0.0000	
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:09																																																																																																																																																													
Control Test	0.047	11:09																																																																																																																																																													
Air Blank	0.000	11:10																																																																																																																																																													
Control Test	0.047	11:11																																																																																																																																																													
Air Blank	0.000	11:11																																																																																																																																																													
Control Test	0.047	11:12																																																																																																																																																													
Air Blank	0.000	11:12																																																																																																																																																													
Control Test	0.047	11:12																																																																																																																																																													
Control Test Stats																																																																																																																																																															
Average	0.0470																																																																																																																																																														
Std Dev	0.0000																																																																																																																																																														
Rel Std Dev(%)	0.0000																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:16																																																																																																																																																													
Control Test	0.081	11:16																																																																																																																																																													
Air Blank	0.000	11:17																																																																																																																																																													
Control Test	0.080	11:18																																																																																																																																																													
Air Blank	0.000	11:18																																																																																																																																																													
Control Test	0.079	11:19																																																																																																																																																													
Air Blank	0.000	11:20																																																																																																																																																													
Control Test	0.079	11:20																																																																																																																																																													
Control Test Stats																																																																																																																																																															
Average	0.0800																																																																																																																																																														
Std Dev	0.0010																																																																																																																																																														
Rel Std Dev(%)	1.2500																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:43																																																																																																																																																													
Control Test	0.200	11:44																																																																																																																																																													
Air Blank	0.000	11:44																																																																																																																																																													
Control Test	0.199	11:45																																																																																																																																																													
Air Blank	0.000	11:45																																																																																																																																																													
Control Test	0.200	11:46																																																																																																																																																													
Air Blank	0.000	11:47																																																																																																																																																													
Control Test	0.200	11:47																																																																																																																																																													
Control Test Stats																																																																																																																																																															
Average	0.1997																																																																																																																																																														
Std Dev	0.0006																																																																																																																																																														
Rel Std Dev(%)	0.2892																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:23																																																																																																																																																													
Control Test	0.079	11:23																																																																																																																																																													
Air Blank	0.000	11:24																																																																																																																																																													
Control Test	0.079	11:24																																																																																																																																																													
Air Blank	0.000	11:25																																																																																																																																																													
Control Test	0.079	11:25																																																																																																																																																													
Air Blank	0.000	11:26																																																																																																																																																													
Control Test	0.079	11:26																																																																																																																																																													
Control Test Stats																																																																																																																																																															
Average	0.0790																																																																																																																																																														
Std Dev	0.0000																																																																																																																																																														
Rel Std Dev(%)	0.0000																																																																																																																																																														

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: COCOA PD

Time of Inspection: 13:39

Date of Inspection: 06/23/2025

Serial Number: 80-001260

Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.049	0.080	0.200	0.079
0.000	0.050	0.081	0.200	0.079
0.000	0.050	0.081	0.200	0.079
0.000	0.050	0.080	0.200	0.079
0.000	0.050	0.080	0.200	0.079
0.000	0.051	0.081	0.199	0.078
0.000	0.050	0.080	0.200	0.079
0.000	0.050	0.081	0.200	0.079
0.000	0.050	0.080	0.200	0.079
0.000	0.051	0.080	0.200	0.079


Standard Deviations	0.0005	0.0005	0.0003	0.0003
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.


I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

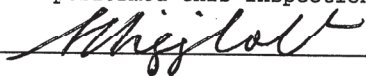

LEANDRA HIGGINBOTHAM

Signature and Printed Name

06/23/2025
Date

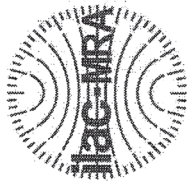

Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001260, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001260</u>	UNCERTAINTY * $\pm$	
Owning Agency:	<u>COCOA PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>06/23/2025</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>13:39</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$)

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.
This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

FDLE/ATP Form 69 October 2024
Issuing Authority: Alcohol Testing Program

06/23/2025

Date


LEANDRA HIGGINBOTHAM,
Department Inspector

Service • Integrity • Respect • Quality