



# INSTRUMENT PROCESSING SHEET

Agency Fort Myers Police DepartmentS/N 80-000914Florida Department of  
Law EnforcementDate In 07/14/2025 DI Completion Date 07/29/2025 Ship  P/U  H/D  CMI  EE

<b>Intake</b> By <u>TDG</u> Date <u>07/22/2025</u> <input checked="" type="checkbox"/> Annual <input type="checkbox"/> Registration <input type="checkbox"/> Return from CMI / EE Visual Inspection: <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight Other Equipment/ Accessories: <input type="checkbox"/> Power cord <input type="checkbox"/> Printer Cable <input type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable Notes: <u>Dropped off. No box. The exhaust port cover has a large hole broken in the side.</u>	<b>Quality Checks</b> By <u>TDG</u> Date <u>07/22/2025</u> <input checked="" type="checkbox"/> Breath Tube Screen <input checked="" type="checkbox"/> Replace External O-Rings <input checked="" type="checkbox"/> Instrument Set Up Verified <input checked="" type="checkbox"/> R-Value <u>136</u> <input checked="" type="checkbox"/> Flow Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>0.144</u> (.139 - .169) 36 mm <u>0.156</u> (.156 - .190) 53 mm <u>0.230</u> (.228 - .278) 103 mm <u>0.496</u> (.447 - .547) <input checked="" type="checkbox"/> Barometric Pressure Check Gauge ID # <u>28199</u> <input checked="" type="checkbox"/> Stability Checks	<b>Flow Calibration</b> By _____ Date _____ Flow Column # _____ <input type="checkbox"/> 5L/min – 17mm <input type="checkbox"/> 15L/min – 53mm <input type="checkbox"/> 30L/min – 103mm <input type="checkbox"/> R-Value _____ <input type="checkbox"/> Post Calibration Verification (L/s) Flow Column # _____ 32 mm _____ (.139 - .169) 36 mm _____ (.156 - .190) 53 mm _____ (.228 - .278) 103 mm _____ (.447 - .547)																												
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td rowspan="2">MP6286</td> <td>202406K 06/19/2026</td> </tr> <tr> <td>0.080</td> <td>202406L 06/19/2026</td> </tr> <tr> <td>0.200</td> <td rowspan="2">MP6288</td> <td>202406N 06/20/2026</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A AG429602 10/22/2026</td> </tr> </tbody> </table>			Simulator	Serial #	Lot #/Exp	0.050	MP6286	202406K 06/19/2026	0.080	202406L 06/19/2026	0.200	MP6288	202406N 06/20/2026	0.080 DGS	N/A AG429602 10/22/2026															
Simulator	Serial #	Lot #/Exp																												
0.050	MP6286	202406K 06/19/2026																												
0.080		202406L 06/19/2026																												
0.200	MP6288	202406N 06/20/2026																												
0.080 DGS		N/A AG429602 10/22/2026																												
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>			Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A		
Simulator	Serial #	Lot #	Expiration																											
0.000		N/A	N/A																											
0.040																														
0.100																														
0.200																														
0.300																														
0.080 DGS	N/A																													
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A										
Simulator	Serial #	Lot #	Expiration																											
0.050																														
0.080																														
0.200																														
0.080 DGS	N/A																													
<b>Calibration Adjustment</b> By _____ Barometric Pressure Gauge _____ ID # _____ <input type="checkbox"/> Post Calibration Adjustment Stability Checks																														
<b>Department Inspection</b> By <u>TDG</u> Barometric Pressure ID# <u>28199</u> Gauge <u>1019</u> Instrument <u>1018</u> Mouth Alcohol Solution Lot # <u>2024-A</u> Acetone Stock Solution Lot # <u>2023-B</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6284</td> </tr> <tr> <td>Interferent</td> <td>MP6285</td> </tr> <tr> <td>0.050</td> <td>MP6286</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> </tr> </tbody> </table>			Simulator	Serial Number	0.000	MP6284	Interferent	MP6285	0.050	MP6286	0.080	MP6287	0.200	MP6288																
Simulator	Serial Number																													
0.000	MP6284																													
Interferent	MP6285																													
0.050	MP6286																													
0.080	MP6287																													
0.200	MP6288																													
<b>Attachments</b> <input checked="" type="checkbox"/> Form 41 <input checked="" type="checkbox"/> Stability Checks <input checked="" type="checkbox"/> Calibration Certificate <input type="checkbox"/> Calibration Adjustment <input type="checkbox"/> Post-Stability Checks <input type="checkbox"/> Flow Calibration <input type="checkbox"/> Form 40 <input type="checkbox"/> Other _____																														
Notes/Suggested Service: _____ _____ _____ _____ _____ _____ _____ _____																														
<input checked="" type="checkbox"/> Instrument Complies with Chapter 11D-8, FAC <input type="checkbox"/> Instrument Does Not Comply with Chapter 11D-8, FAC <input checked="" type="checkbox"/> Return to/Place into Evidentiary Use <input type="checkbox"/> Remain Out of Evidentiary Use <input checked="" type="checkbox"/> Conduct an Agency Inspection Before Evidentiary Use																														
<table style="width:100%;"> <tr> <td style="width:50%;">           Phil Nicodemo  <small>Digitally signed by Phil Nicodemo Date: 2025.08.12 21:05:52 -0400</small> </td> <td style="width:50%; text-align: right;">           Shayla Platt  <small>Digitally signed by Shayla Platt Date: 2025.08.13 09:49:33 -0400</small> </td> </tr> <tr> <td>Tech Review / Date</td> <td>Admin Review / Date</td> </tr> </table>			Phil Nicodemo <small>Digitally signed by Phil Nicodemo Date: 2025.08.12 21:05:52 -0400</small>	Shayla Platt <small>Digitally signed by Shayla Platt Date: 2025.08.13 09:49:33 -0400</small>	Tech Review / Date	Admin Review / Date																								
Phil Nicodemo <small>Digitally signed by Phil Nicodemo Date: 2025.08.12 21:05:52 -0400</small>	Shayla Platt <small>Digitally signed by Shayla Platt Date: 2025.08.13 09:49:33 -0400</small>																													
Tech Review / Date	Admin Review / Date																													

# Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083	≤0.003 of Wet																																																																																																																																											
<p>FORT MYERS PD Intoxilyzer - Alcohol Analyzer Model: 8000 SN 80-000914 07/22/2025 Software: 8100.27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:41</td></tr> <tr><td>Control Test</td><td>0.050</td><td>14:42</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:42</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:43</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:44</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:44</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:45</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:45</td></tr> <tr><td>Average</td><td>0.0493</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel. Std Dev(%)</td><td>1.1703</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>ML</i></p>	Test	g/210L	Time	Air Blank	0.000	14:41	Control Test	0.050	14:42	Air Blank	0.000	14:42	Control Test	0.049	14:43	Air Blank	0.000	14:44	Control Test	0.049	14:44	Air Blank	0.000	14:45	Control Test	0.049	14:45	Average	0.0493		Std Dev	0.0006		Rel. Std Dev(%)	1.1703		<p>FORT MYERS PD Intoxilyzer - Alcohol Analyzer Model: 8000 SN 80-000914 07/22/2025 Software: 8100.27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:58</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:59</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:59</td></tr> <tr><td>Control Test</td><td>0.080</td><td>15:00</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:01</td></tr> <tr><td>Control Test</td><td>0.080</td><td>15:01</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:02</td></tr> <tr><td>Control Test</td><td>0.080</td><td>15:02</td></tr> <tr><td>Average</td><td>0.0800</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel. Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>ML</i></p>	Test	g/210L	Time	Air Blank	0.000	14:58	Control Test	0.080	14:59	Air Blank	0.000	14:59	Control Test	0.080	15:00	Air Blank	0.000	15:01	Control Test	0.080	15:01	Air Blank	0.000	15:02	Control Test	0.080	15:02	Average	0.0800		Std Dev	0.0000		Rel. Std Dev(%)	0.0000		<p>FORT MYERS PD Intoxilyzer - Alcohol Analyzer Model: 8000 SN 80-000914 07/22/2025 Software: 8100.27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>15:09</td></tr> <tr><td>Control Test</td><td>0.199</td><td>15:10</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:10</td></tr> <tr><td>Control Test</td><td>0.199</td><td>15:11</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:12</td></tr> <tr><td>Control Test</td><td>0.197</td><td>15:12</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:13</td></tr> <tr><td>Control Test</td><td>0.1983</td><td></td></tr> <tr><td>Average</td><td>0.1983</td><td></td></tr> <tr><td>Std Dev</td><td>0.0012</td><td></td></tr> <tr><td>Rel. Std Dev(%)</td><td>0.5822</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>ML</i></p>	Test	g/210L	Time	Air Blank	0.000	15:09	Control Test	0.199	15:10	Air Blank	0.000	15:10	Control Test	0.199	15:11	Air Blank	0.000	15:12	Control Test	0.197	15:12	Air Blank	0.000	15:13	Control Test	0.1983		Average	0.1983		Std Dev	0.0012		Rel. Std Dev(%)	0.5822		<p>FORT MYERS PD Intoxilyzer - Alcohol Analyzer Model: 8000 SN 80-000914 07/22/2025 Software: 8100.27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>15:14</td></tr> <tr><td>Control Test</td><td>0.083</td><td>15:15</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:15</td></tr> <tr><td>Control Test</td><td>0.083</td><td>15:15</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:16</td></tr> <tr><td>Control Test</td><td>0.083</td><td>15:16</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>15:17</td></tr> <tr><td>Control Test</td><td>0.0830</td><td></td></tr> <tr><td>Average</td><td>0.0830</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel. Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>ML</i></p>	Test	g/210L	Time	Air Blank	0.000	15:14	Control Test	0.083	15:15	Air Blank	0.000	15:15	Control Test	0.083	15:15	Air Blank	0.000	15:16	Control Test	0.083	15:16	Air Blank	0.000	15:17	Control Test	0.0830		Average	0.0830		Std Dev	0.0000		Rel. Std Dev(%)	0.0000	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:41																																																																																																																																																	
Control Test	0.050	14:42																																																																																																																																																	
Air Blank	0.000	14:42																																																																																																																																																	
Control Test	0.049	14:43																																																																																																																																																	
Air Blank	0.000	14:44																																																																																																																																																	
Control Test	0.049	14:44																																																																																																																																																	
Air Blank	0.000	14:45																																																																																																																																																	
Control Test	0.049	14:45																																																																																																																																																	
Average	0.0493																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel. Std Dev(%)	1.1703																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:58																																																																																																																																																	
Control Test	0.080	14:59																																																																																																																																																	
Air Blank	0.000	14:59																																																																																																																																																	
Control Test	0.080	15:00																																																																																																																																																	
Air Blank	0.000	15:01																																																																																																																																																	
Control Test	0.080	15:01																																																																																																																																																	
Air Blank	0.000	15:02																																																																																																																																																	
Control Test	0.080	15:02																																																																																																																																																	
Average	0.0800																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel. Std Dev(%)	0.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	15:09																																																																																																																																																	
Control Test	0.199	15:10																																																																																																																																																	
Air Blank	0.000	15:10																																																																																																																																																	
Control Test	0.199	15:11																																																																																																																																																	
Air Blank	0.000	15:12																																																																																																																																																	
Control Test	0.197	15:12																																																																																																																																																	
Air Blank	0.000	15:13																																																																																																																																																	
Control Test	0.1983																																																																																																																																																		
Average	0.1983																																																																																																																																																		
Std Dev	0.0012																																																																																																																																																		
Rel. Std Dev(%)	0.5822																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	15:14																																																																																																																																																	
Control Test	0.083	15:15																																																																																																																																																	
Air Blank	0.000	15:15																																																																																																																																																	
Control Test	0.083	15:15																																																																																																																																																	
Air Blank	0.000	15:16																																																																																																																																																	
Control Test	0.083	15:16																																																																																																																																																	
Air Blank	0.000	15:17																																																																																																																																																	
Control Test	0.0830																																																																																																																																																		
Average	0.0830																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel. Std Dev(%)	0.0000																																																																																																																																																		

# Florida Department of Law Enforcement Alcohol Testing Program

## DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: FORT MYERS PD  
Time of Inspection: 13:59

Date of Inspection: 07/29/2025

Serial Number: 80-000914  
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.050	0.080	0.200	0.082
0.000	0.050	0.080	0.199	0.083
0.000	0.050	0.080	0.199	0.082
0.000	0.051	0.080	0.199	0.083
0.000	0.050	0.080	0.199	0.082
0.000	0.050	0.080	0.199	0.082
0.000	0.050	0.081	0.200	0.082
0.000	0.051	0.080	0.200	0.082
0.000	0.051	0.080	0.200	0.082
0.000	0.050	0.080	0.200	0.082

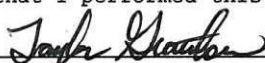
Standard Deviations	0.0004	0.0003	0.0005	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (  ) does not comply (  ) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.


TAYLOR D GUTSCHOW  
 \_\_\_\_\_  
 Signature and Printed Name

07/29/2025  
 Date





# Calibration Certificate

Florida Department of Law Enforcement  
Alcohol Testing Program  
4700 Terminal Drive, Suite 1  
Ft. Myers, FL 33907

This is to certify the calibration of Intoxilyzer 8000 serial number 80-000914, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-000914</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>FORT MYERS PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>07/29/2025</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>13:59</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within  $\pm 0.005$  or 5%, whichever is greater, of the target alcohol concentration.

\*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

## TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards. This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Taylor  
Gutschow  
Digitally signed by Taylor  
Gutschow  
Date: 2025.07.29 15:35:58  
-04'00'

07/29/2025

Date

TAYLOR D GUTSCHOW,

Department Inspector

FDLE/ATP Form 69 December 2021

Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality