

Alcohol Testing Program - Instrument Processing Sheet

Agency: LARGO PD

Instrument Serial Number: 80-001079

Date In: 6/2/2026

DI Completion Date: 6/11/26

☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake By: SLH Date: 6/2/2026		Quality Checks By: IAS Date: 6/5/2026		Flow Adjustment By:																
☒ Annual ☐ Dropped Off ☐ Registration ☐ Return from CMI / EE ☐ Training Instrument Visual Inspection ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/Accessories ☒ Power Cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes:		☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value: 168 ☒ Flow Verification (L/s) Flow Column #: ATP105 32 mm .160 (.139-.169) 36 mm .175 (.156-.190) 53 mm .250 (.228-.278) 103 mm .515 (.447-.547) ☒ Barometric Pressure Check Gauge ID #: 28662 Gauge: 1016 Instrument: 1018 ☒ Stability Checks <table border="1"><thead><tr><th>Simulator</th><th>Serial #</th><th>Lot#/Exp</th></tr></thead><tbody><tr><td>0.050</td><td>MP6291</td><td>202603A 3/3/2028</td></tr><tr><td>0.080</td><td>MP6292</td><td>202603B 3/3/2028</td></tr><tr><td>0.200</td><td>MP6293</td><td>202406N 6/20/2026</td></tr><tr><td>0.080 DGS</td><td>N/A</td><td>23825080A1 10/5/2027</td></tr></tbody></table>		Simulator	Serial #	Lot#/Exp	0.050	MP6291	202603A 3/3/2028	0.080	MP6292	202603B 3/3/2028	0.200	MP6293	202406N 6/20/2026	0.080 DGS	N/A	23825080A1 10/5/2027	Flow Column #: _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value: _____ ☐ Post Adjustment Verification (L/S) Flow Column #: _____ 32 mm (.139-.169) 36 mm (.156-.190) 53 mm (.228-.278) 103 mm (.447-.547)	
Simulator	Serial #	Lot#/Exp																		
0.050	MP6291	202603A 3/3/2028																		
0.080	MP6292	202603B 3/3/2028																		
0.200	MP6293	202406N 6/20/2026																		
0.080 DGS	N/A	23825080A1 10/5/2027																		
Optical Bench Adjustment By: _____		Department Inspection By: KTS																		
Barometric Pressure Gauge: _____ ID#: _____		Barometric Pressure ID#: 28427																		
Simulator	Serial #	Lot #	Expiration	Gauge: 1011	Instrument: 1014															
0.000		N/A	N/A	Mouth Alcohol Solution Lot #: 2025-D	Exp: 9/25/27															
0.040				Acetone Stock Solution Lot #: 2025-B	Exp: 9/22/27															
0.100				Simulator	Serial Number															
0.200				0.000	MP6289															
0.300				Interferent	MP6290															
0.080 DGS	N/A			0.050	MP6291															
☐ Post Optical Bench Adjustment Stability Checks				0.080	MP6292															
Simulator	Serial #	Lot #	Expiration	0.200	MP6293															
0.050				Attachments																
0.080				☒ Form 41																
0.200				☒ Stability Checks																
0.080 DGS	N/A			☒ Calibration Certificate																
Gauge ID #: _____				☐ Post-Stability Checks																
Gauge: _____ Instrument: _____				☐ Flow Adjustment																
Notes/Suggested Service: Tech Review: re-attached Form 41 due to formatting issues, KTS 6/17/26				☐ Form 40																
				☐ Other:																
				☒ Instrument Complies with Chapter 11D-8, FAC																
				☐ Instrument Does Not Comply with Chapter 11D-8, FAC																
				☒ Return to/Place into Evidentiary Use																
				☐ Remain Out of Evidentiary Use																
				☒ Conduct an Agency Inspection Before Evidentiary Use																
Wen-Chi Pierson Digitally signed by Wen-Chi Pierson Date: 2026.06.17 11:29:30 -04'00'				Taylor Gutschow Digitally signed by Taylor Gutschow Date: 2026.06.19 15:11:38 -04'00'																
Tech Review				Admin Review																

Stability Checks

0.050 g/210L	0.080 g/210L	0.200 g/210L	DGS 0.080 g/210L																																																																																																																																				
0.047 to 0.053 g/210L	0.077 to 0.083 g/210L	0.194 to 0.206 g/210L	0.077 to 0.083 g/210L ≤0.003 g/210L of Wet																																																																																																																																				
✓ LARGO PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001079 06/05/2026 Software: 8100.27 Test g/210L Time <table border="1"> <tr><td>Air Blank</td><td>0.000</td><td>12:45</td></tr> <tr><td>Control Test</td><td>0.049</td><td>12:45</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:46</td></tr> <tr><td>Control Test</td><td>0.048</td><td>12:47</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:47</td></tr> <tr><td>Control Test</td><td>0.049</td><td>12:48</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:48</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0487</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.1863</td><td></td></tr> </table> Operator's Signature: <i>Isabelle Layhowski</i>	Air Blank	0.000	12:45	Control Test	0.049	12:45	Air Blank	0.000	12:46	Control Test	0.048	12:47	Air Blank	0.000	12:47	Control Test	0.049	12:48	Air Blank	0.000	12:48	Control Test Stats			Average	0.0487		Std Dev	0.0006		Rel Std Dev(%)	1.1863		LARGO PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001079 06/05/2026 Software: 8100.27 Test g/210L Time <table border="1"> <tr><td>Air Blank</td><td>0.000</td><td>12:39</td></tr> <tr><td>Control Test</td><td>0.078</td><td>12:39</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:40</td></tr> <tr><td>Control Test</td><td>0.077</td><td>12:41</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:41</td></tr> <tr><td>Control Test</td><td>0.078</td><td>12:42</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:42</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0777</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7434</td><td></td></tr> </table> Operator's Signature: <i>Isabelle Layhowski</i>	Air Blank	0.000	12:39	Control Test	0.078	12:39	Air Blank	0.000	12:40	Control Test	0.077	12:41	Air Blank	0.000	12:41	Control Test	0.078	12:42	Air Blank	0.000	12:42	Control Test Stats			Average	0.0777		Std Dev	0.0006		Rel Std Dev(%)	0.7434		LARGO PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001079 06/05/2026 Software: 8100.27 Test g/210L Time <table border="1"> <tr><td>Air Blank</td><td>0.000</td><td>12:52</td></tr> <tr><td>Control Test</td><td>0.197</td><td>12:53</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:53</td></tr> <tr><td>Control Test</td><td>0.196</td><td>12:54</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:55</td></tr> <tr><td>Control Test</td><td>0.197</td><td>12:55</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:56</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1967</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2936</td><td></td></tr> </table> Operator's Signature: <i>Isabelle Layhowski</i>	Air Blank	0.000	12:52	Control Test	0.197	12:53	Air Blank	0.000	12:53	Control Test	0.196	12:54	Air Blank	0.000	12:55	Control Test	0.197	12:55	Air Blank	0.000	12:56	Control Test Stats			Average	0.1967		Std Dev	0.0006		Rel Std Dev(%)	0.2936		LARGO PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001079 06/05/2026 Software: 8100.27 Test g/210L Time <table border="1"> <tr><td>Air Blank</td><td>0.000</td><td>13:00</td></tr> <tr><td>Control Test</td><td>0.081</td><td>13:01</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:01</td></tr> <tr><td>Control Test</td><td>0.080</td><td>13:01</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:02</td></tr> <tr><td>Control Test</td><td>0.080</td><td>13:02</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:03</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0803</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7187</td><td></td></tr> </table> Operator's Signature: <i>Isabelle Layhowski</i>	Air Blank	0.000	13:00	Control Test	0.081	13:01	Air Blank	0.000	13:01	Control Test	0.080	13:01	Air Blank	0.000	13:02	Control Test	0.080	13:02	Air Blank	0.000	13:03	Control Test Stats			Average	0.0803		Std Dev	0.0006		Rel Std Dev(%)	0.7187	
Air Blank	0.000	12:45																																																																																																																																					
Control Test	0.049	12:45																																																																																																																																					
Air Blank	0.000	12:46																																																																																																																																					
Control Test	0.048	12:47																																																																																																																																					
Air Blank	0.000	12:47																																																																																																																																					
Control Test	0.049	12:48																																																																																																																																					
Air Blank	0.000	12:48																																																																																																																																					
Control Test Stats																																																																																																																																							
Average	0.0487																																																																																																																																						
Std Dev	0.0006																																																																																																																																						
Rel Std Dev(%)	1.1863																																																																																																																																						
Air Blank	0.000	12:39																																																																																																																																					
Control Test	0.078	12:39																																																																																																																																					
Air Blank	0.000	12:40																																																																																																																																					
Control Test	0.077	12:41																																																																																																																																					
Air Blank	0.000	12:41																																																																																																																																					
Control Test	0.078	12:42																																																																																																																																					
Air Blank	0.000	12:42																																																																																																																																					
Control Test Stats																																																																																																																																							
Average	0.0777																																																																																																																																						
Std Dev	0.0006																																																																																																																																						
Rel Std Dev(%)	0.7434																																																																																																																																						
Air Blank	0.000	12:52																																																																																																																																					
Control Test	0.197	12:53																																																																																																																																					
Air Blank	0.000	12:53																																																																																																																																					
Control Test	0.196	12:54																																																																																																																																					
Air Blank	0.000	12:55																																																																																																																																					
Control Test	0.197	12:55																																																																																																																																					
Air Blank	0.000	12:56																																																																																																																																					
Control Test Stats																																																																																																																																							
Average	0.1967																																																																																																																																						
Std Dev	0.0006																																																																																																																																						
Rel Std Dev(%)	0.2936																																																																																																																																						
Air Blank	0.000	13:00																																																																																																																																					
Control Test	0.081	13:01																																																																																																																																					
Air Blank	0.000	13:01																																																																																																																																					
Control Test	0.080	13:01																																																																																																																																					
Air Blank	0.000	13:02																																																																																																																																					
Control Test	0.080	13:02																																																																																																																																					
Air Blank	0.000	13:03																																																																																																																																					
Control Test Stats																																																																																																																																							
Average	0.0803																																																																																																																																						
Std Dev	0.0006																																																																																																																																						
Rel Std Dev(%)	0.7187																																																																																																																																						

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: LARGO PD

Serial Number: 80-001079

Time of Inspection: 13:13

Date of Inspection: 06/11/2026

Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#: 202603A Exp: 03/03/2028	0.08g/210L Test (g/210L) Lot#: 202603B Exp: 03/03/2028	0.20g/210L Test (g/210L) Lot#: 202603D Exp: 03/04/2028	0.08 g/210L Dry Gas Std Test (g/210L) Lot#: 23825080A1 Exp: 10/05/2027
0.000	0.000 / 0.050	0.078	0.197	0.080
0.000	0.000 / 0.049	0.078	0.197	0.079
0.000	0.000 / 0.051	0.078	0.197	0.080
0.000	0.000 / 0.050	0.077	0.197	0.080
0.000	0.000 / 0.050	0.077	0.197	0.080
0.000	0.000 / 0.051	0.078	0.197	0.080
0.000	0.000 / 0.051	0.078	0.196	0.080
0.000	0.000 / 0.050	0.078	0.195	0.080
0.000	0.000 / 0.050	0.078	0.194	0.080
0.000	0.000 / 0.051	0.078	0.195	0.080

Standard Deviations	0.0000 / 0.0006	0.0004	0.0011	0.0003
---------------------	-----------------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0006 Number of Simulators Used: 5

Remarks:

05: Control Outside Tolerance. *0.05g/210L simulator was not attached when analysis started. Attached & re-ran for 6/11/26*

The above instrument complies (☒) does not comply () with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

KATIE T SPEARIN

Signature and Printed Name

06/11/2026

Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001079, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001079</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>LARGO PD</u>	0.050 g/210 L	0.004
Calibration Date:	<u>06/11/2026</u>	0.080 g/210 L	0.004
Calibration Time:	<u>13:13</u>	0.200 g/210 L	0.008
		0.080 g/210 L Dry Gas Control	0.005

All results are reported in g/210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/IEC 17025 standards.
This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

FDLE/ATP Form 69 January 2026
Issuing Authority: Alcohol Testing Program

Date

06/11/2026

Katie T Spearin
KATIE T SPEARIN,
Department Inspector

Service • Integrity • Respect • Quality