

Alcohol Testing Program - Instrument Processing Sheet

Agency: HOLMES BEACH PD Instrument Serial Number: 80-006692  
 Date In: 1/29/2026 DI Completion Date: 2/6/2026  Ship  P/U  H/D  CMI  EE

<b>Intake By:</b> <u>KTS</u> <b>Date:</b> <u>1/29/2026</u>	<b>Quality Checks By:</b> <u>KTS</u> <b>Date:</b> <u>2/4/2026</u>	<b>Flow Adjustment By:</b> _____																			
<input checked="" type="checkbox"/> Annual <input type="checkbox"/> Dropped Off <input type="checkbox"/> Registration <input checked="" type="checkbox"/> Return from CMI / EE <input type="checkbox"/> Training Instrument Visual Inspection <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight Other Equipment/Accessories <input type="checkbox"/> Power Cord <input type="checkbox"/> Printer Cable <input type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable Notes: Instrument came in pelican case. 1/29/26	<input checked="" type="checkbox"/> Breath Tube Screen <input checked="" type="checkbox"/> Replace External O-Rings <input checked="" type="checkbox"/> Instrument Set Up Verified <input checked="" type="checkbox"/> R-Value: <u>176</u> <input checked="" type="checkbox"/> Flow Verification (L/s) Flow Column #: <u>ATP103</u> 32 mm <u>0.144</u> (.139-.169) 36 mm <u>0.160</u> (.156-.190) 53 mm <u>0.230</u> (.228-.278) 103 mm <u>0.496</u> (.447-.547) <input checked="" type="checkbox"/> Barometric Pressure Check Gauge ID #: <u>28421</u> Gauge: <u>1019</u> Instrument: <u>1020</u> <input checked="" type="checkbox"/> Stability Checks	Flow Column #: _____ <input type="checkbox"/> 5L/min – 17mm <input type="checkbox"/> 15L/min – 53mm <input checked="" type="checkbox"/> 30L/min – 103mm <input type="checkbox"/> R-Value: _____ <input type="checkbox"/> Post Adjustment Verification (L/S) Flow Column #: _____ 32 mm _____ (.139-.169) 36 mm _____ (.156-.190) 53 mm _____ (.228-.278) 103 mm _____ (.447-.547)																			
	<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot#/Exp</th> </tr> </thead> <tbody> <tr> <td rowspan="2">0.050</td> <td rowspan="2">MP5088</td> <td>202406K</td> </tr> <tr> <td>6/19/2026</td> </tr> <tr> <td rowspan="2">0.080</td> <td rowspan="2">MP5089</td> <td>202406L</td> </tr> <tr> <td>6/19/2026</td> </tr> <tr> <td rowspan="2">0.200</td> <td rowspan="2">MP5090</td> <td>202406N</td> </tr> <tr> <td>6/20/2026</td> </tr> <tr> <td rowspan="2">0.080 DGS</td> <td rowspan="2">N/A</td> <td>AG510701</td> </tr> <tr> <td>4/17/2027</td> </tr> </tbody> </table>	Simulator	Serial #	Lot#/Exp	0.050	MP5088	202406K	6/19/2026	0.080	MP5089	202406L	6/19/2026	0.200	MP5090	202406N	6/20/2026	0.080 DGS	N/A	AG510701	4/17/2027	<b>Maintenance By:</b> _____ <b>Date:</b> _____ <input type="checkbox"/> Battery Replacement <input type="checkbox"/> Dry Gas Regulator Replacement <input type="checkbox"/> Tank Sensor Tare <input type="checkbox"/> Breath Tube Replacement <input type="checkbox"/> Other: _____
Simulator	Serial #	Lot#/Exp																			
0.050	MP5088	202406K																			
		6/19/2026																			
0.080	MP5089	202406L																			
		6/19/2026																			
0.200	MP5090	202406N																			
		6/20/2026																			
0.080 DGS	N/A	AG510701																			
		4/17/2027																			

<b>Optical Bench Adjustment</b> By: _____	<b>Department Inspection</b> By: <u>KTS</u>																																								
Barometric Pressure Gauge: _____ ID#: _____	Barometric Pressure ID#: <u>28421</u>																																								
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Gauge: <u>1016</u> Instrument: <u>1013</u> Mouth Alcohol Solution Lot #: <u>2025-D</u> Exp: <u>9/25/2027</u> Acetone Stock Solution Lot #: <u>2025-B</u> Exp: <u>9/22/2027</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP5086</td> </tr> <tr> <td>Interferent</td> <td>MP5087</td> </tr> <tr> <td>0.050</td> <td>MP5088</td> </tr> <tr> <td>0.080</td> <td>MP5089</td> </tr> <tr> <td>0.200</td> <td>MP5090</td> </tr> </tbody> </table>	Simulator	Serial Number	0.000	MP5086	Interferent	MP5087	0.050	MP5088	0.080	MP5089	0.200	MP5090
Simulator	Serial #	Lot #	Expiration																																						
0.000		N/A	N/A																																						
0.040																																									
0.100																																									
0.200																																									
0.300																																									
0.080 DGS	N/A																																								
Simulator	Serial Number																																								
0.000	MP5086																																								
Interferent	MP5087																																								
0.050	MP5088																																								
0.080	MP5089																																								
0.200	MP5090																																								
<input type="checkbox"/> Post Optical Bench Adjustment Stability Checks																																									
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			<b>Attachments</b> <input checked="" type="checkbox"/> Form 41 <input checked="" type="checkbox"/> Stability Checks <input checked="" type="checkbox"/> Calibration Certificate <input type="checkbox"/> Optical Bench Adjustment <input type="checkbox"/> Post-Stability Checks <input type="checkbox"/> Flow Adjustment <input checked="" type="checkbox"/> Form 40 <input type="checkbox"/> Other: _____																				
Simulator	Serial #	Lot #	Expiration																																						
0.050																																									
0.080																																									
0.200																																									
0.080 DGS	N/A																																								
Gauge ID #: _____ Gauge: _____ Instrument: _____																																									

Notes/Suggested Service:  Admin Review: crossed out check mark in flow adjustment section. KTS 2/10/26	<input checked="" type="checkbox"/> <b>Instrument Complies with Chapter 11D-8, FAC</b> <input type="checkbox"/> <b>Instrument Does Not Comply with Chapter 11D-8, FAC</b>  <input checked="" type="checkbox"/> <b>Return to/Place into Evidentiary Use</b> <input type="checkbox"/> <b>Remain Out of Evidentiary Use</b>  <input checked="" type="checkbox"/> <b>Conduct an Agency Inspection Before Evidentiary Use</b>
	Digitally signed by Taylor Gutschow Date: 2026.02.09 12:07:45 -05'00'
	Digitally signed by Wen-Chi Pierson Date: 2026.02.13 13:44:16 -05'00'
<b>Tech Review</b>	<b>Admin Review</b>

# Florida Department of Law Enforcement Alcohol Testing Program

## AGENCY INSPECTION REPORT - INTOXILYZER 8000

Agency: HOLMES BEACH PD  
Time of Inspection: 08:33

Date of Inspection: 02/04/2026

Serial Number: 80-006692  
Software: 8100.27

Check or Test	YES	NO
Date and/or Time Adjusted		No
Diagnostic Check (Pre-Inspection): OK		No
Alcohol Free Subject Test: 0.000		No
Mouth Alcohol Test: Slope Not Met		No
Interferent Detect Test: Interferent Detect		No
Diagnostic Check (Post-Inspection): OK		No


Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#: _____ Exp: _____	0.08g/210L Test (g/210L) Lot#: _____ Exp: _____	0.20g/210L Test (g/210L) Lot#: _____ Exp: _____	0.08 g/210L Dry Gas Std Test (g/210L) Lot#: _____ Exp: _____

Number of Simulators Used: \_\_\_\_\_

Remarks:  
BYPASSED TO OPERATE INSTRUMENT. COMPLIANCE NOT DETERMINED.

The above instrument complies (  ) does not comply (  ) with Chapter 11D-8, FAC.

I certify that I hold a valid Florida Department of Law Enforcement Agency Inspector Permit and that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

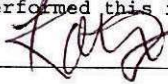 \_\_\_\_\_  
Signature and Printed Name

02/04/2026  
Date

# Stability Checks

<p>0.050 g/210L 0.0477 to 0.053 g/210L</p>	<p>0.080 g/210L 0.077 to 0.083 g/210L</p>	<p>0.200 g/210L 0.194 to 0.206 g/210L</p>	<p>DGS 0.080 g/210L 0.077 to 0.083 g/210L 50.003 g/210L of Wet</p>																																																																																																																																																												
<p>HOLMES BECHT PD IntoxiLizer - Alcohol Analyzer Model: 8100 SN: 80-116692 12/04/2026 Software: 810L27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:34</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:34</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:35</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:35</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:36</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:37</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:37</td></tr> <tr><td>Control Test</td><td>0.050</td><td>11:37</td></tr> <tr><td>Average</td><td>0.050</td><td></td></tr> <tr><td>Std Dev</td><td>0.000</td><td></td></tr> <tr><td>Rel. Std. Dev(%)</td><td>0.000</td><td></td></tr> </tbody> </table> <p><i>Koors</i> Operator's Signature</p>	Test	g/210L	Time	Air Blank	0.000	11:34	Control Test	0.050	11:34	Air Blank	0.000	11:35	Control Test	0.050	11:35	Air Blank	0.000	11:36	Control Test	0.050	11:37	Air Blank	0.000	11:37	Control Test	0.050	11:37	Average	0.050		Std Dev	0.000		Rel. Std. Dev(%)	0.000		<p>HOLMES BECHT PD IntoxiLizer - Alcohol Analyzer Model: 8100 SN: 80-116692 12/04/2026 Software: 810L27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:43</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:43</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:44</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:44</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:45</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:45</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:46</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:46</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:47</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:47</td></tr> <tr><td>Average</td><td>0.080</td><td></td></tr> <tr><td>Std Dev</td><td>0.005</td><td></td></tr> <tr><td>Rel. Std. Dev(%)</td><td>0.757</td><td></td></tr> </tbody> </table> <p><i>Koors</i> Operator's Signature</p>	Test	g/210L	Time	Air Blank	0.000	11:43	Control Test	0.080	11:43	Air Blank	0.000	11:44	Control Test	0.080	11:44	Air Blank	0.000	11:45	Control Test	0.080	11:45	Air Blank	0.000	11:46	Control Test	0.080	11:46	Air Blank	0.000	11:47	Control Test	0.080	11:47	Average	0.080		Std Dev	0.005		Rel. Std. Dev(%)	0.757		<p>HOLMES BECHT PD IntoxiLizer - Alcohol Analyzer Model: 8100 SN: 80-116692 12/04/2026 Software: 810L27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:48</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:48</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:49</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:49</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:50</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:50</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:51</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:51</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:52</td></tr> <tr><td>Control Test</td><td>0.200</td><td>11:52</td></tr> <tr><td>Average</td><td>0.200</td><td></td></tr> <tr><td>Std Dev</td><td>0.005</td><td></td></tr> <tr><td>Rel. Std. Dev(%)</td><td>2.287</td><td></td></tr> </tbody> </table> <p><i>Koors</i> Operator's Signature</p>	Test	g/210L	Time	Air Blank	0.000	11:48	Control Test	0.200	11:48	Air Blank	0.000	11:49	Control Test	0.200	11:49	Air Blank	0.000	11:50	Control Test	0.200	11:50	Air Blank	0.000	11:51	Control Test	0.200	11:51	Air Blank	0.000	11:52	Control Test	0.200	11:52	Average	0.200		Std Dev	0.005		Rel. Std. Dev(%)	2.287		<p>HOLMES BECHT PD IntoxiLizer - Alcohol Analyzer Model: 8100 SN: 80-116692 12/04/2026 Software: 810L27</p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>11:27</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:27</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:28</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:28</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:29</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:29</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>11:30</td></tr> <tr><td>Control Test</td><td>0.080</td><td>11:30</td></tr> <tr><td>Average</td><td>0.080</td><td></td></tr> <tr><td>Std Dev</td><td>0.006</td><td></td></tr> <tr><td>Rel. Std. Dev(%)</td><td>7.187</td><td></td></tr> </tbody> </table> <p><i>Koors</i> Operator's Signature</p>	Test	g/210L	Time	Air Blank	0.000	11:27	Control Test	0.080	11:27	Air Blank	0.000	11:28	Control Test	0.080	11:28	Air Blank	0.000	11:29	Control Test	0.080	11:29	Air Blank	0.000	11:30	Control Test	0.080	11:30	Average	0.080		Std Dev	0.006		Rel. Std. Dev(%)	7.187	
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:34																																																																																																																																																													
Control Test	0.050	11:34																																																																																																																																																													
Air Blank	0.000	11:35																																																																																																																																																													
Control Test	0.050	11:35																																																																																																																																																													
Air Blank	0.000	11:36																																																																																																																																																													
Control Test	0.050	11:37																																																																																																																																																													
Air Blank	0.000	11:37																																																																																																																																																													
Control Test	0.050	11:37																																																																																																																																																													
Average	0.050																																																																																																																																																														
Std Dev	0.000																																																																																																																																																														
Rel. Std. Dev(%)	0.000																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:43																																																																																																																																																													
Control Test	0.080	11:43																																																																																																																																																													
Air Blank	0.000	11:44																																																																																																																																																													
Control Test	0.080	11:44																																																																																																																																																													
Air Blank	0.000	11:45																																																																																																																																																													
Control Test	0.080	11:45																																																																																																																																																													
Air Blank	0.000	11:46																																																																																																																																																													
Control Test	0.080	11:46																																																																																																																																																													
Air Blank	0.000	11:47																																																																																																																																																													
Control Test	0.080	11:47																																																																																																																																																													
Average	0.080																																																																																																																																																														
Std Dev	0.005																																																																																																																																																														
Rel. Std. Dev(%)	0.757																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:48																																																																																																																																																													
Control Test	0.200	11:48																																																																																																																																																													
Air Blank	0.000	11:49																																																																																																																																																													
Control Test	0.200	11:49																																																																																																																																																													
Air Blank	0.000	11:50																																																																																																																																																													
Control Test	0.200	11:50																																																																																																																																																													
Air Blank	0.000	11:51																																																																																																																																																													
Control Test	0.200	11:51																																																																																																																																																													
Air Blank	0.000	11:52																																																																																																																																																													
Control Test	0.200	11:52																																																																																																																																																													
Average	0.200																																																																																																																																																														
Std Dev	0.005																																																																																																																																																														
Rel. Std. Dev(%)	2.287																																																																																																																																																														
Test	g/210L	Time																																																																																																																																																													
Air Blank	0.000	11:27																																																																																																																																																													
Control Test	0.080	11:27																																																																																																																																																													
Air Blank	0.000	11:28																																																																																																																																																													
Control Test	0.080	11:28																																																																																																																																																													
Air Blank	0.000	11:29																																																																																																																																																													
Control Test	0.080	11:29																																																																																																																																																													
Air Blank	0.000	11:30																																																																																																																																																													
Control Test	0.080	11:30																																																																																																																																																													
Average	0.080																																																																																																																																																														
Std Dev	0.006																																																																																																																																																														
Rel. Std. Dev(%)	7.187																																																																																																																																																														

# Florida Department of Law Enforcement Alcohol Testing Program

## DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: HOLMES BEACH PD  
Time of Inspection: 09:22

Date of Inspection: 02/06/2026

Serial Number: 80-006692  
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG510701 Exp: 04/17/2027
0.000	0.052	0.083	0.201	0.082
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.082	0.202	0.081
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.082
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.081
0.000	0.052	0.083	0.202	0.081


Standard Deviations	0.0000	0.0003	0.0003	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0002 Number of Simulators Used: 5

Remarks:

The above instrument complies (  ) does not comply (  ) with Chapter 11D-8, FAC.


I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

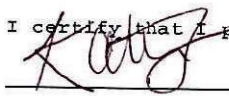

KATIE T SPEARIN

Signature and Printed Name

02/06/2026  
Date

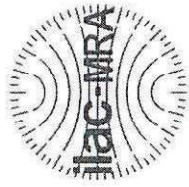

# Calibration Certificate

Florida Department of Law Enforcement  
Alcohol Testing Program  
2331 Phillips Road  
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-006692, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-006692</u>	UNCERTAINTY* ±	
Owning Agency:	<u>HOLMES BEACH PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>02/06/2026</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>09:22</u>	0.200 g/ 210 L	0.008
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.  
Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.  
\*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).  
The instrument results before and after any adjustment are found in the associated pre and post stability checks.

### TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

02/06/2026

\_\_\_\_\_  
Date  
**KATIE T SPEARIN,**  
Department Inspector

FDLE/ATP Form 69 January 2026  
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality