

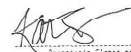
Stability Checks

0.050 g/210L	0.080 g/210L	0.200 g/210L	DGS 0.080 g/210L																																																																																																																																																
0.047 to 0.053 g/210L	0.077 to 0.083 g/210L	0.194 to 0.206 g/210L	0.077 to 0.083 g/210L ±0.003 g/210L of Wet																																																																																																																																																
Performed Root Case Analysis	Performed Root Case Analysis	Performed Root Case Analysis	Performed Root Case Analysis																																																																																																																																																
FFWCC Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-000892 08/27/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>07:41</td></tr> <tr><td>Control Test</td><td>0.153</td><td>07:42</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:42</td></tr> <tr><td>Control Test</td><td>0.051</td><td>07:43</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:43</td></tr> <tr><td>Control Test</td><td>0.051</td><td>07:44</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:45</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0517</td><td></td></tr> <tr><td>Std Dev</td><td>0.1012</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>2.2349</td><td></td></tr> </tbody> </table> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	07:41	Control Test	0.153	07:42	Air Blank	0.000	07:42	Control Test	0.051	07:43	Air Blank	0.000	07:43	Control Test	0.051	07:44	Air Blank	0.000	07:45	Control Test Stats			Average	0.0517		Std Dev	0.1012		Rel Std Dev(%)	2.2349		FFWCC Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-000892 08/27/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>07:48</td></tr> <tr><td>Control Test</td><td>0.082</td><td>07:49</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:50</td></tr> <tr><td>Control Test</td><td>0.081</td><td>07:50</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:51</td></tr> <tr><td>Control Test</td><td>0.082</td><td>07:52</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:52</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0817</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7070</td><td></td></tr> </tbody> </table> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	07:48	Control Test	0.082	07:49	Air Blank	0.000	07:50	Control Test	0.081	07:50	Air Blank	0.000	07:51	Control Test	0.082	07:52	Air Blank	0.000	07:52	Control Test Stats			Average	0.0817		Std Dev	0.0006		Rel Std Dev(%)	0.7070		FFWCC Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-000892 08/27/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>08:01</td></tr> <tr><td>Control Test</td><td>0.206</td><td>08:02</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>08:03</td></tr> <tr><td>Control Test</td><td>0.204</td><td>08:03</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>08:04</td></tr> <tr><td>Control Test</td><td>0.204</td><td>08:05</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>08:05</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.2047</td><td></td></tr> <tr><td>Std Dev</td><td>0.1012</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.5642</td><td></td></tr> </tbody> </table> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	08:01	Control Test	0.206	08:02	Air Blank	0.000	08:03	Control Test	0.204	08:03	Air Blank	0.000	08:04	Control Test	0.204	08:05	Air Blank	0.000	08:05	Control Test Stats			Average	0.2047		Std Dev	0.1012		Rel Std Dev(%)	0.5642		FFWCC Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-000892 08/27/2025 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>07:32</td></tr> <tr><td>Control Test</td><td>0.078</td><td>07:32</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:33</td></tr> <tr><td>Control Test</td><td>0.079</td><td>07:33</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:34</td></tr> <tr><td>Control Test</td><td>0.080</td><td>07:34</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>07:34</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0790</td><td></td></tr> <tr><td>Std Dev</td><td>0.0010</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.2658</td><td></td></tr> </tbody> </table> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	07:32	Control Test	0.078	07:32	Air Blank	0.000	07:33	Control Test	0.079	07:33	Air Blank	0.000	07:34	Control Test	0.080	07:34	Air Blank	0.000	07:34	Control Test Stats			Average	0.0790		Std Dev	0.0010		Rel Std Dev(%)	1.2658	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	07:41																																																																																																																																																	
Control Test	0.153	07:42																																																																																																																																																	
Air Blank	0.000	07:42																																																																																																																																																	
Control Test	0.051	07:43																																																																																																																																																	
Air Blank	0.000	07:43																																																																																																																																																	
Control Test	0.051	07:44																																																																																																																																																	
Air Blank	0.000	07:45																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0517																																																																																																																																																		
Std Dev	0.1012																																																																																																																																																		
Rel Std Dev(%)	2.2349																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	07:48																																																																																																																																																	
Control Test	0.082	07:49																																																																																																																																																	
Air Blank	0.000	07:50																																																																																																																																																	
Control Test	0.081	07:50																																																																																																																																																	
Air Blank	0.000	07:51																																																																																																																																																	
Control Test	0.082	07:52																																																																																																																																																	
Air Blank	0.000	07:52																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0817																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7070																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	08:01																																																																																																																																																	
Control Test	0.206	08:02																																																																																																																																																	
Air Blank	0.000	08:03																																																																																																																																																	
Control Test	0.204	08:03																																																																																																																																																	
Air Blank	0.000	08:04																																																																																																																																																	
Control Test	0.204	08:05																																																																																																																																																	
Air Blank	0.000	08:05																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.2047																																																																																																																																																		
Std Dev	0.1012																																																																																																																																																		
Rel Std Dev(%)	0.5642																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	07:32																																																																																																																																																	
Control Test	0.078	07:32																																																																																																																																																	
Air Blank	0.000	07:33																																																																																																																																																	
Control Test	0.079	07:33																																																																																																																																																	
Air Blank	0.000	07:34																																																																																																																																																	
Control Test	0.080	07:34																																																																																																																																																	
Air Blank	0.000	07:34																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0790																																																																																																																																																		
Std Dev	0.0010																																																																																																																																																		
Rel Std Dev(%)	1.2658																																																																																																																																																		

NOTE: 0.200 g/210L was repeated, but not needed. SLH 9/12/25

FWCC
Intelligence - Alcohol Analyzer
Model 8000 SN 00-110090
08/27/2025
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	07:56
Control Test	0.205	07:57
Air Blank	0.000	07:58
Control Test	0.204	07:58
Air Blank	0.000	07:59
Control Test	0.204	08:00
Air Blank	0.000	08:00
Control Test Status		
Average	0.2047	
Std Dev	0.0010	
Rel Std Dev(%)	0.5642	


Operator's Signature

FWCC
Intelligence - Alcohol Analyzer
Model 8000 SN 00-110090
08/27/2025
Software: 8100.27

Test	g/210L	Time
Air Blank	0.000	07:56
Control Test	0.056	07:56
Air Blank	0.000	07:57
Control Test	0.053	07:58
Air Blank	0.000	07:58
Control Test	0.051	07:59
Air Blank	0.000	07:40
Control Test Status		
Average	0.0533	
Std Dev	0.0025	
Rel Std Dev(%)	4.7166	


Operator's Signature

Florida Department of Law Enforcement

Alcohol Testing Program

AGENCY INSPECTION REPORT - INTOXILYZER 8000

Agency: FFWCC

Time of Inspection: 09:42

Date of Inspection: 09/10/2025

Serial Number: 80-000892

Software: 8100.27

Check or Test	YES	NO
Date and/or Time Adjusted		No
Diagnostic Check (Pre-Inspection): OK		No
Alcohol Free Subject Test: 0.000		No
Mouth Alcohol Test: Slope Not Met		No
Interferent Detect Test: Interferent Detect		No
Diagnostic Check (Post-Inspection): OK		No

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:_____ Exp:_____	0.08g/210L Test (g/210L) Lot#:_____ Exp:_____	0.20g/210L Test (g/210L) Lot#:_____ Exp:_____	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:_____ Exp:_____

Number of Simulators Used: _____

Remarks:

BYPASS AI FOR OPERATION, COMPLIANCE UNDETERMINED

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I hold a valid Florida Department of Law Enforcement Agency Inspector Permit and that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.


 LEANDRA HIGGINBOTHAM
 Signature and Printed Name

09/10/2025
 Date

Flow Calibration

Root Cause Analysis Performed Prior



yes

Performed by SLH

80-000892
SMT 9/10/25

INTOXILYZER 8000
Instrument Initialization
08:31 09/10/2025

FFWCC
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-000892
09/10/2025
Software: 8100.27

Flow Rate Calibration*****

1: Rate (Liters/min) = 5

SQRT(Diff) = 7.070

2: Rate (Liters/min) = 15

SQRT(Diff) = 11.574

3: Rate (Liters/min) = 30

SQRT(Diff) = 20.344

Dependent Data Scale Factor = 100000 L/min

Independent Data Scale Factor = 256

Rounded Slope = 726

Rounded Intercept = -750332

Correlation = 0.99766

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: FFWCC

Time of Inspection: 15:11

Date of Inspection: 09/10/2025

Serial Number: 80-000892

Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.050	0.081	0.203	0.078
0.000	0.050	0.082	0.203	0.077
0.000	0.049	0.081	0.202	0.077
0.000	0.051	0.081	0.202	0.077
0.000	0.051	0.081	0.202	0.077
0.000	0.050	0.080	0.202	0.077
0.000	0.051	0.081	0.203	0.077
0.000	0.051	0.082	0.204	0.078
0.000	0.050	0.082	0.204	0.077
0.000	0.051	0.081	0.204	0.077

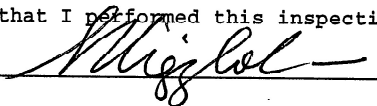
Standard Deviations	0.0006	0.0006	0.0008	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0006 Number of Simulators Used: 5

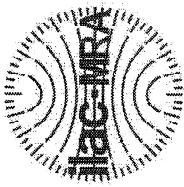
Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.


LEANDRA HIGGINBOTHAM
Signature and Printed Name

09/10/2025
Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-000892, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-000892</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>FFWCC</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>09/10/2025</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>15:11</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.
*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3)

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.
This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.


LEANDRA HIGINBOTHAM,
Department Inspector

09/10/2025
Date

FDLE/ATP Form 69 October 2024
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality