

Alcohol Testing Program - Instrument Processing Sheet

Agency: Broward County Sheriff's Office **Instrument Serial Number:** 80-007108

Date In: 3/19/2026 **DI Completion Date:** 4/28/2026 ☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake By: <u>TDG</u> Date: <u>3/20/2026</u>		Quality Checks By: <u>TDG</u> Date: <u>See Notes</u>		Flow Adjustment By: _____																																																											
☒ Annual ☐ Dropped Off ☐ Registration ☐ Return from CMI / EE ☐ Training Instrument Visual Inspection ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/Accessories ☒ Power Cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes: Left keyboard pin and corner of casing is broken off. Needs new battery.		☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value: <u>166</u> ☒ Flow Verification (L/s) Flow Column #: <u>ATP106</u> 32 mm <u>0.160</u> (.139-.169) 36 mm <u>0.171</u> (.156-.190) 53 mm <u>0.242</u> (.228-.278) 103 mm <u>0.496</u> (.447-.547) ☒ Barometric Pressure Check Gauge ID #: <u>26932</u> Gauge: <u>1014</u> Instrument: <u>1014</u> ☒ Stability Checks		Flow Column #: _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value: _____ ☐ Post Adjustment Verification (L/S) Flow Column #: _____ 32 mm _____ (.139-.169) 36 mm _____ (.156-.190) 53 mm _____ (.228-.278) 103 mm _____ (.447-.547)																																																											
						<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="width: 15%;">Simulator</th> <th style="width: 20%;">Serial #</th> <th style="width: 25%;">Lot#/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td>MP6286</td> <td>202406K 6/19/2026</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> <td>202406L 6/19/2026</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> <td>202406N 6/20/2026</td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td>AG429602 10/22/2026</td> </tr> </tbody> </table>		Simulator	Serial #	Lot#/Exp	0.050	MP6286	202406K 6/19/2026	0.080	MP6287	202406L 6/19/2026	0.200	MP6288	202406N 6/20/2026	0.080 DGS	N/A	AG429602 10/22/2026	Maintenance By: <u>TDG</u> Date: <u>4/24/2026</u> ☒ Battery Replacement ☐ Dry Gas Regulator Replacement ☐ Tank Sensor Tare ☐ Breath Tube Replacement ☐ Other: _____																																								
						Simulator	Serial #	Lot#/Exp																																																							
						0.050	MP6286	202406K 6/19/2026																																																							
						0.080	MP6287	202406L 6/19/2026																																																							
0.200	MP6288	202406N 6/20/2026																																																													
0.080 DGS	N/A	AG429602 10/22/2026																																																													
Optical Bench Adjustment By: _____		Department Inspection By: <u>TDG</u>																																																													
Barometric Pressure Gauge: _____ ID#: _____ <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="width: 15%;">Simulator</th> <th style="width: 20%;">Serial #</th> <th style="width: 25%;">Lot #</th> <th style="width: 20%;">Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> ☐ Post Optical Bench Adjustment Stability Checks <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="width: 15%;">Simulator</th> <th style="width: 20%;">Serial #</th> <th style="width: 25%;">Lot #</th> <th style="width: 20%;">Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> Gauge ID #: _____ Gauge: _____ Instrument: _____		Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			Barometric Pressure ID#: <u>26932</u> Gauge: <u>1019</u> Instrument: <u>1019</u> Mouth Alcohol Solution Lot #: <u>2025-D</u> Exp: <u>9/25/2027</u> Acetone Stock Solution Lot #: <u>2024-B</u> Exp: <u>7/19/2026</u> <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="width: 50%;">Simulator</th> <th style="width: 50%;">Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6284</td> </tr> <tr> <td>Interferent</td> <td>MP6285</td> </tr> <tr> <td>0.050</td> <td>MP6286</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> </tr> </tbody> </table>		Simulator	Serial Number	0.000	MP6284	Interferent	MP6285	0.050	MP6286	0.080	MP6287	0.200	MP6288
Simulator	Serial #	Lot #	Expiration																																																												
0.000		N/A	N/A																																																												
0.040																																																															
0.100																																																															
0.200																																																															
0.300																																																															
0.080 DGS	N/A																																																														
Simulator	Serial #	Lot #	Expiration																																																												
0.050																																																															
0.080																																																															
0.200																																																															
0.080 DGS	N/A																																																														
Simulator	Serial Number																																																														
0.000	MP6284																																																														
Interferent	MP6285																																																														
0.050	MP6286																																																														
0.080	MP6287																																																														
0.200	MP6288																																																														
Notes/Suggested Service: Checked breath tube and replaced o-rings on 3/20. Verified set-up on 4/24 during battery change. Finished Quality Checks on 4/27. (TDG 4/27/26)		Attachments ☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Optical Bench Adjustment ☐ Post-Stability Checks ☐ Flow Adjustment ☐ Form 40 ☐ Other: _____																																																													
		☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use																																																													
		☒ Conduct an Agency Inspection Before Evidentiary Use																																																													
		LeAndra Higginbotham Digitally signed by LeAndra Higginbotham Date: 2026.05.01 17:22:51 -04'00' Wen-Chi Pierson Digitally signed by Wen-Chi Pierson Date: 2026.05.04 10:59:55 -04'00'																																																													
		Tech Review Admin Review																																																													

Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083 ≤ 0.003 of Wet																																																																																																																																																
BROWARD COUNTY SO Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-007108 04/27/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:36</td></tr> <tr><td>Control Test</td><td>0.048</td><td>14:36</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:37</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:38</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:38</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:39</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:39</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0487</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.1863</td><td></td></tr> </tbody> </table> Operator's Signature: <i>MC</i>	Test	g/210L	Time	Air Blank	0.000	14:36	Control Test	0.048	14:36	Air Blank	0.000	14:37	Control Test	0.049	14:38	Air Blank	0.000	14:38	Control Test	0.049	14:39	Air Blank	0.000	14:39	Control Test Stats			Average	0.0487		Std Dev	0.0006		Rel Std Dev(%)	1.1863		BROWARD COUNTY SO Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-007108 04/27/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:45</td></tr> <tr><td>Control Test</td><td>0.079</td><td>14:46</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:47</td></tr> <tr><td>Control Test</td><td>0.079</td><td>14:47</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:48</td></tr> <tr><td>Control Test</td><td>0.079</td><td>14:49</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:49</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0790</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature: <i>MC</i>	Test	g/210L	Time	Air Blank	0.000	14:45	Control Test	0.079	14:46	Air Blank	0.000	14:47	Control Test	0.079	14:47	Air Blank	0.000	14:48	Control Test	0.079	14:49	Air Blank	0.000	14:49	Control Test Stats			Average	0.0790		Std Dev	0.0000		Rel Std Dev(%)	0.0000		BROWARD COUNTY SO Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-007108 04/27/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:45</td></tr> <tr><td>Control Test</td><td>0.198</td><td>14:46</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:47</td></tr> <tr><td>Control Test</td><td>0.199</td><td>14:48</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:49</td></tr> <tr><td>Control Test</td><td>0.198</td><td>14:49</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:49</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1983</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2911</td><td></td></tr> </tbody> </table> Operator's Signature: <i>MC</i>	Test	g/210L	Time	Air Blank	0.000	14:45	Control Test	0.198	14:46	Air Blank	0.000	14:47	Control Test	0.199	14:48	Air Blank	0.000	14:49	Control Test	0.198	14:49	Air Blank	0.000	14:49	Control Test Stats			Average	0.1983		Std Dev	0.0006		Rel Std Dev(%)	0.2911		BROWARD COUNTY SO Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-007108 04/27/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:48</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:29</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:29</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:30</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:30</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:30</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:30</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0800</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> Operator's Signature: <i>MC</i>	Test	g/210L	Time	Air Blank	0.000	14:48	Control Test	0.080	14:29	Air Blank	0.000	14:29	Control Test	0.080	14:30	Air Blank	0.000	14:30	Control Test	0.080	14:30	Air Blank	0.000	14:30	Control Test Stats			Average	0.0800		Std Dev	0.0000		Rel Std Dev(%)	0.0000	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:36																																																																																																																																																	
Control Test	0.048	14:36																																																																																																																																																	
Air Blank	0.000	14:37																																																																																																																																																	
Control Test	0.049	14:38																																																																																																																																																	
Air Blank	0.000	14:38																																																																																																																																																	
Control Test	0.049	14:39																																																																																																																																																	
Air Blank	0.000	14:39																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0487																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	1.1863																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:45																																																																																																																																																	
Control Test	0.079	14:46																																																																																																																																																	
Air Blank	0.000	14:47																																																																																																																																																	
Control Test	0.079	14:47																																																																																																																																																	
Air Blank	0.000	14:48																																																																																																																																																	
Control Test	0.079	14:49																																																																																																																																																	
Air Blank	0.000	14:49																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0790																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:45																																																																																																																																																	
Control Test	0.198	14:46																																																																																																																																																	
Air Blank	0.000	14:47																																																																																																																																																	
Control Test	0.199	14:48																																																																																																																																																	
Air Blank	0.000	14:49																																																																																																																																																	
Control Test	0.198	14:49																																																																																																																																																	
Air Blank	0.000	14:49																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.1983																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.2911																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:48																																																																																																																																																	
Control Test	0.080	14:29																																																																																																																																																	
Air Blank	0.000	14:29																																																																																																																																																	
Control Test	0.080	14:30																																																																																																																																																	
Air Blank	0.000	14:30																																																																																																																																																	
Control Test	0.080	14:30																																																																																																																																																	
Air Blank	0.000	14:30																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0800																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		

Florida Department of Law Enforcement Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: BROWARD COUNTY SO
Time of Inspection: 11:09

Date of Inspection: 04/28/2026

Serial Number: 80-007108
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202603B Exp: 03/03/2028	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.048	0.078	0.197	0.080
0.000	0.048	0.078	0.198	0.080
0.000	0.048	0.078	0.198	0.080
0.000	0.048	0.079	0.198	0.080
0.000	0.048	0.078	0.198	0.080
0.000	0.049	0.078	0.198	0.080
0.000	0.049	0.078	0.198	0.080
0.000	0.049	0.079	0.198	0.080
0.000	0.049	0.078	0.198	0.079
0.000	0.049	0.078	0.198	0.079

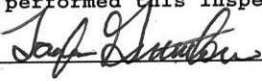
Standard Deviations	0.0005	0.0004	0.0003	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

 TAYLOR D GUTSCHOW
Signature and Printed Name

04/28/2026
Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
4700 Terminal Drive, Suite 1
Ft. Myers, FL 33907

This is to certify the calibration of Intoxilyzer 8000 serial number 80-007108, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-007108</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>BROWARD COUNTY SO</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>04/28/2026</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>11:09</u>	0.200 g/ 210 L	0.008
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

04/28/2026

Date

**Taylor
Gutschow**

Digitally signed by Taylor
Gutschow
Date: 2026.04.28 12:52:22
-04'00'

**TAYLOR D GUTSCHOW,
Department Inspector**