

Alcohol Testing Program - Instrument Processing Sheet

Agency: COCOA PD Instrument Serial Number: 80-001000
 Date In: 7/9/2026 DI Completion Date: 7/24/2026 ☒ Ship ☐ P/U ☐ H/D ☐ CMI ☐ EE

Intake By: <u>WKP</u> Date: <u>7/9/2026</u> ☒ Annual ☐ Dropped Off ☐ Registration ☐ Return from CMI / EE ☐ Training Instrument Visual Inspection ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/Accessories ☒ Power Cord ☐ Printer Cable ☒ Static Bag ☐ 12V DC Cable Notes:	Quality Checks By: <u>WKP</u> Date: <u>7/13/2026</u> ☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value: <u>165</u> ☒ Flow Verification (L/s) Flow Column #: <u>ATP 105</u> 32 mm <u>0.148</u> (.139-.169) 36 mm <u>0.160</u> (.156-.190) 53 mm <u>0.238</u> (.228-.278) 103 mm <u>0.546</u> (.447-.547) ☒ Barometric Pressure Check Gauge ID #: <u>34419</u> Gauge: <u>1017</u> Instrument: <u>1012</u> ☒ Stability Checks	Flow Adjustment By: _____ Flow Column #: _____ ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value: _____ ☐ Post Adjustment Verification (L/S) Flow Column #: _____ 32 mm _____ (.139-.169) 36 mm _____ (.156-.190) 53 mm _____ (.228-.278) 103 mm _____ (.447-.547)																																		
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot#/Exp</th> <th>Maintenance By:</th> <th>Date:</th> </tr> </thead> <tbody> <tr> <td rowspan="2">0.050</td> <td rowspan="2">MP5088</td> <td>202603A</td> <td rowspan="2">☐ Battery Replacement</td> <td rowspan="2"></td> </tr> <tr> <td>3/3/2028</td> </tr> <tr> <td rowspan="2">0.080</td> <td rowspan="2">MP5089</td> <td>202603B</td> <td rowspan="2">☐ Dry Gas Regulator Replacement</td> <td rowspan="2"></td> </tr> <tr> <td>3/3/2028</td> </tr> <tr> <td rowspan="2">0.200</td> <td rowspan="2">MP5090</td> <td>202603D</td> <td rowspan="2">☐ Tank Sensor Tare</td> <td rowspan="2"></td> </tr> <tr> <td>3/4/2028</td> </tr> <tr> <td rowspan="2">0.080 DGS</td> <td rowspan="2">N/A</td> <td>23825080A1</td> <td rowspan="2">☐ Breath Tube Replacement</td> <td rowspan="2"></td> </tr> <tr> <td>10/5/2027</td> </tr> <tr> <td colspan="2"></td> <td></td> <td>☐ Other:</td> <td></td> </tr> </tbody> </table>			Simulator	Serial #	Lot#/Exp	Maintenance By:	Date:	0.050	MP5088	202603A	☐ Battery Replacement		3/3/2028	0.080	MP5089	202603B	☐ Dry Gas Regulator Replacement		3/3/2028	0.200	MP5090	202603D	☐ Tank Sensor Tare		3/4/2028	0.080 DGS	N/A	23825080A1	☐ Breath Tube Replacement		10/5/2027				☐ Other:	
Simulator	Serial #	Lot#/Exp	Maintenance By:	Date:																																
0.050	MP5088	202603A	☐ Battery Replacement																																	
		3/3/2028																																		
0.080	MP5089	202603B	☐ Dry Gas Regulator Replacement																																	
		3/3/2028																																		
0.200	MP5090	202603D	☐ Tank Sensor Tare																																	
		3/4/2028																																		
0.080 DGS	N/A	23825080A1	☐ Breath Tube Replacement																																	
		10/5/2027																																		
			☐ Other:																																	
Optical Bench Adjustment By: _____		Department Inspection By: <u>SLH</u>																																		
Barometric Pressure Gauge: _____ ID#: _____		Barometric Pressure ID#: <u>28427</u>																																		
<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Gauge: <u>1014</u> Instrument: <u>1013</u> Mouth Alcohol Solution Lot #: <u>2025-D</u> Exp: <u>09/25/2027</u> Acetone Stock Solution Lot #: <u>2025-B</u> Exp: <u>09/22/2027</u>							
Simulator	Serial #	Lot #	Expiration																																	
0.000		N/A	N/A																																	
0.040																																				
0.100																																				
0.200																																				
0.300																																				
0.080 DGS	N/A																																			
☐ Post Optical Bench Adjustment Stability Checks		<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A																
Simulator	Serial #	Lot #	Expiration																																	
0.050																																				
0.080																																				
0.200																																				
0.080 DGS	N/A																																			
Gauge ID #: _____ Gauge: _____ Instrument: _____		<table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP5086</td> </tr> <tr> <td>Interferent</td> <td>MP5087</td> </tr> <tr> <td>0.050</td> <td>MP5088</td> </tr> <tr> <td>0.080</td> <td>MP5089</td> </tr> <tr> <td>0.200</td> <td>MP5090</td> </tr> </tbody> </table>	Simulator	Serial Number	0.000	MP5086	Interferent	MP5087	0.050	MP5088	0.080	MP5089	0.200	MP5090																						
Simulator	Serial Number																																			
0.000	MP5086																																			
Interferent	MP5087																																			
0.050	MP5088																																			
0.080	MP5089																																			
0.200	MP5090																																			
Attachments ☒ Form 41 ☒ Stability Checks ☒ Calibration Certificate ☐ Optical Bench Adjustment		☐ Post-Stability Checks ☐ Flow Adjustment ☐ Form 40 ☐ Other:																																		
Notes/Suggested Service:		☒ Instrument Complies with Chapter 11D-8, FAC ☐ Instrument Does Not Comply with Chapter 11D-8, FAC ☒ Return to/Place into Evidentiary Use ☐ Remain Out of Evidentiary Use ☒ Conduct an Agency Inspection Before Evidentiary Use																																		
Digitally signed by Taylor Gutschow Date: 2026.07.27 13:18:45 -04'00'		Digitally signed by Shayla Platt Date: 2026.07.29 12:43:29 -04'00'																																		
Tech Review		Admin Review																																		

Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083 ☒ ≤ 0.003 of Wet ☒																																																																																																																																																
COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001000 07/13/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:02</td></tr> <tr><td>Control Test</td><td>0.050</td><td>14:03</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:03</td></tr> <tr><td>Control Test</td><td>0.050</td><td>14:04</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:05</td></tr> <tr><td>Control Test</td><td>0.050</td><td>14:05</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:06</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0500</td><td></td></tr> <tr><td>Std Dev</td><td>0.0000</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.0000</td><td></td></tr> </tbody> </table> <i>menpi</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	14:02	Control Test	0.050	14:03	Air Blank	0.000	14:03	Control Test	0.050	14:04	Air Blank	0.000	14:05	Control Test	0.050	14:05	Air Blank	0.000	14:06	Control Test Stats			Average	0.0500		Std Dev	0.0000		Rel Std Dev(%)	0.0000		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001000 07/13/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:16</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:16</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:17</td></tr> <tr><td>Control Test</td><td>0.079</td><td>14:17</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:18</td></tr> <tr><td>Control Test</td><td>0.079</td><td>14:19</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:19</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0793</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7277</td><td></td></tr> </tbody> </table> <i>menpi</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	14:16	Control Test	0.080	14:16	Air Blank	0.000	14:17	Control Test	0.079	14:17	Air Blank	0.000	14:18	Control Test	0.079	14:19	Air Blank	0.000	14:19	Control Test Stats			Average	0.0793		Std Dev	0.0006		Rel Std Dev(%)	0.7277		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001000 07/13/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:22</td></tr> <tr><td>Control Test</td><td>0.199</td><td>14:23</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:23</td></tr> <tr><td>Control Test</td><td>0.197</td><td>14:24</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:25</td></tr> <tr><td>Control Test</td><td>0.196</td><td>14:25</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:26</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1973</td><td></td></tr> <tr><td>Std Dev</td><td>0.0015</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7741</td><td></td></tr> </tbody> </table> <i>menpi</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	14:22	Control Test	0.199	14:23	Air Blank	0.000	14:23	Control Test	0.197	14:24	Air Blank	0.000	14:25	Control Test	0.196	14:25	Air Blank	0.000	14:26	Control Test Stats			Average	0.1973		Std Dev	0.0015		Rel Std Dev(%)	0.7741		COCOA PD Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-001000 07/13/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:12</td></tr> <tr><td>Control Test</td><td>0.082</td><td>14:12</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:13</td></tr> <tr><td>Control Test</td><td>0.081</td><td>14:13</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:14</td></tr> <tr><td>Control Test</td><td>0.081</td><td>14:14</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:14</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0813</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7099</td><td></td></tr> </tbody> </table> <i>menpi</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	14:12	Control Test	0.082	14:12	Air Blank	0.000	14:13	Control Test	0.081	14:13	Air Blank	0.000	14:14	Control Test	0.081	14:14	Air Blank	0.000	14:14	Control Test Stats			Average	0.0813		Std Dev	0.0006		Rel Std Dev(%)	0.7099	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:02																																																																																																																																																	
Control Test	0.050	14:03																																																																																																																																																	
Air Blank	0.000	14:03																																																																																																																																																	
Control Test	0.050	14:04																																																																																																																																																	
Air Blank	0.000	14:05																																																																																																																																																	
Control Test	0.050	14:05																																																																																																																																																	
Air Blank	0.000	14:06																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0500																																																																																																																																																		
Std Dev	0.0000																																																																																																																																																		
Rel Std Dev(%)	0.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:16																																																																																																																																																	
Control Test	0.080	14:16																																																																																																																																																	
Air Blank	0.000	14:17																																																																																																																																																	
Control Test	0.079	14:17																																																																																																																																																	
Air Blank	0.000	14:18																																																																																																																																																	
Control Test	0.079	14:19																																																																																																																																																	
Air Blank	0.000	14:19																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0793																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7277																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:22																																																																																																																																																	
Control Test	0.199	14:23																																																																																																																																																	
Air Blank	0.000	14:23																																																																																																																																																	
Control Test	0.197	14:24																																																																																																																																																	
Air Blank	0.000	14:25																																																																																																																																																	
Control Test	0.196	14:25																																																																																																																																																	
Air Blank	0.000	14:26																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.1973																																																																																																																																																		
Std Dev	0.0015																																																																																																																																																		
Rel Std Dev(%)	0.7741																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:12																																																																																																																																																	
Control Test	0.082	14:12																																																																																																																																																	
Air Blank	0.000	14:13																																																																																																																																																	
Control Test	0.081	14:13																																																																																																																																																	
Air Blank	0.000	14:14																																																																																																																																																	
Control Test	0.081	14:14																																																																																																																																																	
Air Blank	0.000	14:14																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0813																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7099																																																																																																																																																		

Florida Department of Law Enforcement

Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: COCOA PD
Time of Inspection: 15:01

Date of Inspection: 07/24/2026

Serial Number: 80-001000
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202603A Exp: 03/03/2028	0.08g/210L Test (g/210L) Lot#:202603B Exp: 03/03/2028	0.20g/210L Test (g/210L) Lot#:202603D Exp: 03/04/2028	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:23825080A1 Exp: 10/05/2027
0.000	0.050	0.077	0.196	0.081
0.000	0.052	0.077	0.196	0.080
0.000	0.051	0.077	0.196	0.080
0.000	0.051	0.076	0.202	0.081
0.000	0.051	0.078	0.190	0.081
0.000	0.051	0.077	0.190	0.081
0.000	0.051	0.077	0.190	0.081
0.000	0.051	0.078	0.190	0.081
0.000	0.052	0.078	0.190	0.081
0.000	0.051	0.077	0.191	0.080

Standard Deviations	0.0005	0.0006	0.0041	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0014 Number of Simulators Used: 5


Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.


LEANDRA HIGGINBOTHAM
Signature and Printed Name

07/24/2026
Date

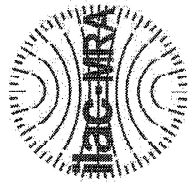

ANAB
ANSI National Accreditation Board
ACCREDITED
ISO/IEC 17025
FORENSIC CALIBRATION
LABORATORY

Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001000, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-001000</u>	UNCERTAINTY* $\pm$
Owning Agency:	<u>COCOA PD</u>	0.050 g/ 210 L 0.004
Calibration Date:	<u>07/24/2026</u>	0.080 g/ 210 L 0.004
Calibration Time:	<u>15:01</u>	0.200 g/ 210 L 0.008
		0.080 g/ 210 L Dry Gas Control 0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Mejlatte

07/24/2026

Date

LEANDRA HIGGINBOTHAM,

Department Inspector

FDLE/ATP Form 69 January 2026
Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality