



# INSTRUMENT PROCESSING SHEET

Agency Hialeah Police DepartmentS/N 80-000876Florida Department of  
Law EnforcementDate In 04/14/2025 DI Completion Date 05/16/2025 Ship  P/U  H/D  CMI  EE

Intake	Quality Checks	Flow Calibration																							
By <u>TDG</u> Date <u>05/05/2025</u> <input checked="" type="checkbox"/> Annual <input type="checkbox"/> Registration <input type="checkbox"/> Return from CMI / EE  Visual Inspection: <input checked="" type="checkbox"/> Case <input checked="" type="checkbox"/> Handle <input checked="" type="checkbox"/> Keyboard <input checked="" type="checkbox"/> Dry Gas Shelf <input checked="" type="checkbox"/> Feet <input checked="" type="checkbox"/> Breath Tube <input checked="" type="checkbox"/> Ports <input checked="" type="checkbox"/> Screws Tight  Other Equipment/ Accessories: <input checked="" type="checkbox"/> Power cord <input type="checkbox"/> Printer Cable <input checked="" type="checkbox"/> Static Bag <input type="checkbox"/> 12V DC Cable  Notes: _____ _____ _____ _____ _____ _____	By <u>TDG</u> Date <u>05/06/2025</u> <input checked="" type="checkbox"/> Breath Tube Screen <input checked="" type="checkbox"/> Replace External O-Rings <input checked="" type="checkbox"/> Instrument Set Up Verified <input checked="" type="checkbox"/> R-Value <u>122</u> <input checked="" type="checkbox"/> Flow Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>0.140</u> (.139 - .169) 36 mm <u>0.152*</u> (.156 - .190) 53 mm <u>0.234</u> (.228 - .278) 103 mm <u>0.496</u> (.447 - .547) <input checked="" type="checkbox"/> Barometric Pressure Check Gauge ID # <u>28199</u> <input checked="" type="checkbox"/> Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #/Exp</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td rowspan="2">MP6286</td> <td>202406K</td> </tr> <tr> <td></td> <td>06/19/2026</td> </tr> <tr> <td>0.080</td> <td rowspan="2">MP6287</td> <td>202406L</td> </tr> <tr> <td></td> <td>06/19/2026</td> </tr> <tr> <td>0.200</td> <td rowspan="2">MP6288</td> <td>202406N</td> </tr> <tr> <td></td> <td>06/20/2026</td> </tr> <tr> <td>0.080 DGS</td> <td rowspan="2">N/A</td> <td>AG429602</td> </tr> <tr> <td></td> <td>10/22/2026</td> </tr> </tbody> </table>	Simulator	Serial #	Lot #/Exp	0.050	MP6286	202406K		06/19/2026	0.080	MP6287	202406L		06/19/2026	0.200	MP6288	202406N		06/20/2026	0.080 DGS	N/A	AG429602		10/22/2026	By <u>TDG</u> Date <u>05/15/2025</u> Flow Column # <u>ATP104</u> <input checked="" type="checkbox"/> 5L/min – 17mm <input checked="" type="checkbox"/> 15L/min – 53mm <input checked="" type="checkbox"/> 30L/min – 103mm <input checked="" type="checkbox"/> R-Value <u>120</u> <input checked="" type="checkbox"/> Post Calibration Verification (L/s) Flow Column # <u>ATP106</u> 32 mm <u>0.144</u> (.139 - .169) 36 mm <u>0.160</u> (.156 - .190) 53 mm <u>0.246</u> (.228 - .278) 103 mm <u>0.503</u> (.447 - .547)  Maintenance By _____ Date _____ <input type="checkbox"/> Battery Replacement <input type="checkbox"/> Dry Gas Regulator Replacement <input type="checkbox"/> Breath Tube Replacement <input type="checkbox"/> Other _____ _____ _____ _____
Simulator	Serial #	Lot #/Exp																							
0.050	MP6286	202406K																							
		06/19/2026																							
0.080	MP6287	202406L																							
		06/19/2026																							
0.200	MP6288	202406N																							
		06/20/2026																							
0.080 DGS	N/A	AG429602																							
		10/22/2026																							

Calibration Adjustment	Department Inspection																																																												
By _____ Barometric Pressure Gauge _____ ID # _____ <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td></td> <td>N/A</td> <td>N/A</td> </tr> <tr> <td>0.040</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.100</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.300</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table> <input type="checkbox"/> Post Calibration Adjustment Stability Checks <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial #</th> <th>Lot #</th> <th>Expiration</th> </tr> </thead> <tbody> <tr> <td>0.050</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.200</td> <td></td> <td></td> <td></td> </tr> <tr> <td>0.080 DGS</td> <td>N/A</td> <td></td> <td></td> </tr> </tbody> </table>	Simulator	Serial #	Lot #	Expiration	0.000		N/A	N/A	0.040				0.100				0.200				0.300				0.080 DGS	N/A			Simulator	Serial #	Lot #	Expiration	0.050				0.080				0.200				0.080 DGS	N/A			By <u>TDG</u> Barometric Pressure ID# <u>28199</u> Gauge <u>1017 / 1018</u> Instrument <u>1016 / 1017</u> Mouth Alcohol Solution Lot # <u>2024-A</u> Acetone Stock Solution Lot # <u>2023-B</u> <table border="1" style="width:100%; border-collapse: collapse;"> <thead> <tr> <th>Simulator</th> <th>Serial Number</th> </tr> </thead> <tbody> <tr> <td>0.000</td> <td>MP6284</td> </tr> <tr> <td>Interferent</td> <td>MP6285</td> </tr> <tr> <td>0.050</td> <td>MP6286</td> </tr> <tr> <td>0.080</td> <td>MP6287</td> </tr> <tr> <td>0.200</td> <td>MP6288</td> </tr> </tbody> </table> <b>Attachments</b> <input checked="" type="checkbox"/> Form 41 (x2) <input type="checkbox"/> Post-Stability Checks <input checked="" type="checkbox"/> Stability Checks <input checked="" type="checkbox"/> Flow Calibration <input checked="" type="checkbox"/> Calibration Certificate <input type="checkbox"/> Form 40 <input type="checkbox"/> Calibration Adjustment <input type="checkbox"/> Other _____	Simulator	Serial Number	0.000	MP6284	Interferent	MP6285	0.050	MP6286	0.080	MP6287	0.200	MP6288
Simulator	Serial #	Lot #	Expiration																																																										
0.000		N/A	N/A																																																										
0.040																																																													
0.100																																																													
0.200																																																													
0.300																																																													
0.080 DGS	N/A																																																												
Simulator	Serial #	Lot #	Expiration																																																										
0.050																																																													
0.080																																																													
0.200																																																													
0.080 DGS	N/A																																																												
Simulator	Serial Number																																																												
0.000	MP6284																																																												
Interferent	MP6285																																																												
0.050	MP6286																																																												
0.080	MP6287																																																												
0.200	MP6288																																																												

Notes/Suggested Service: <u>*Outside nominal. (TDG 5/7/25)</u>  <u>Flow cal adjust was not performed prior to inspection.</u> <u>Will perform flow cal adjust and repeat the inspection.</u> <u>(TDG 5/7/25)</u>  <u>The second inspection used the same standards and</u> <u>serial numbers as the first inspection. (TDG 5/16/25)</u>	<input checked="" type="checkbox"/> Instrument Complies with Chapter 11D-8, FAC <input type="checkbox"/> Instrument Does Not Comply with Chapter 11D-8, FAC <input checked="" type="checkbox"/> Return to/Place into Evidentiary Use <input type="checkbox"/> Remain Out of Evidentiary Use <input checked="" type="checkbox"/> Conduct an Agency Inspection Before Evidentiary Use <div style="text-align: center;">   <small>Phil Nicodemo</small>  <small>Digitally signed by Phil Nicodemo Date: 2025.05.21 14:55:19 -0400</small> </div> <div style="text-align: center;">   <small>Shayla Platt</small>  <small>Date: 2025.05.21 15:02:42 -04'00'</small> </div>
Tech Review / Date _____	Admin Review / Date _____

# Flow Calibration Adjustment(s)

Performed by TDG

HIALEAH.PD  
Intoxilyzer = Alcohol Analyzer  
Model: 8000 SN 80-000876  
05/15/2025  
Software: 8100.27

Flow Rate Calibration\*\*\*\*\*  
1: Rate (Liters/min) = 5  
SQRT(Diff) = 6.480  
2: Rate (Liters/min) = 15  
SQRT(Diff) = 11.445  
3: Rate (Liters/min) = 30  
SQRT(Diff) = 20.855  
Dependent Data Scale Factor = 100000 L/min  
Independent Data Scale Factor = 256  
Rounded Slope = 672  
Rounded Intercept = -556987  
Correlation = 0.99810

# Stability Checks

0.05g/210L 0.047 to 0.053	0.08g/210L 0.077 to 0.083	0.20g/210L 0.194 to 0.206	DGS 0.08g/210L 0.077 to 0.083																																																																																																																																																
<p>✓</p>	<p>✓</p>	<p>✓</p>	<p>✓</p>																																																																																																																																																
<p>                     H/ALEAH PD                      Intoxilyzer - Alcohol Analyzer                      Model 8000 SN 80-000876                      05/06/2025                      Software: 8100.27                 </p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:00</td></tr> <tr><td>Control Test</td><td>0.051</td><td>14:01</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:02</td></tr> <tr><td>Control Test</td><td>0.049</td><td>14:02</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:03</td></tr> <tr><td>Control Test</td><td>0.050</td><td>14:04</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:04</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0500</td><td></td></tr> <tr><td>Std Dev</td><td>0.0010</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>2.0000</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>W</i></p>	Test	g/210L	Time	Air Blank	0.000	14:00	Control Test	0.051	14:01	Air Blank	0.000	14:02	Control Test	0.049	14:02	Air Blank	0.000	14:03	Control Test	0.050	14:04	Air Blank	0.000	14:04	Control Test Stats			Average	0.0500		Std Dev	0.0010		Rel Std Dev(%)	2.0000		<p>                     H/ALEAH PD                      Intoxilyzer - Alcohol Analyzer                      Model 8000 SN 80-000876                      05/06/2025                      Software: 8100.27                 </p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:09</td></tr> <tr><td>Control Test</td><td>0.081</td><td>14:10</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:11</td></tr> <tr><td>Control Test</td><td>0.080</td><td>14:11</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:12</td></tr> <tr><td>Control Test</td><td>0.081</td><td>14:13</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:13</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0807</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7157</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>M</i></p>	Test	g/210L	Time	Air Blank	0.000	14:09	Control Test	0.081	14:10	Air Blank	0.000	14:11	Control Test	0.080	14:11	Air Blank	0.000	14:12	Control Test	0.081	14:13	Air Blank	0.000	14:13	Control Test Stats			Average	0.0807		Std Dev	0.0006		Rel Std Dev(%)	0.7157		<p>                     H/ALEAH PD                      Intoxilyzer - Alcohol Analyzer                      Model 8000 SN 80-000876                      05/06/2025                      Software: 8100.27                 </p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>14:18</td></tr> <tr><td>Control Test</td><td>0.204</td><td>14:19</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:20</td></tr> <tr><td>Control Test</td><td>0.201</td><td>14:21</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:21</td></tr> <tr><td>Control Test</td><td>0.202</td><td>14:22</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>14:23</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.2023</td><td></td></tr> <tr><td>Std Dev</td><td>0.0015</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7550</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>M</i></p>	Test	g/210L	Time	Air Blank	0.000	14:18	Control Test	0.204	14:19	Air Blank	0.000	14:20	Control Test	0.201	14:21	Air Blank	0.000	14:21	Control Test	0.202	14:22	Air Blank	0.000	14:23	Control Test Stats			Average	0.2023		Std Dev	0.0015		Rel Std Dev(%)	0.7550		<p>                     H/ALEAH PD                      Intoxilyzer - Alcohol Analyzer                      Model 8000 SN 80-000876                      05/06/2025                      Software: 8100.27                 </p> <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>13:33</td></tr> <tr><td>Control Test</td><td>0.081</td><td>13:33</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:34</td></tr> <tr><td>Control Test</td><td>0.081</td><td>13:34</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:34</td></tr> <tr><td>Control Test</td><td>0.080</td><td>13:35</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>13:35</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0807</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7157</td><td></td></tr> </tbody> </table> <p>Operator's Signature <i>M</i></p>	Test	g/210L	Time	Air Blank	0.000	13:33	Control Test	0.081	13:33	Air Blank	0.000	13:34	Control Test	0.081	13:34	Air Blank	0.000	13:34	Control Test	0.080	13:35	Air Blank	0.000	13:35	Control Test Stats			Average	0.0807		Std Dev	0.0006		Rel Std Dev(%)	0.7157	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:00																																																																																																																																																	
Control Test	0.051	14:01																																																																																																																																																	
Air Blank	0.000	14:02																																																																																																																																																	
Control Test	0.049	14:02																																																																																																																																																	
Air Blank	0.000	14:03																																																																																																																																																	
Control Test	0.050	14:04																																																																																																																																																	
Air Blank	0.000	14:04																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0500																																																																																																																																																		
Std Dev	0.0010																																																																																																																																																		
Rel Std Dev(%)	2.0000																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:09																																																																																																																																																	
Control Test	0.081	14:10																																																																																																																																																	
Air Blank	0.000	14:11																																																																																																																																																	
Control Test	0.080	14:11																																																																																																																																																	
Air Blank	0.000	14:12																																																																																																																																																	
Control Test	0.081	14:13																																																																																																																																																	
Air Blank	0.000	14:13																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0807																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7157																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	14:18																																																																																																																																																	
Control Test	0.204	14:19																																																																																																																																																	
Air Blank	0.000	14:20																																																																																																																																																	
Control Test	0.201	14:21																																																																																																																																																	
Air Blank	0.000	14:21																																																																																																																																																	
Control Test	0.202	14:22																																																																																																																																																	
Air Blank	0.000	14:23																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.2023																																																																																																																																																		
Std Dev	0.0015																																																																																																																																																		
Rel Std Dev(%)	0.7550																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	13:33																																																																																																																																																	
Control Test	0.081	13:33																																																																																																																																																	
Air Blank	0.000	13:34																																																																																																																																																	
Control Test	0.081	13:34																																																																																																																																																	
Air Blank	0.000	13:34																																																																																																																																																	
Control Test	0.080	13:35																																																																																																																																																	
Air Blank	0.000	13:35																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0807																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7157																																																																																																																																																		

# Florida Department of Law Enforcement Alcohol Testing Program

## DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: HIALEAH PD  
Time of Inspection: 13:24

Date of Inspection: 05/07/2025

Serial Number: 80-000876  
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.051	0.081	0.202	0.080
0.000	0.050	0.081	0.202	0.079
0.000	0.051	0.081	0.202	0.079
0.000	0.050	0.081	0.203	0.079
0.000	0.051	0.081	0.202	0.079
0.000	0.050	0.081	0.202	0.080
0.000	0.051	0.081	0.201	0.079
0.000	0.051	0.081	0.202	0.079
0.000	0.051	0.081	0.202	0.079
0.000	0.051	0.081	0.202	0.079

Standard Deviations	0.0004	0.0000	0.0004	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0003 Number of Simulators Used: 5

Remarks:

The above instrument complies (  ) does not comply (  ) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.



TAYLOR D GUTSCHOW

Signature and Printed Name

05/07/2025  
Date



# Florida Department of Law Enforcement Alcohol Testing Program

## DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: HIALEAH PD  
Time of Inspection: 12:36

Date of Inspection: 05/16/2025

Serial Number: 80-000876  
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202406K Exp: 06/19/2026	0.08g/210L Test (g/210L) Lot#:202406L Exp: 06/19/2026	0.20g/210L Test (g/210L) Lot#:202406N Exp: 06/20/2026	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:AG429602 Exp: 10/22/2026
0.000	0.050	0.081	0.204	0.079
0.000	0.050	0.081	0.203	0.080
0.000	0.050	0.081	0.203	0.079
0.000	0.050	0.081	0.202	0.080
0.000	0.050	0.082	0.203	0.080
0.000	0.050	0.081	0.202	0.079
0.000	0.050	0.082	0.203	0.079
0.000	0.051	0.082	0.203	0.079
0.000	0.050	0.081	0.203	0.079
0.000	0.050	0.081	0.202	0.079

Standard Deviations	0.0003	0.0004	0.0006	0.0004
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0004 Number of Simulators Used: 5

Remarks:

The above instrument complies (  ) does not comply (  ) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

Taylor D Gutschow  
Signature and Printed Name

05/16/2025  
Date



# Calibration Certificate

Florida Department of Law Enforcement  
Alcohol Testing Program  
4700 Terminal Drive, Suite 1  
Ft. Myers, FL 33907

This is to certify the calibration of Intoxilyzer 8000 serial number 80-000876, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	<u>80-000876</u>	UNCERTAINTY* $\pm$	
Owning Agency:	<u>HIALEAH PD</u>	0.050 g/ 210 L	0.004
Calibration Date:	<u>05/16/2025</u>	0.080 g/ 210 L	0.004
Calibration Time:	<u>12:36</u>	0.200 g/ 210 L	0.007
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within  $\pm 0.005$  or 5%, whichever is greater, of the target alcohol concentration.

\*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence (k=3).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

## TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards. This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Taylor  
Gutschow  
Digitally signed by Taylor  
Gutschow  
Date: 2025.05.16 13:19:01  
-04'00'

05/16/2025

Date

TAYLOR D GUTSCHOW,

Department Inspector

FDLE/ATP Form 69 December 2021

Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality