

Alcohol Testing Program - Instrument Processing Sheet

Agency: PINELLAS PARK PD

Instrument Serial Number: 80-001042

Date In: 6/25/2026

DI Completion Date: 6/25/2026

☐ Ship ☒ P/U ☐ H/D ☐ CMI ☐ EE

Intake By: KTS Date: 6/25/2026		Quality Checks By: KTS Date: 6/25/2026		Flow Adjustment By:	
☒ Annual ☒ Dropped Off ☐ Registration ☐ Return from CMI / EE ☐ Training Instrument Visual Inspection ☒ Case ☒ Handle ☒ Keyboard ☒ Dry Gas Shelf ☒ Feet ☒ Breath Tube ☒ Ports ☒ Screws Tight Other Equipment/Accessories ☐ Power Cord ☐ Printer Cable ☐ Static Bag ☐ 12V DC Cable Notes: Missing screw on exhaust cover.		☒ Breath Tube Screen ☒ Replace External O-Rings ☒ Instrument Set Up Verified ☒ R-Value: 189 ☒ Flow Verification (L/s) Flow Column #: ATP103 32 mm 0.152 (.139-.169) 36 mm 0.167 (.156-.190) 53 mm 0.238 (.228-.278) 103 mm 0.488 (.447-.547) ☒ Barometric Pressure Check Gauge ID #: 28427 Gauge: 1013 Instrument: 1013 ☒ Stability Checks		Flow Column #: ☐ 5L/min – 17mm ☐ 15L/min – 53mm ☐ 30L/min – 103mm ☐ R-Value: ☐ Post Adjustment Verification (L/S) Flow Column #: 32 mm (.139-.169) 36 mm (.156-.190) 53 mm (.228-.278) 103 mm (.447-.547)	
		Simulator		Serial #	Lot#/Exp
		0.050		MP5088	202603A 3/3/2028
		0.080		MP5089	202603B 3/3/2028
		0.200		MP5090	202603D 3/4/2028
		0.080 DGS		N/A	23825080A1 10/5/2027
Optical Bench Adjustment		By:		Department Inspection	
Barometric Pressure Gauge: ID#:				By: KTS	
Simulator		Serial #	Lot #	Expiration	Gauge: 1013 Instrument: 1012
0.000			N/A	N/A	Mouth Alcohol Solution Lot #: 2025-D Exp: 9/25/2027
0.040					Acetone Stock Solution Lot #: 2025-B Exp: 9/22/2027
0.100					Simulator
0.200					Serial Number
0.300					0.000 MP5086
0.080 DGS		N/A			Interferent MP5087
☐ Post Optical Bench Adjustment Stability Checks					0.050 MP5088
Simulator		Serial #	Lot #	Expiration	0.080 MP5089
0.050					0.200 MP5090
0.080					Attachments
0.200					☒ Form 41
0.080 DGS		N/A			☒ Stability Checks
Gauge ID #:					☐ Post-Stability Checks
Gauge: Instrument:					☐ Flow Adjustment
Notes/Suggested Service:					☐ Form 40
					☐ Other:
					☒ Instrument Complies with Chapter 11D-8, FAC
					☐ Instrument Does Not Comply with Chapter 11D-8, FAC
					☒ Return to/Place into Evidentiary Use
					☐ Remain Out of Evidentiary Use
					☒ Conduct an Agency Inspection Before Evidentiary Use
					LeAndra Digitally signed by LeAndra Shayla Platt Digitally signed by Shayla Platt
					Higginbotham Date: 2026.06.25 13:56:37 -04'00' Date: 2026.06.25
					Tech Review Admin Review 14:16:15 -04'00'

Stability Checks

0.050 g/210L 0.047 to 0.053 g/210L	0.080 g/210L 0.077 to 0.083 g/210L	0.200 g/210L 0.194 to 0.206 g/210L	DGS 0.080 g/210L 0.077 to 0.083 g/210L ≤0.003 g/210L of Wet																																																																																																																																																
PINELLAS PARK PD Intoxilyzer - Alcotest Analyzer Model 8100 SN 80-001042 16/25/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>12:08</td></tr> <tr><td>Control Test</td><td>0.048</td><td>12:08</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:09</td></tr> <tr><td>Control Test</td><td>0.048</td><td>12:10</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:10</td></tr> <tr><td>Control Test</td><td>0.049</td><td>12:11</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:11</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0483</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>1.1945</td><td></td></tr> </tbody> </table> <i>Katz</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	12:08	Control Test	0.048	12:08	Air Blank	0.000	12:09	Control Test	0.048	12:10	Air Blank	0.000	12:10	Control Test	0.049	12:11	Air Blank	0.000	12:11	Control Test Stats			Average	0.0483		Std Dev	0.0006		Rel Std Dev(%)	1.1945		PINELLAS PARK PD Intoxilyzer - Alcotest Analyzer Model 8100 SN 80-001042 16/25/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>12:12</td></tr> <tr><td>Control Test</td><td>0.078</td><td>12:13</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:13</td></tr> <tr><td>Control Test</td><td>0.077</td><td>12:14</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:15</td></tr> <tr><td>Control Test</td><td>0.077</td><td>12:15</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:16</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0773</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7466</td><td></td></tr> </tbody> </table> <i>Katz</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	12:12	Control Test	0.078	12:13	Air Blank	0.000	12:13	Control Test	0.077	12:14	Air Blank	0.000	12:15	Control Test	0.077	12:15	Air Blank	0.000	12:16	Control Test Stats			Average	0.0773		Std Dev	0.0006		Rel Std Dev(%)	0.7466		PINELLAS PARK PD Intoxilyzer - Alcotest Analyzer Model 8100 SN 80-001042 16/25/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>12:17</td></tr> <tr><td>Control Test</td><td>0.197</td><td>12:17</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:18</td></tr> <tr><td>Control Test</td><td>0.196</td><td>12:18</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:19</td></tr> <tr><td>Control Test</td><td>0.196</td><td>12:19</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:20</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.1963</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.2941</td><td></td></tr> </tbody> </table> <i>Katz</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	12:17	Control Test	0.197	12:17	Air Blank	0.000	12:18	Control Test	0.196	12:18	Air Blank	0.000	12:19	Control Test	0.196	12:19	Air Blank	0.000	12:20	Control Test Stats			Average	0.1963		Std Dev	0.0006		Rel Std Dev(%)	0.2941		PINELLAS PARK PD Intoxilyzer - Alcotest Analyzer Model 8100 SN 80-001042 16/25/2026 Software: 8100.27 <table border="1"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td>Air Blank</td><td>0.000</td><td>12:04</td></tr> <tr><td>Control Test</td><td>0.079</td><td>12:05</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:05</td></tr> <tr><td>Control Test</td><td>0.078</td><td>12:05</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:06</td></tr> <tr><td>Control Test</td><td>0.079</td><td>12:06</td></tr> <tr><td>Air Blank</td><td>0.000</td><td>12:07</td></tr> <tr><td>Control Test Stats</td><td></td><td></td></tr> <tr><td>Average</td><td>0.0787</td><td></td></tr> <tr><td>Std Dev</td><td>0.0006</td><td></td></tr> <tr><td>Rel Std Dev(%)</td><td>0.7339</td><td></td></tr> </tbody> </table> <i>Katz</i> Operator's Signature	Test	g/210L	Time	Air Blank	0.000	12:04	Control Test	0.079	12:05	Air Blank	0.000	12:05	Control Test	0.078	12:05	Air Blank	0.000	12:06	Control Test	0.079	12:06	Air Blank	0.000	12:07	Control Test Stats			Average	0.0787		Std Dev	0.0006		Rel Std Dev(%)	0.7339	
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	12:08																																																																																																																																																	
Control Test	0.048	12:08																																																																																																																																																	
Air Blank	0.000	12:09																																																																																																																																																	
Control Test	0.048	12:10																																																																																																																																																	
Air Blank	0.000	12:10																																																																																																																																																	
Control Test	0.049	12:11																																																																																																																																																	
Air Blank	0.000	12:11																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0483																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	1.1945																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	12:12																																																																																																																																																	
Control Test	0.078	12:13																																																																																																																																																	
Air Blank	0.000	12:13																																																																																																																																																	
Control Test	0.077	12:14																																																																																																																																																	
Air Blank	0.000	12:15																																																																																																																																																	
Control Test	0.077	12:15																																																																																																																																																	
Air Blank	0.000	12:16																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0773																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7466																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	12:17																																																																																																																																																	
Control Test	0.197	12:17																																																																																																																																																	
Air Blank	0.000	12:18																																																																																																																																																	
Control Test	0.196	12:18																																																																																																																																																	
Air Blank	0.000	12:19																																																																																																																																																	
Control Test	0.196	12:19																																																																																																																																																	
Air Blank	0.000	12:20																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.1963																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.2941																																																																																																																																																		
Test	g/210L	Time																																																																																																																																																	
Air Blank	0.000	12:04																																																																																																																																																	
Control Test	0.079	12:05																																																																																																																																																	
Air Blank	0.000	12:05																																																																																																																																																	
Control Test	0.078	12:05																																																																																																																																																	
Air Blank	0.000	12:06																																																																																																																																																	
Control Test	0.079	12:06																																																																																																																																																	
Air Blank	0.000	12:07																																																																																																																																																	
Control Test Stats																																																																																																																																																			
Average	0.0787																																																																																																																																																		
Std Dev	0.0006																																																																																																																																																		
Rel Std Dev(%)	0.7339																																																																																																																																																		

Florida Department of Law Enforcement

Alcohol Testing Program

DEPARTMENT INSPECTION REPORT - INTOXILYZER 8000

Agency: PINELLAS PARK PD
Time of Inspection: 13:39

Date of Inspection: 06/25/2026

Serial Number: 80-001042
Software: 8100.27

Check or Test	YES	NO	Check or Test	YES	NO
Diagnostic Check (Pre-Inspection): OK	Yes		Date and/or Time Adjusted		No
Minimum Sample Volume Check: OK	Yes		Barometric Pressure Sensor Check: OK	Yes	
Alcohol Free Subject Test: 0.000	Yes		Mouth Alcohol Test: Slope Not Met	Yes	
Interferent Detect Test: Interferent Detect	Yes		Diagnostic Check (Post-Inspection): OK	Yes	

Alcohol Free Test (g/210L)	0.05g/210L Test (g/210L) Lot#:202603A Exp: 03/03/2028	0.08g/210L Test (g/210L) Lot#:202603B Exp: 03/03/2028	0.20g/210L Test (g/210L) Lot#:202603D Exp: 03/04/2028	0.08 g/210L Dry Gas Std Test (g/210L) Lot#:23825080A1 Exp: 10/05/2027
0.000	0.049	0.078	0.198	0.079
0.000	0.049	0.079	0.198	0.079
0.000	0.049	0.078	0.199	0.079
0.000	0.050	0.079	0.199	0.079
0.000	0.050	0.079	0.199	0.079
0.000	0.050	0.079	0.198	0.079
0.000	0.050	0.079	0.198	0.079
0.000	0.050	0.079	0.199	0.079
0.000	0.050	0.080	0.198	0.079
0.000	0.050	0.080	0.199	0.079

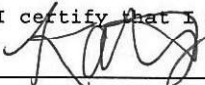
Standard Deviations	0.0004	0.0006	0.0005	0.0000
---------------------	--------	--------	--------	--------

Average Standard Deviation of 0.05, 0.08 and 0.20 g/210L Tests: 0.0003 Number of Simulators Used: 5

Remarks:

The above instrument complies (☒) does not comply (☐) with Chapter 11D-8, FAC.

I certify that I performed this inspection in accordance with the provisions of Chapter 11D-8, FAC.

 _____
Signature and Printed Name

KATIE T SPEARIN

06/25/2026
Date



Calibration Certificate

Florida Department of Law Enforcement
Alcohol Testing Program
2331 Phillips Road
Tallahassee, FL 32308

This is to certify the calibration of Intoxilyzer 8000 serial number 80-001042, manufactured by CMI, Inc. was calibrated in accordance with FDLE/ATP Form 36 - Department Inspection Procedures - Intoxilyzer 8000.

Serial Number:	80-001042	UNCERTAINTY* $\pm$	
Owning Agency:	PINELLAS PARK PD	0.050 g/ 210 L	0.004
Calibration Date:	06/25/2026	0.080 g/ 210 L	0.004
Calibration Time:	13:39	0.200 g/ 210 L	0.008
		0.080 g/ 210 L Dry Gas Control	0.005

All results are reported in g/ 210 L.

Bias is limited by calibration acceptance criteria. All calibration results must be within ± 0.005 or 5%, whichever is greater, of the target alcohol concentration.

*Uncertainty is based on fleet-wide data and is expressed to a 99.73% level of confidence ($k=3$).

The instrument results before and after any adjustment are found in the associated pre and post stability checks.

TRACEABILITY INFORMATION

This instrument was calibrated using solutions prepared by Alcohol Countermeasure Systems, Inc. (ACS). ACS prepared and certified these CRMs in accordance with ISO 17034 and ISO/ IEC 17025 Standards.

Simulator temperatures are traceable to NIST. Simulator temperatures are checked with NIST traceable digital thermometers calibrated by Precision Metrology in accordance with ISO/ IEC 17025 standards.

Dry gas control measurements are traceable to NIST through the use of CRMs supplied by an accredited CRM supplier. The supplier of dry gas standard controls prepared and certified the CRMs in accordance with ISO Guide 34 and ISO/ IEC 17025 standards.

This document shall not be reproduced except in full, without written approval of the Florida Department of Law Enforcement Alcohol Testing Program.

Date

06/25/2026

KATIE T SPEARIN,

Department Inspector

FDLE/ATP Form 69 January 2026

Issuing Authority: Alcohol Testing Program

Service • Integrity • Respect • Quality